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Smart Blood Pressure Temperature Watch User Guide

Thanking, I met you in the years like songs. Thank you for choosing this smart watch. This watch will provide higher help and enjoyment for your exercise and health. For your convenience, please read the instructions carefully and follow the steps in the instructions.

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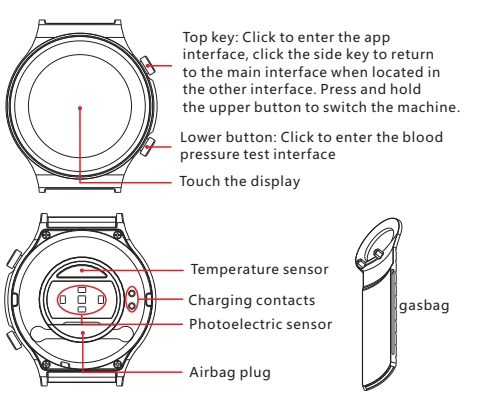
Quick description of the watch Part introduction

Top key: Click to enter the app interface, click the side key to return to the main interface when located in the other interface. Press and hold the upper button to switch the machine. Lower button: Click to enter the blood pressure test interface. Touch the display. Temperature sensor. Charging contacts. Photoelectric sensor. Airbag plug. gasbag.

Charging method

Charging activation is required before the first use of this watch, using the equipped magnetic suction charging cable to attach to the metal point on the back of the hand table, the other end of the charging line access to the USB charging head or computer USB port.

Install the watch APP

Scan the following QR codes or enter major application markets to download and install SmartHealth

Device requirements: iOS 9.0 and above; Android 4.4 and above, Bluetooth 4.0 support.

Open the mobile client to set up personal information

Switch to the "Device" page and click on the binding device

Click on your device in the list of scanned devices

Binding completed

The main feature page

Main interface/dial

In the main interface press and hold touch display or in the APP menu, click "theme selection" into the main interface/dial switch

Press the side button or swipe up in the main interface to enter the function page, then press the side button or swipe down to return to the main interface.

Step/Distance/Calories

View the number of steps you walk, the distance you walk, and calories you consume at any time of the day, and update to the APP simultaneously to view your motion data.

Blood pressure measurement

Accurate measurement

Before testing, prepare to remove the airbag plug from the back of the watch, install the airbag, and install the airbag as shown

Note: A poor airbag assembly will cause the measurement to fail

Bend the airbag, wear the watch correctly, loosely moderately, not too loose or too tight (watch placed on the back of the wrist, airbags naturally extend, cannot curl between the strap and wrist)

Inaccurate wearing will result in inaccurate blood pressure measurement!

Measurement precautions

The subjects sat comfortably, their legs were not crossed, their feet were flat, their backs, elbows, forearms were supported, their watches were as tall as their hearts, their hearts relaxed and they did not speak.

Note: Failure to observe the Measurement Precautions will result in inaccurate measurement results!

The measurement begins

This watch uses pressurized synchronous measurement technology to detect the measurer's blood pressure and pulse during the pressurization process, and complete the measurement at the same time as the pressure is over. When the air pressure value increases, the blood pressure watch detects the pulse and begins to calculate the pulse on its own.

Displays the measurement results

After the measurement is complete, the blood pressure watch automatically exhausts and then automatically displays the measurement results.

Note: When the airbag is over-inflated for long time and causes discomfort to the user, the user can untie the strap directly.

The cause and handling of the measurement failure

1. Make sure that the strap and airbag are worn correctly.
2. Confirm that the airbag and its connections are tight and that there is a leak.
3. Make sure the airbag is curled between the strap and the wrist.
4. Whether the arms and body remain stationary when measuring.
5. Keep quiet during the measurement and do not speak.
6. After the determination is complete, wait 3 minutes for re-measurement.

Breathing rate

Click to enter the precision measurement mode, click to start the measurement, follow the watch prompts to adjust the posture, waiting for the measurement. Please keep your watch as tall as your heart!

Note: The palms stretch, you can't hold your fist, your thumb is above.

Quick measurements

The photoelectric test method is quick and easy to test blood pressure, this data is for reference only. Requires greater accuracy! Please use precise measurements for blood pressure testing.

body temperature

Switch to the temperature test interface to access real-time temperature detection, monitoring data can be updated to the APP simultaneously, and there are test reports.

Note: When testing body temperature, the bottom temperature must be fully exposed to the skin (wear 10 minutes after wearing test). temperature test will be affected by environmental temperature difference, only serve as a warning reminder, please take medical professional equipment and doctor diagnosis

heart rate

Switch to the heart rate test interface for real-time heart rate detection, monitoring data can be updated to the APP simultaneously, and test reports are available.

Blood oxygen

Switch to the blood oxygen test interface to enter the blood oxygen real-time detection, monitoring data can be updated to the APP simultaneously, and there are test reports.

Weather

The weather page displays information about the current weather and air quality. Weather information needs to be connected to the client before it can be obtained, and weather information cannot be updated if it is disconnected for a long time.

Note: The watch is not connected to the APP before there is no weather interface, need to connect app after the weather interface, the phone needs to turn on positioning.

sleep

When you fall asleep, your watch automatically determines your sleep detection mode, automatically detects your sleep/light sleep/sleep duration throughout the night, calculates your sleep quality, and your sleep data can be updated to the APP simultaneously.

Note: Sleep data is only available if you fall asleep with your watch.

stopwatch

Click on the stopwatch to enter the stopwatch function and slide right to exit.

Information

Click on the information to view the content of the information, slide up and turn the page. Slide right to exit.

camera

In the connection state, click on the camera to remotely control the phone to take pictures, click to take a picture, right slip exit.

Music

In the connection state, click music to control the music player on your phone.

concerning

Click on the About icon to display the machine model and Bluetooth address in the last four digits.

brightness

Click on the brightness icon to freely select the screen brightness.

Raise your wrist

Click on the wrist lift icon to enter and select the wrist lift to turn on or off.

reposition

Click on the reset icon to enter and restore the factory settings.

Shutdown

Click the shutdown icon to enter and you can shut down.

Other feature reminders

The reminder function needs to set on the alert switch on the APP side and keep the phone and watch connected successfully in Bluetooth, the message is a vibration alert.

Precautions

- 1: Do not use an adapter with a charging voltage greater than 5V and a charging current greater than 2A for 2-3 hours
- 2: Do not charge in the event of water stains.
- 3: This product is an electronic monitoring product, not as a medical reference, measurement data for reference only.
- 4: Keep the surface of the blood pressure watch clean and tidy, which helps to extend the life of the blood pressure watch.
- 5: This product does not have waterproof performance, please do not wear for immersion, swimming, bathing and a series of may be the inside of the watch into liquid or water vapor operation.

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