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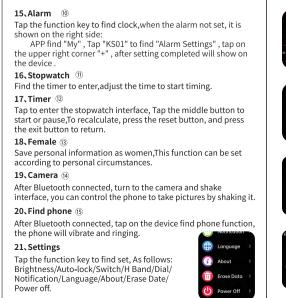
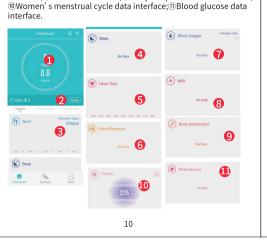
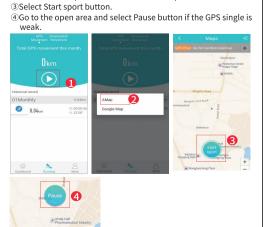
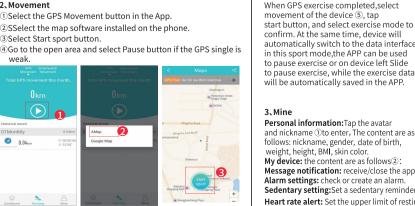
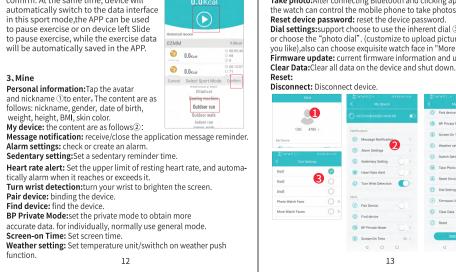
①Select the GPS Movement button in the App.



Switch Setting: HR Automatic monitoring/BP Automatic

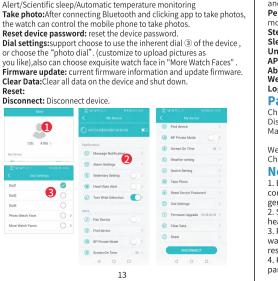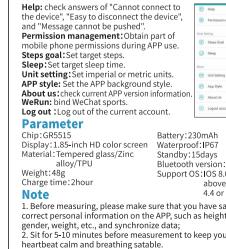nonitoring/Blood Oxygen Nighttime monitoring/Disconnect

Battery: 230mAh Bluetooth version: 5.1 Support OS:IOS 8.0 or above Android heartbeat calm and breathing satable.
3. Please keep relaxed during measurement, panting, talking

1. Before measuring, please make sure that you have saved correct personal information on the APP, such as height, age, gender, weight, etc., and synchronize data; 2. Sit for 5-10 minutes before measurement to keep your

walking and exercise will affect the accuracy of the measurement 4. Keep a correct sitting posture, make the hands flat and keep it parallel with heart.

## Wear and maintenance

The products are suitable for wear all day, so when you wear and care for the smart device, please follow the following brief

1. Keep the smart device dry: 1.1. Clean your wrist and the smart band/smart watch you are wearing regular, especially after exercise, sweating, or skin contact with soap or detergent and other substances, the substance may get inside of the product so should be cleaned regularly.

2.1. Please do not wear the device too tightly, should be able to move back and forth on the wrist.

2.2. If use a heart rate device to record your heart rate throughout a day, the device should be flat one finger wide among the wrist bone

In order to better check the data during active exercises, wear the device among the wrist bone about 2-3 Finger-width position. 2.3, Prolonged friction and restraint may irritate your skin, after

a long time wearing, take off the device and give skin a rest period. 3. Keep smart devices clean: 3.1. If device is damp, such as sweating or splashing, please clean

and dry it, make sure the skin layer is in a suitable dry humidity, and The data collected by this device is for reference only, not for medical purpose