

O2Vent Optima[®]



Replacing Connector Bands - Instructions

The connector bands (bands) are essential for keeping your O2Vent Optima device assembled and to maintain the lower jaw position.

The bands are available in nine lengths (13mm-21mm) and two strengths: 'soft' (light blue) and 'firm' (dark blue).

When should I replace the bands?

- It is recommended to change both bands monthly as they may lose elasticity.

How to identify that the bands need changing

- If the length of the current band is longer than a new band
- If the band's holes have stretched from circular to oval shape

Replacing connector bands on your O2Vent Optima

Step 1 – Separate the upper and lower device trays, as shown.

Step 2 – To remove, peel the band away from the wide end of the lug. Repeat on the second side.

Step 3 – Add the new band with the size/number facing out, by hooking the band opening over the narrow end of the lug and pulling it over the wide end.



To remove, peel the band away from the wide end of the lug.

Step 3 continued – Repeat this action with the other end of the band to connect the upper and lower trays.

Step 4 – Repeat step 3 with the second band.



To add band, hook over narrow end of the lug and pull over the wide end.

RX ONLY

CONNECTOR BANDS CAN ONLY BE USED IN CONJUNCTION WITH **O2VENT OPTIMA** DEVICES



MANUFACTURER AND AUSTRALIAN SPONSOR:

**Open Airway Dental Solutions
Australia Pty Ltd
5 Moorak Street, Taringa QLD 4068
Australia**

www.openairway.au

info@openairway.com

Ph: +61-1300-533-159

MANUFACTURED FOR:

**Open Airway Dental Solutions Ltd,
Canada**

www.openairway.ca

info@openairway.ca

Ph: +1-866-835-977-8267



Consult instructions for use.



US Federal Law restricts this device to sale by or on the order of a healthcare professional.



O2Vent
Optima[®]

R130-03