



## Gutta Percha IFU

### **INDICATION AND PURPOSE**

1. Gutta percha points are used during the obturation process in root canal therapy.

### **INSTRUCTIONS FOR USE**

1. Prepare root canal by using preferred instrumentation.
2. After cleaning and drying the canal, apply the sealer directly to cone or into the canal.
3. Select the appropriately sized cone for obturation
4. Irrigate a gutta percha point with 3% NaOCl disinfecting solution.
5. Insert master point into the root canal to working length.

### **CAUTIONS**

1. Do not use this product for any patients with a history of latex allergy.

### **STORAGE**

Store at room temperature out of sunlight.

### **SHELF LIFE**

3 years from manufacture date