



## Paper Points IFU

### **INDICATION AND PURPOSE**

1. Paper points are used to dry the root canal before obturation.

### **INSTRUCTION FOR USE**

1. Measure the length of root canal.
2. Dry the root canal using paper points.
3. Sealer or cement can be applied by using paper points or a hand instrument.

### **CAUTIONS**

1. Use caution not to extrude debris or sealer beyond the apex
2. After using the paper points, unused should be kept in a cool/dry place

### **STORAGE**

Store at room temperature

### **SHELF LIFE**

4 years from manufacture date