

MiniBuddyPro warranty information:

General warranty: One Year limited warranty from date of purchase. ALDS warrants the **MiniBuddyPro** amplifier against defects in materials and/or workmanship for a period of 12 months from the date of purchase.

Accessories such as the following have a 6 month warranty from date of purchase as a kit with the **MiniBuddyPro**:
Microphones, charge cables, charger (if supplied with the original sale) belt clip and carry case.

Exclusions: The warranty does not cover damage to or failure of the unit or accessories due to heat, moisture, shock, misuse, poor maintenance, improper charging, unusual wear and tear or tampering.

Warranty repair or replacement: In the event of failure, the **MiniBuddyPro** and/or accessories will be repaired or replaced at the manufacturers discretion.

Warranty Procedure:

All warranties are handled directly by the manufacturer. To initiate a warranty claim, the procedure is as follows:
Contact ALDS warranty department on +1.866.845.2537 or email info@alds.com to receive a returns goods authorization number. Send the item(s) back to ALDS with your supplied returns goods authorization along with a copy of the original bill of purchase. Include with the package a brief description of the problem. The item(s) must be sent back to the warranty depot freight pre-paid and in a safely packaged method. It is advisable to use a shipping method which provides you with a tracking number of your shipment. Include your full contact information such as name, phone number, email, preferred method of contact and return shipping address.

Please Note: Any items returned to the warranty repair depot without a returns goods authorization will be refused. Any items returned collect will be refused. Any items returned without a copy of the original bill of purchase will be refused.

Your MiniBuddyPro was distributed by:

Special Needs Computer Solutions
300 Welland Ave., Unit 2B
St. Catharines, ON L2R 7L9 Canada

1-877-724-4922 | 1-905-641-4922

MiniBuddyPro



**SPECIAL NEEDS
COMPUTERS**
1-877-724-4922

ALDS
MiniBuddyPro

Instruction Manual


Congratulations on your purchase of an **ALDS MiniBuddyPRO**

The **ALDS MiniBuddyPro voice amplifier** has been designed for those with temporary or permanent voice impairment. When using the **MiniBuddyPro**, a comfortable speech level can be achieved for effective everyday communications while reducing vocal stress and strain.

The **MiniBuddyPro** features:


- Patented stylish design and appearance
- Ergonomic & comfortable design
- Lithium light weight rechargeable battery
- Extended battery life
- Powerful double magnetic trumpet for cleaner penetrating sound
- 10 watts of powerful output with excellent sound coverage
- Superior MP3 audio encoding technology supporting TF card & U-flash drive disk

Wireless Microphone information:

- Put the headset microphone on in a way that feels most comfortable on your head with the boom microphone coming down along the side of your face towards your mouth.
- Turn the amplifier power on and then turn on the microphone power on
- The microphone and amplifier will automatically pair with each other
- If the microphone needs to be paired manually to the amplifier; make sure the amplifier is on, press the M key on the amplifier for 5 seconds or until the blue indicator light blinks, then turn on the microphone and press the  (located on top of the microphone transmitter body just above the volume buttons) for 5 seconds. The blue icon on the voice amplifier will stop flashing when the microphone has successfully paired with the amplifier.

UHF Wireless Microphone:

Note: DO NOT press and hold the + button or you will unpair the mic from the amp!

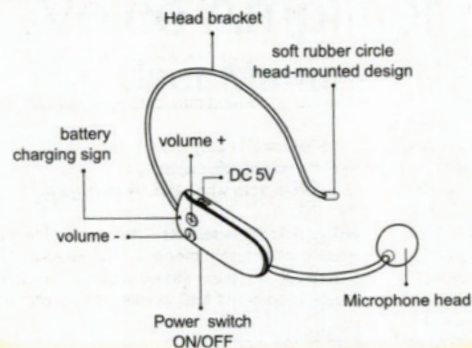

1. **Power switch & volume control:** Rotate the power wheel to turn on or adjust the amplifier volume. When the amplifier is turned on the blue light (8) will be continuously on.
2. **Mode:** short press: Short press the mode button to select a user mode (USB, TF). When a USB or TF card is inserted, press the mode button to cycle to the storage card you wish to play.
3. **Play/Pause:** A short press of the play/pause button will play or pause the sound effects from the USB or TF cards. A long press of the button till the blue light (8) blinks quickly will cause the playback to continuous cycle the current sound track.
4. **Previous:** A short press of the button to go to previous sound effects or a long press to fast backwards to initial starting point.
5. **Next:** Short press of the button to go to next sound effect track. A long press will enable fast forwarding.
6. **TF card slot:** Insert a TF card to play MP3 files. The **MiniBuddyPro** is compatible with most brands to a max capacity of 32GB.
7. **3.5MM microphone jack port** (For use with a wired microphone. Wired microphone not included with kit, but can be purchased separately)
8. **Indicator light:** Blue light will be on all the time in the state of power on and in use. Red Light will blink when charging the unit. The light will be solid red when **MiniBuddyPro** is fully charged
9. **USB port:** Insert a USB to play MP3 files, The **MiniBuddyPro** is compatible with most brands to a max capacity of 32GB
10. **Mini-USB charge cable port (DC 5V):** Use only the supplied charge cable for charging the **MiniBuddyPro**. IMPORTANT – connect the charge cable with a compatible USB charger of DC 5V 9 500mA – 1.5A) charger. Unplug the **MiniBuddyPro** as soon as it is fully charged to maintain safety and battery integrity.
11. **Detachable belt clip**

