Manual

Gewa Control 10

SE: IR-sändare

GB: IR transmitter

NO: IR-sender

Art.no 425710

ABILIA

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GB: English Manual Gewa Control 10

Introduction

The Gewa Control 10 is a programmable IR transmitter, which replaces ordinary remote controls for TV, DVD, stereo etc. It has in all 36 different programming positions on four levels. Simply press the buttons to send signals, and change level by pressing the *Level*-button.

The Control 10 is a part of the “Gewa Smart House” concept and therefore can be used together with all of these products.

News and improvements

Control 10 is improved in several ways compared to the earlier generation IR-9SP:

- New design on casing

- Improved battery warning

- Improved labelling of the buttons

- Place for level indication has been moved

General information about Control 10

To be able to use your Control 10 there are certain things that you should know:

The Control 10 is delivered preset with GewaLink channels, meaning that it is possible to use the transmitter immediately after the batteries has been inserted. See section *Inserting Batteries*.

To enter the programming mode, press the Programming button using a paper clip, for example, **at the same time** as you choose a *Function*-button. Different *Function*-buttons are chosen depending on what is to be programmed.

If you press the wrong button or make a mistake when programming you can always press the Programming button for two seconds. It works as a *Reset*-button.

It is important not to wait too long between the different steps when programming. After a certain time Control 10 automatically leaves the programming state to save power.

If a button on your Control 10 is pressed for more than 25 seconds, your Control 10 will stop transmitting to save power.

The settings can be protected by using the Settings Lock.

You can only change the settings of levels that are unlocked.

This picture shows the various terms and functions which are described in the text.

Fig 1

Symbols

The following symbols appear in the text throughout this manual:

- P** Programming button
- X** and **Y** Any button on your Control 10
- Y** Any button on another transmitter to be programmed
- 1** - **8** Button 1-8 on Control 10
- !** Code-button
- Level-button
- 2 sec Level-button held pressed for approximately 2 seconds

Inserting Batteries

Unscrew the battery lid screw on the back of Control 10 using a screwdriver and insert four alkaline LR03 (AAA) 1.5 V batteries. Insert the batteries according to the marking at the bottom of the battery compartment. Pay attention to the polarity! It is important to use long life **alkaline** batteries that are protected against leaking battery acid.

When you have inserted the new batteries, **press P**.

Battery Warning Indicator

Control 10 has a built-in battery warning ♪♪♪ indicating low battery voltage. A sound signal with short beeps is heard at an interval of 30 seconds. After a while it starts warning every ten minutes. Replace the batteries as soon as possible.

Settings

Below is described how to program your Control 10 for different setting options.

Levels in Control 10

Your Control 10 has totally four levels with 9 programmable buttons. On delivery the buttons are factory-programmed with GewaLink channels 00-31 on level 1-4. You can freely use one, several or all four levels. Four indication lamps indicate the current level (see Fig 1).

Your Control 10 is delivered with only level 3 open, other levels are closed. To open further levels, see section *Opening/closing a level*.

When only one level is open, the *Level*-button  automatically contains one channel which is always is the first channel on the next level. In this way, it is possible to control 10 functions even if only one level is open. See ¹ in *Table 1*.

When several levels are active you can change to another level by pressing the *Level*-button . Each time you press the button the next level is chosen.

Table 1: Default setting Gewa Link channels

Table 1 shows the channels and codes for Control 10 on delivery.

GewaLink channels position				
Button on Control 10	Level 1	Level 2	Level 3	Level 4
	00	08	16	24
	01	09	17	25
	02	10	18	26
	03	11	19	27
	04	12	20	28
	05	13	21	29
	06	14	22	30
	07	15	23	31
¹ 	08	16	24	00
	Code			

¹ If only one level is open the *Level*-button automatically becomes one channel. It is the first channel on the next level.

Locking/Unlocking the Settings Lock

In order to protect channels/codes from being erased by accident when new codes are programmed it is possible to lock each level separately. All levels are unlocked on delivery. A locked level is indicated by a beep ♪♪♪ when you try to program it.

Do as follows to lock/unlock the Settings Lock:

1. Press **P** and **---** at the same time. The indication lamp for the current level is now lit.
2. Change to the level to be locked/unlocked by pressing **---**.
3. Press button **3** to lock or button **4** to unlock.
The indication lamp for the levels is automatically moved to the next level, which thereby can be locked/unlocked immediately.
4. Press **---**^{2sec} for approximately 2 seconds until the indication lamp for programming flashes green and goes out.

Locking/Unlocking the Settings Lock

Press **P** + **---**, choose level with **---**, then press your desired option **3** or **4** then **---**^{2sec}.

3 = Lock. **4** = Unlock.

Default setting: Unlocked

Opening/Closing a Level

Your Control 10 has four different levels, which means that each button can have four different functions. You can select the number of levels that are open and hence, how many functions each button should have. You should close levels that are not to be used.

1. Press **P** and **---** at the same time. The indication lamp for the current level is now lit.
2. Press **---** a number of times until the level to be opened or closed is reached.
3. Press **1** to open or **2** to close the chosen level.
The level indication lamp is automatically moved to the next level, which thereby directly can be opened or closed.
4. Press **---**^{2sec} for approximately 2 seconds until the indication lamp for programming flashes green and goes out.

Opening/Closing a Level

Press **P** + **---**, choose level with **---**, then press the desired option **1** or **2**, then **---**^{2sec}.

1 = Open. **2** = Close.

Default setting: Level 3 open

Recording IR Channels from another Transmitter

Make sure the batteries in both transmitters are fully charged. Do not use the transmitters in intense light (e.g., sunlight) when programming.

Place the two transmitters on a table with the original transmitter directed towards the rear of your Control 10, see picture.

1. Choose the level where the channels are to be recorded with . Only one level at a time can be programmed.

2. Press **P** and **1** at the same time. The indication lamp on button 1 and the present level now flashes.


To check that the transmitters are in the right position, keep any button pressed on the original transmitter. The indication lamp for the *Level*-button now flashes green. Move the original transmitter to the left or to the right until the indication lamp goes out. Then move the original transmitter backwards until the indication lamp goes out (one meter for certain transmitters). Place the original transmitter centrally between these positions. Certain types of transmitters only emit a short flash each time the button is pressed. In these cases, press several times.

3. Press and release the button **X** on Control 10, which is to be programmed. The indication lamp for the *Level*-button now emits a steady orange light.

4. Keep the corresponding button **Y** on the original transmitter pressed. Release the button when the indication lamp for the *Level*-button goes green. It might take up to 10 seconds.

If the indication lamp goes orange again press the button one or more times until the light flashes green and goes out.

If the indication lamp flashes red the programming has failed. Try once more from step 2.
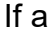
If a beep is heard  the level is locked and no programming can be made. See section *Locking/Unlocking the Settings Lock* how to unlock a level.

5. If the programming of the first button was successful, repeat steps 3 and 4 until all buttons are programmed. Finish by pressing .
6. The indication lamp for programming flashes green to indicate that the programming is complete.
7. Now test the programmed functions on the device which is to be controlled.

Original transmitter

Recording IR Channels

Press **P** + **1**, select button **X**, select button **Y**, then continue or finish with .

Important! When just **one level** is used, the *Level*-button automatically becomes the next levels first button, see *Table 1*. It means that it is possible to record an IR channel on the *Level*-button, but only when just one level is being used.

Tip! Control 10 can record codes from most IR transmitters on the market with a few exceptions. You may have problems with a few or all of the functions for a certain IR transmitter. In these cases it is possible to try another recording method. Instead, do as follows in step 2; Press **P** and **1** at the same time but keep them pressed for approximately 2 seconds until the indication lamp for programming flashes green and goes out. Control 10 is now in another recording mode. Otherwise use the same recording procedure as above.

Adding a 4096-code


Control 10 is delivered with a special coded channel (4096) on the *Code*-button **!**. The code is selected randomly. This coded channel is normally used for functions that require strict authorization such as apartment or house doors. The code has 4096 different combinations and has always the same code irrespective of which level you use. If you have to program a new coded channel there are two methods of doing so.

Method one: record the code from another transmitter, as described under *Recording IR channels from another Transmitter*. By this method it's possible to program a coded channel **on any button independently of level**.

Method two: Enter a binary code. The binary code will always be associated with the *Code*-button **!** and it is **the same for all levels**. Make up your own binary code or use the code already programmed in an IR receiver or transmitter.

There are two types of IR receivers: an older type of receiver where you set the code with a 12-pole switch, or a newer type where the IR receiver is programmable. The principle is the same when it comes to creating the binary code. *Fig 3* shows a 12-pole switch. Switches 1, 4, 6 and 12 are in the “On” position, the others are in the “Off” position.

Fig 3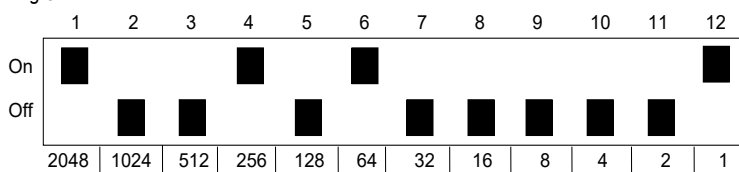

The following example describes how the binary code is created.

1. Press **P** and **!** at the same time. The indication lamp for the current level are now lit.
2. Enter the code by pressing on **!** or **!**.
! = switch in Off position and **!** = switch in On position.
 Press twelve times in turn; **!**, **!**, **!**, **!**, **!**, **!**, **!**, **!**, **!**, **!**, **!**, **!**, (Fig 3).
3. When you have pressed twelve times the indication lamp for programming flashes green to show that the operation is finished.

The code is now programmed on the **!**-button. Make a note of the chosen code for future usage.

Adding a 4096 code

Press **P** + **!**, press **!** or **!** 12 times.

! = Off position. **!** = On position.

Default setting: Random code

Copying a button

Using your Control 10, it is possible to copy a buttons' function to another button.

NOTE! The code/channel remains in its original place.

1. Choose level and press the button **X** with the code to be copied.
2. Choose the level to which you want to copy the code.
3. Press **P** and **!** at the same time. The indication lamp for the current level are now lit.
4. Press **3**.
5. Press the button **Y** to which you want to copy the code.
6. The indication lamp for programming flashes green to show that the operation is complete.

Copying a button

Copying code/channel from button **X** to button **Y**:

Choose level, choose **X**, choose level, **P** + **!**, **3**, choose **Y**.

Setting Beep Volume

Control 10 has a beep signal which sounds when a button is pressed. Set the beep volume in 6 stages (**---**, **1**, **2**, **3**, **4**, **!**).

--- = no sound at all and **!** = maximum sound level.

Do as follows:

1. Press **P** and **4** at the same time. The indication lamp for the current level are now lit.
2. Press a button, **---**, **1**, **2**, **3**, **4**, **!** to choose beep volume.
3. The indication lamp for programming flashes green to show that the operation is complete.

Setting Beep Volume

Press **P** + **4**, choose the setting.

--- = no sound at all and **!** = maximum sound level.

Default setting: Beep Volume = 3 (0 – 5)

Setting an acceptance delay

Control 10 makes it possible to set an acceptance delay. By acceptance delay is meant the time a button has to be pressed before a result is reached. The acceptance delay can be set in 6 steps (---, 1, 2, 3, 4, !).

--- = no delay time and ! = maximum delay time (0.5 second).

Do as follows:

1. Press **P** and **2** at the same time. The indication lamp for the current level are now lit.
2. Press a button, ---, 1, 2, 3, 4, ! to choose delay time.
3. The indication lamp for programming flashes green to show that the operation is complete.

Setting an acceptance delay

Press **P** + **2**, choose setting.

--- = no delay time and ! = maximum delay time (0.5 second).

Default setting: Acceptance delay = OFF (0 sec).

Creating an automatic sequence (Macro)

Your Control 10 is able to send sequences of codes. When you press **one** button, Control 10 automatically carries out what is equivalent to pressing several buttons. One available application for instance is muting the TV at the same time as you answer the telephone. Your Control 10 can also be programmed to change levels and it remembers how long a button should transmit. The transmission of a sequence can be terminated at any time by pressing another optional button. When a sequence is programmed your Control 10 transmits at the same time what is already on the button. Max. 25 seconds and 48 button presses per sequence can be programmed.

NOTE! The button used to store the sequence must not be part of the sequence.

1. Press **P** and **!** at the same time. The indication lamp for the current level are now lit.
2. Press **1**.
3. Press the buttons which should be used for the sequence.
4. Choose the level where you want the sequence to be.
5. Press **P** and **!** at the same time. The indication lamp for level B and the current level are now lit.
6. Press **2**.
7. Choose the button **X** where you want the sequence to be stored.
8. The indication lamp for programming flashes green to indicate that the operation is complete.

Creating an automatic sequence (Macro)


Press **P** + **!**, **1**, create the sequence, choose level, **P** + **!**, **2**, choose button **X**.

Max 25 seconds per press, 48 presses

Direct Choice of Level

Control 10 makes it possible to create quick buttons which means that you can reach a level by pressing a selected button. If you are at level 1 for instance and press button 2 you will directly reach level 2. And by pressing button 3 you will directly reach level 3 etc. When you are at level 2, 3 or 4 you can press a new quick button to get back to level 1. You need to create the quick buttons for these levels in advance. See Fig. 4.

Fig 4:

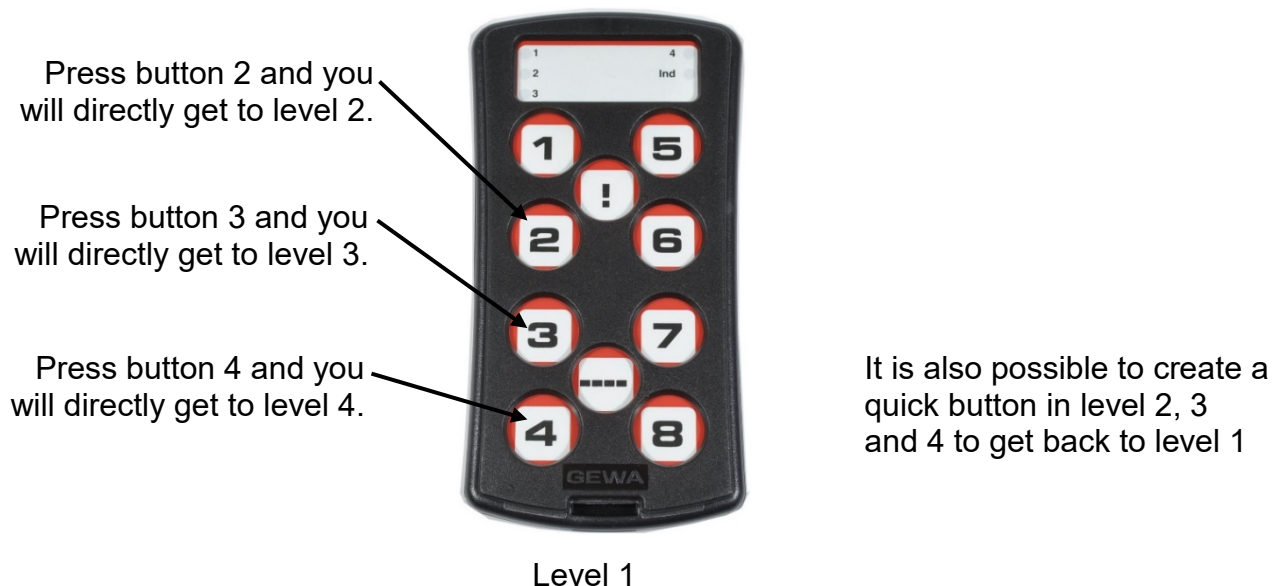Do as follows to create quick buttons:

1. Choose the level you want to reach with a quick button.
2. Press **P** and **!** at the same time. The indication lamp for the current level are now lit.
3. Press **1**.
4. Choose the level where the quick button is to be stored.
5. Press **P** and **!** at the same time. The indication lamp for the current level are now lit.
6. Press **2**.
7. Press the button **X** which is to be used as a quick button.
8. The indication lamp for programming flashes green to indicate that the operation is complete.

Direct Choice of Level

Create a quick button:

Choose level to be reached through direct choice, **P** + **!**, **1**, choose level where the quick button is to be stored, **P** + **!**, **2**, choose button **X** which is going to be a quick button.

Setting the channel range

Your Control 10 has the option to set different channel ranges for the transmitter to operate within. When the change of channel range is applied, programmed channels will disappear and a new random code is added on the *Code*-button .

Normally, the channel range does not need to be changed. If several GewaLink transmitters are to be used in the same area and you do NOT want to control the same equipment, it may be necessary to use different channel ranges.

Table 2 shows the different settings.

Table 2: Channel range

	Channel range, GewaLink channels		
	00-31 	32-63 	64-95 
Level 1	00-07	32-39	64-71
Level 2	08-15	40-47	72-79
Level 3	16-23	48-55	80-87
Level 4	24-31	¹ 56-63	88-95

¹ Channel 62 is a stop code and cannot control any function. On the other hand you can store channels/codes on this button which in turn can control a function.

Do as follows to set the channel range:

1. Press  and  at the same time. The indication lamp for the current level are now lit.
2. Press .
3. Choose channel range ,  or  according to table 2 above.
4. The indication lamp for programming flashes green to indicate that the operation is complete.

Setting the channel range

Press  + , , choose the channel range.

 = 00-31,  = 32-63,  = 64-95

Default setting: GewaLink channel 00-31


Button Labelling

The buttons on your Control 10 are marked with numbers as standard. The numbered label can be replaced with a personalised design. On Abilia's website it is possible to create your own button labels using pre-designed templates with symbols and text.

The document can also be downloaded. The labeling is printed on a clear sheet or white paper and inserted through the slit on your Control 10. Before the button label is inserted, the keyguard must be removed.

A plastic leg at each corner holds the keyguard. Put a knife blade between the keyguard and Control 10. Remove the keyguard carefully and smoothly, otherwise the plastic legs may break.

Two A4 sheet for making notes is included with your Control 10.

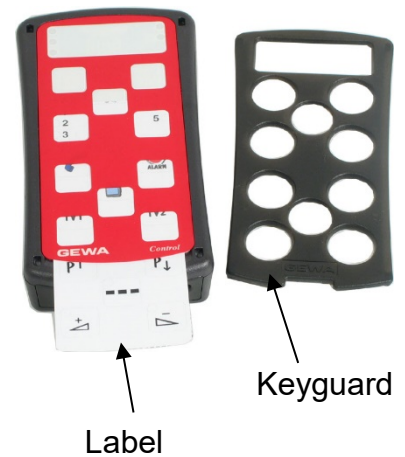Control 10 Holders

Rubber feet are mounted on the underside so that your Control 10 can lie still on flat surfaces. In some cases Control 10 needs to be permanently mounted. As an additional accessory, there is a holder, which can be secured to Control 10. This holder can in its turn be mounted on a table with a 'G' clamp or attached to a tube with a 'U' clamp. The holder can also be mounted on a mobile stand.

Another option is to hang your Control 10 around the neck. A neck strap is supplied with your Control 10.

Warning!

The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap!

Cleaning

Your Control 10 can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

Warning!

Do not use any disinfectant except surgical spirit, Chlorhexidine or ethyl alcohol.

These liquids are inflammable, handle with care!

Default settings in your Control 10

On delivery your Control 10 will be set with certain default settings as follows:

Acceptance delay is set to value 0

Feedback Beep Volume is set to value 3 (0 – 5)

Only level 3 is open, other levels are closed

GewaLink channels 00-31 are on level 1-4

A random code on the -button

Resetting a Button or a Level

Control 10 makes it possible to reset either one button or all buttons on a level. When this is done the button or buttons return to earlier chosen channel setting. Other settings are not affected. The level where the button/s is/are placed must be unlocked before resetting is possible. Read the section *Locking/Unlocking the Settings Lock*.

Do as follows to reset:

1. Unlock the level where the button/s is to be reset. Use the *Level*-button to choose the level for which you want to perform the reset operation.
2. Press and at the same time. The indication lamp for the current level are now lit.
- 3a. Resetting a **single button**.
Press the button to reset.
- 3b. Resetting all buttons for a single level.
Press in turn the buttons , , , , .
4. The indication lamp for programming flashes green to show that the operation is complete.

Resetting to Factory Setting

Control 10 makes it possible to reset all buttons on levels 1-4 at the same time. All buttons are returned to the factory settings which means that they are replaced by the GewaLink channels 00-31. Other settings will also return to default settings, see above.

1. Press and at the same time. The indication lamp for current level are now lit.
2. Press in turn the buttons , , , , .
3. The indication lamps for levels 1-4 flashes red in turn. The indication lamp for the *Level*-button flashes green to show that the operation is complete.

Resetting to default setting

To reset one button:

Choose level, **P** + **3**, choose button **X**.

To reset one level:

Choose level, **P** + **3**, **---**, **1**, **2**, **3**, **---**.

To reset to Factory Settings:

Press **P** + **3**, **!**, **1**, **2**, **3**, **---**.

Technical data

Technical data

Power consumption at rest:	Approx. 12 μ A
Power consumption at transmitting:	Approx. 35 mA (GewaLink channel)
Battery life:	Approx. 2.5 years (100 seconds transmitting per day with GewaLink channel)
Programming locations:	36 places
Operating force buttons:	< 160g
Range:	20-30m (GewaLink channels)
EMC-standard emission:	EN 61000-6-4:2001, EN 60601-1-1-2:2001
” immunity:	EN 61000-6-2:2001, EN 60601-1-1-2:2001
Measurements (L x W x H):	134 x 70 x 28 mm (incl. rubber feet)
Weight (incl. batteries):	180g
Operating temperature (excl. batteries):	-20°C to +45°C

Accessories included with your Control 10

Art.no	Name	Description	Qty
425715	Keyboard Label	For labeling of buttons	1 pcs
-	Note sheet	A4-sheet for noting your programmed channels	2 psc
425712	Control 10 Keyguard	Keyguard for 10 buttons	1 pcs
890410	Neck strap		1 pcs
8375	Battery	LR03 1.5V	4 pcs

Other accessories available for Control 10

Art.no	Name	Description
8105	Holder	To connect Control 10 to a mounting system

Safety and responsibility

Abilia has conducted a risk analysis for the product with reference to SS-EN 14971 for Medical equipment. The instructions for use and the technical specifications for the product have been drawn up in respect of the safety assessments carried out in this document.

It is always important to be aware of safety when first using this product and with its subsequent use. It is not possible to describe in the instructions all the situations in which the product must not be used. If you are not sure whether you have started using the product properly or set it up correctly, please contact your supplier.

Abilia is not liable for any loss, damage or consequential loss resulting from any use, correct or otherwise, lack of follow-up, incorrect set-up, incorrect connection or similar misuse. Abilia financial liability is at all times limited to the value of the product itself.

Recycling Waste Electrical and Electronic Equipment

The Control 10 contains electronics components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

Conformity with the Medical Devices Directive

The Control 10 is marketed as a technical aid for people with disabilities and meets the requirements in the Medical Devices Directive 93/42/EEC including relevant EMC and electrical safety requirements.

Summary and Quick Guide Gewa Control 10

Locking/Unlocking the Settings Lock

Press **P** + **---**, choose level with **---**, then press your desired option **3** or **4** then **---** 2sec.

3 = Lock. **4** = Unlock.

Default setting: Unlocked

Opening/Closing a Level

Press **P** + **---**, choose level with **---**, then press the desired option **1** or **2**, then **---** 2sec.

1 = Open. **2** = Close.

Default setting: Level 3 open

Recording IR Channels

Press **P** + **1**, select button **X**, select button **Y**, then continue or finish with **---**.

Adding a 4096 code

Press **P** + **!**, press **---** or **!** 12 times.

--- = Off position. **!** = On position.

Default setting: Random code

Copying a button

Copying code/channel from button **X** to button **Y**:

Choose level, choose **X**, choose level, **P** + **!**, **3**, choose **Y**.

Setting Beep Volume

P + **4**, choose the setting.

--- = no sound at all and **!** = maximum sound level.

Default setting: Beep Volume = 3 (0 – 5)

Setting an acceptance delay

P + **2**, choose setting.

--- = no delay time and **!** = maximum delay time (0.5 second).

Default setting: Acceptance delay = OFF (0 sec).

Creating an automatic sequence (Macro)

Press **P** + **!**, **1**, create the sequence, choose level, **P** + **!**, **2**, choose button **X**.

Max 25 seconds per press, 48 presses

Direct Choice of Level

Create a quick button:

Choose level to be reached through direct choice, **P** + **!**, **1**, choose level where the quick button is to be stored, **P** + **!**, **2**, choose button **X** which is going to be a quick button.

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Abilia AB, Kung Hans väg 3, Sollentuna, Box 92, 191 22 Sollentuna, Sweden
Toby Churchill Ltd (Abilia), Toby Churchill House,