SPECIAL NEEDS
COMPUTERS 
1-877-724-4922

Gewa Control Medi Easy/Standard HM

GB: IR transmitter

DE: IR-Sender

FR: IR-Émetteur

NL: IR-Zender

Control Easy HM

Art.no 1560 0000 0002

Control Standard HM

Art.no 1562 0000 0002

ABILIA

GB: English Manual Gewa Control Medi Easy HM and Medi Standard HM

Introduction

The Gewa Control Medi Easy/Standard is an IR transmitter. Simply press the buttons to send signals.

When you press a button, confirmation is given as vibration, sound and light.

If a button on your Control Medi is pressed for more than 1 minute, Control Medi will stop transmitting to save power. By just pushing the button again, transmitting will resume.

Inserting Batteries

Unscrew the battery lid screws on the back of your Control Medi using a screwdriver and insert three AAA batteries (alkaline LR03 1.5 V).

Insert the batteries according to the guidelines at the bottom of the battery compartment. Pay attention to the polarity! It is important to use long life **alkaline** batteries that are protected against leaking battery acid.

Battery Warning Indicator

Control Medi has a built-in indicator to warn you of low battery voltage. You will hear a sound alert  with one long beep and three short beeps played twice. This occurs about 10 seconds after the transmitter was last used. Replace the batteries as soon as possible.

Neck Strap

Another option is to hang your Control Medi around the neck. A neck strap is supplied with your Control Medi.

Do as follows to attach the neck strap:

1. Unscrew the battery lid.
2. Place the thin loop of the neck strap in the groove around the screw mount.
3. Screw back the battery lid back down.
4. Adjust the length of the neck strap.

Warning!

The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap!

Cleaning

Your Control Medi can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

Warning!

Do not use any disinfectant except surgical spirit, Chlorhexidine or ethyl alcohol. These liquids are inflammable, handle with care!

Technical data

Power consumption at rest:	Approx. 2 μ A
Power consumption at transmitting	Approx. 40mA (GewaLink channel)
Input control switch type:	3.5mm jack plug 2-pole
Battery life:	Approx. 2 years (100 seconds transmitting per day with GewaLink channel)
Programming locations:	4 places (Medi Easy) 28 places (Medi Standard)
Operating force buttons:	< 160g
Range:	40-50 m (GewaLink channels)
EMC-standard emission: " immunity:	EN 301 489-3 V1.4.1:2002, EN(IEC) 61000-6-2:2005, EN(IEC) 61000-6-3:2007, selected parts from EN 300 220-2 V2.3.1:2010
Computer Connection:	Via 429252 Control Tool USB interface
Measurements (L x W x H):	100 x 55 x 22 mm (incl. rubber feet)
Weight (incl. batteries):	105g
Operating temperature (excl. batteries):	-20°C to +45°C

Safety and responsibility

Abilia has conducted a risk analysis for the product with reference to SS-EN 14971 for Medical equipment. The instructions for use and the technical specifications for the product have been drawn up in respect of the safety assessments carried out in this document.

It is always important to be aware of safety when first using this product and with its subsequent use. It is not possible to describe in the instructions all the situations in which the product must not be used. If you are not sure whether you have started using the product properly or set it up correctly, please contact your supplier.

Abilia is not liable for any loss, damage or consequential loss resulting from any use, correct or otherwise, lack of follow-up, incorrect set-up, incorrect connection or similar misuse. Abilia financial liability is at all times limited to the value of the product itself.

Recycling Waste Electrical and Electronic Equipment

The Control Medi contains electronics components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

Conformity with the Medical Devices Directive

The Control Medi is marketed as a technical aid for people with disabilities and meets the requirements in the Medical Devices Directive 93/42/EEC including relevant EMC and electrical safety requirements.

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Abilia AB, Kung Hans väg 3, Sollentuna, Box 92, 191 22 Sollentuna, Sweden