Somna Kedjetäcke Sensitive

Somna Blanket Sensitive

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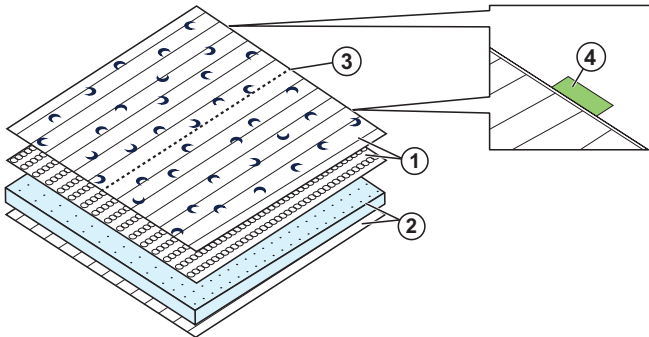
Somna Blanket Sensitive

Somna Blanket Sensitive is a product that is developed to relieve sleeping difficulties, tension in the body, agitation and anxiety by providing an enveloping feeling and tactile stimulation. Put the blanket in a duvet cover and try which side of the blanket feels best against the body. Start by using the blanket for a short time to get used to it and increase the use gradually.

The blanket is available in junior and adult sizes and in different weights. The blanket has two different sides, a chain side (1) and a padded side (2). The padded side against the body gives an even and soft pressure while the chain side gives more tactile pressure. The chain side has a fabric with crescents to make it easier to separate the sides.

1. Chain side with steel chains
2. Padded side
3. Zipper (if applicable)
4. Colour marking (if applicable) that indicates which parts belong together

Somna Blanket Sensitive complies with all necessary requirements, regulations and directives for medical devices. Dispose the product at an applicable disposal location.

For more information and accessories, go to www.abilia.com or scan the QR code.

Scan me!

Maintenance

 **CAUTION:** Use the included laundry bag. If the laundry bag is not used, Somna Blanket Sensitive can break.

Textile: 90% Polyester, 10% Polypropylene **Filling:** 100% Steel chains

If the product has a zipper (3), wash the two parts separately. The colour marking makes it easy to combine parts with the same weight after washing. Dry immediately after washing.

Reconditioning

Gentle wash cycle max. 70 °C.

Important information

 **WARNING:** Read the following warning instructions before you use the product.

- Seek medical advice before using the product if the user has reduced lung or muscle capacity or any surgically inserted devices that can be affected by the chains.
- Do not use Somna Blanket Sensitive for children younger than 3 years.
- When using the hygiene cover, observe the risk of a choking hazard.
- Do not roll the user in the Somna Blanket Sensitive.
- Risk of tripping if Somna Blanket Sensitive is left on the floor.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Phone +46 (0)8-594 694 00 | info@abilia.com | www.abilia.com

ABILIA