Somna Kedjefilt

Somna Comforter

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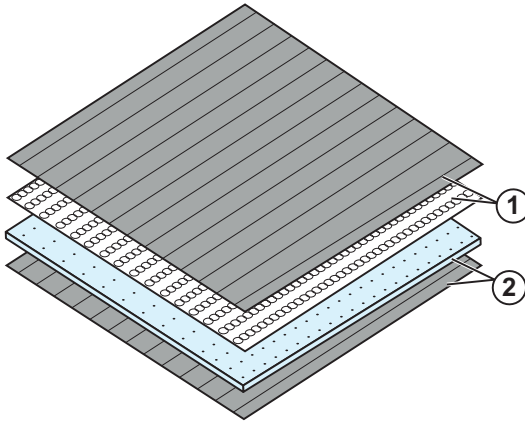
Somna Comforter

Somna Comforter is a daytime product that is developed to relieve tension in the body, agitation and anxiety. The comforter has two different sides, a chain side (1) and a padded side (2). The padded side against the body gives an even and soft pressure while the chain side gives more tactile pressure. Use the comforter in the lap, wrap it around the body or fold it if more weight anywhere on the body is required.

1. Chain side with steel chains
2. Padded side

Somna Comforter complies with all necessary requirements, regulations and directives for medical devices. Dispose the product at an applicable disposal location.

For more information and accessories, go to www.abilia.com or scan the QR code.

Scan me!

Maintenance

 **CAUTION:** Use the included laundry bag. If the laundry bag is not used, Somna Comforter can break.

Textile: 75% Polyester, 17% Cotton, 8% Polypropylene

Filling: 100% Steel chains

Reconditioning

Gentle wash cycle max. 70 °C.

Important information

 **WARNING:** Read the following warning instructions before you use the product.

- Seek medical advice before using the product if the user has reduced lung or muscle capacity or any surgically inserted devices that can be affected by the chains.
- Do not roll the user in the Somna Comforter.
- Risk of tripping if Somna Comforter is left on the floor.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Phone +46 (0)8-594 694 00 | info@abilia.com | www.abilia.com

ABILIA