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Somna Vest 2.0

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
Somna Vest 2.0


Somna Vest 2.0 is used during the day to calm the senses and increase the body perception so that you can more easily concentrate on activities in everyday life. It helps to maintain focus and filter out unnecessary impressions.

Use the vest as close to the body as possible. The vest and weighted collar can be used together or separately.


Use Somna Vest 2.0

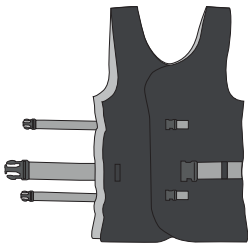1. Open the buckles.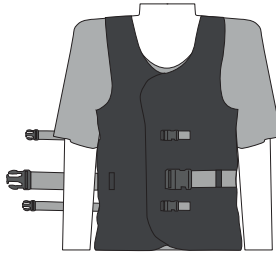

2. Put on the vest.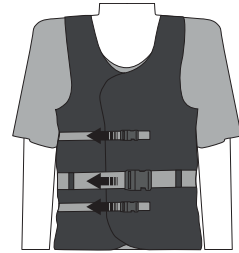

3. Close the buckles and tighten them.

Somna Vest 2.0 complies with all necessary requirements, regulations and directives for medical devices. Dispose the product at an applicable disposal location.

For more information and accessories, go to www.abilia.com or scan the QR code.

Scan me!

Maintenance

 **CAUTION:** Use the included laundry bag. If the laundry bag is not used, Somna Vest 2.0 can break.

Vest:

Textile: 70% Polyester, 30% Cotton

Filling: 100% Steel chains

Collar:

Textile: 55% Polyester, 45% Cotton

Filling: 100% Polyester

Do not dry clean. Do not heat.

Important information

 **WARNING:** Read the following warning instructions before you use the product.

- Seek medical advice before using the product if the user has reduced lung or muscle capacity or any surgically inserted devices that can be affected by the chains.
- Ensure that Somna Vest 2.0 does not catch on something when playing.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Phone +46 (0)8-594 694 00 | info@abilia.com | www.abilia.com

ABILIA