



Quick Guide




Buttons

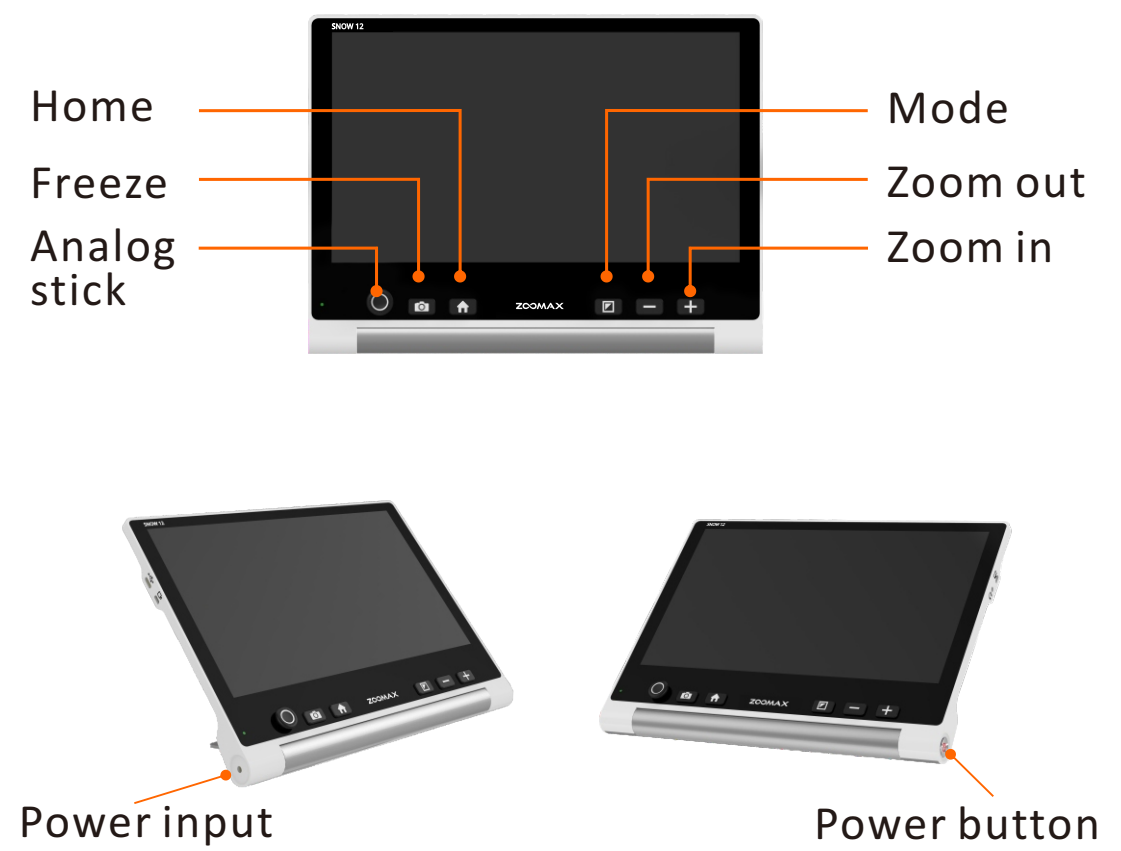


Buttons

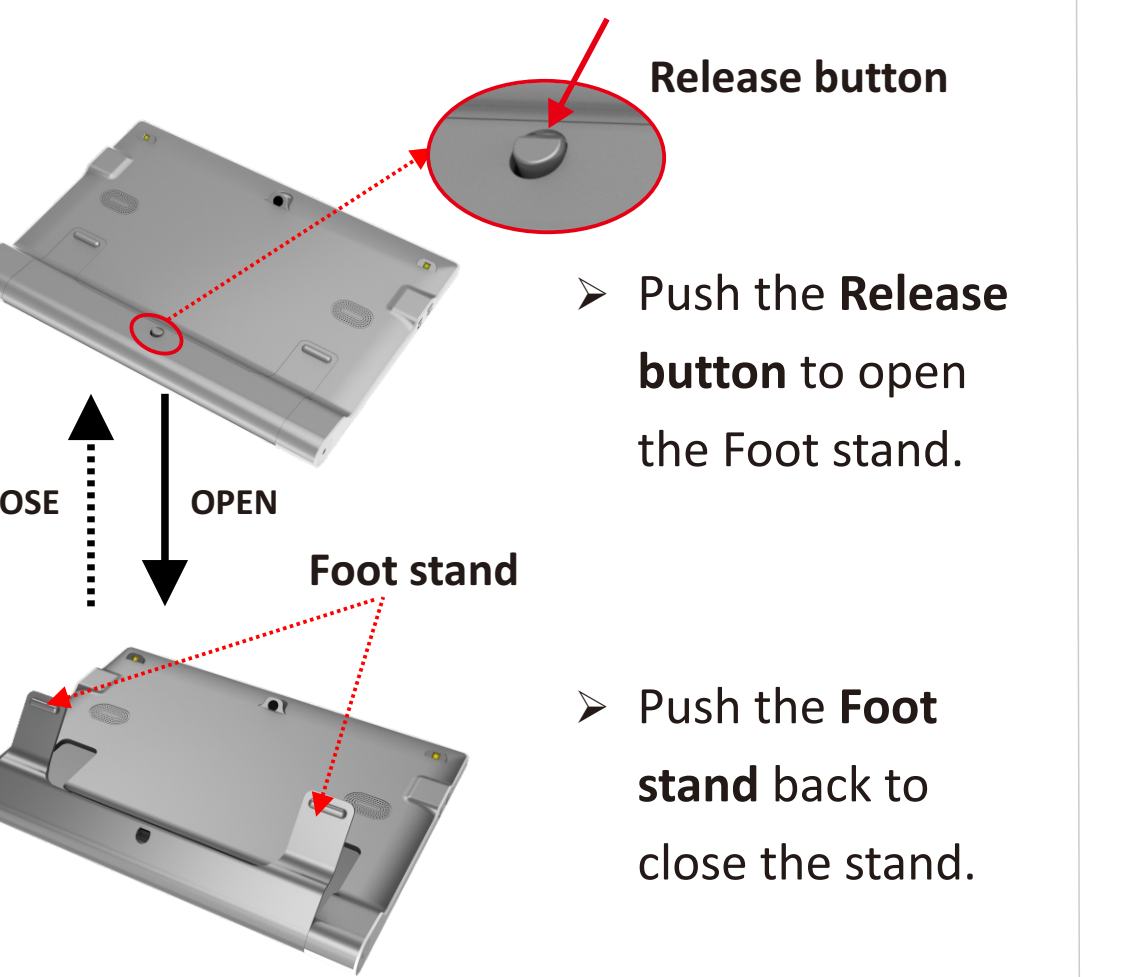

Foldable stand

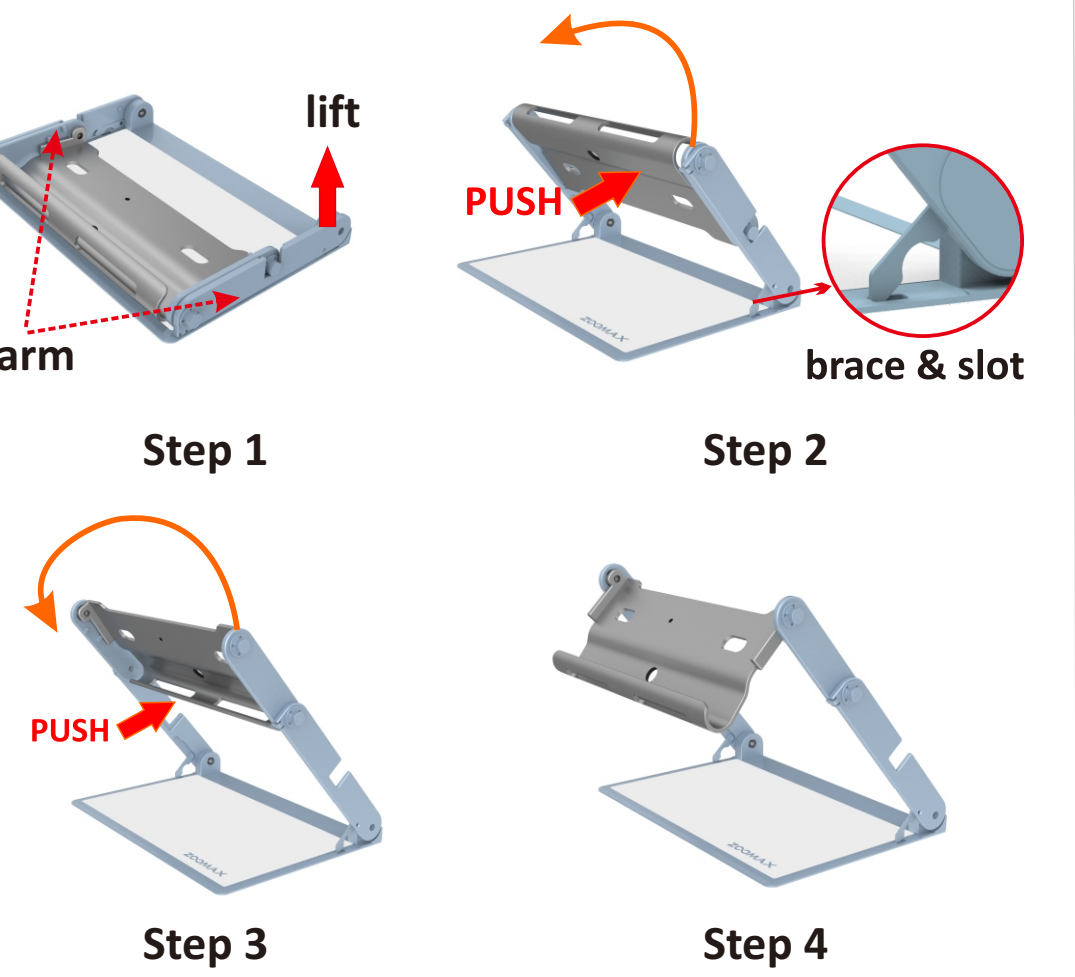

Snow 12 with foldable stand

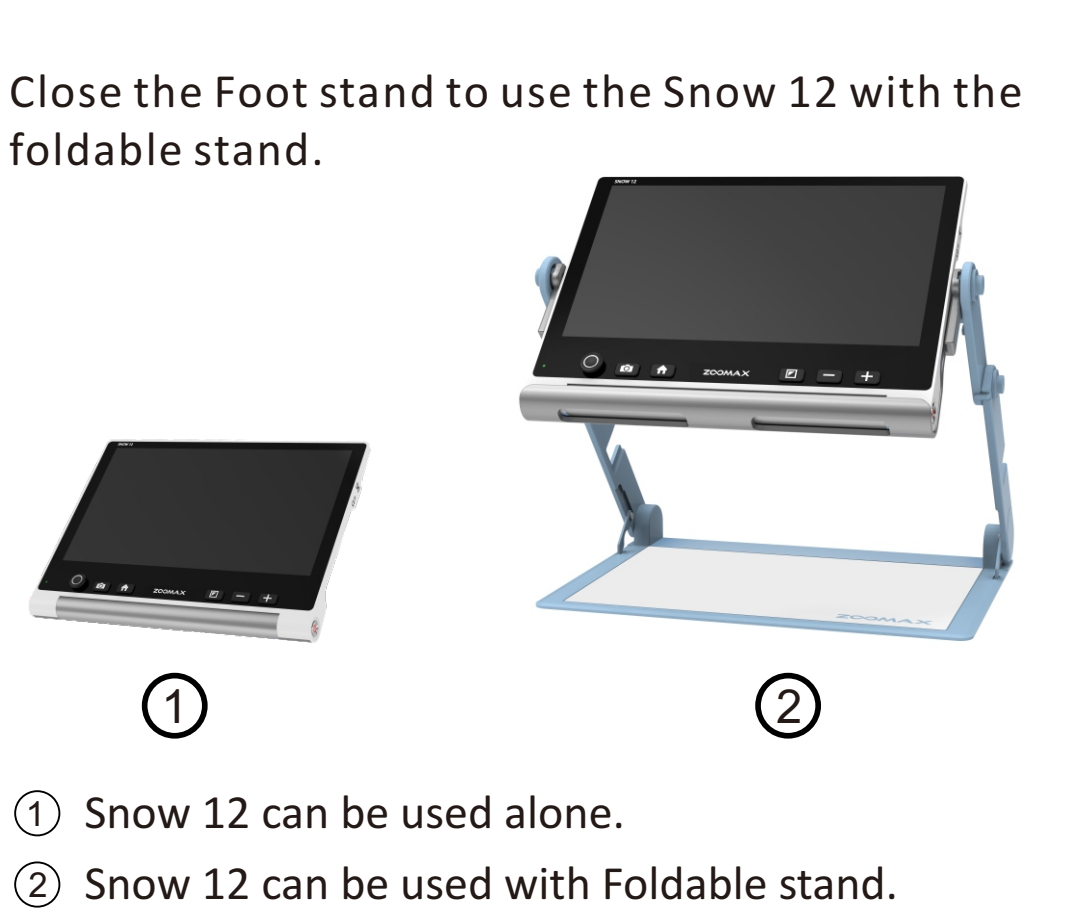

Basic function Operation

	Turn on/off the device	Press power button
	Zoom in/out	Press or
	Open the stand	Press the Release button on the bottom of the back cover
	Change color mode	Short press
	Change to full color fast	Press and hold for 1 second
	Freeze image	Short press
	Move image with analog stick	Move the analog stick up/down/left/right to move within a magnified image .
	Reading line or masks	Press and simultaneously in real-time viewing, or choose in Settings menu.

Advanced function Operation

	Main menu	Short press the home button to enter/exit main menu. Move the analog stick up/down/right/left to cycle through the icons. Press to confirm. Press to go back. Or tap the icons in main menu
	Save photo	Tap in the on-screen button bar after freezing the image.
	Check saved photo	Tap the gallery icon in main menu.
	Settings	Tap the settings icon in main menu.
	System information	Tap the system information icon .
	Adjust high contrast	Press and hold first and short press or , or choose in settings.

Text-to-speech function Operation

	Text-to-speech Mode	Press and hold the for 3 seconds to enter/exit Text-to-speech mode. Tap or select Text-to-speech icon in main menu. Place a document flat underneath Snow 12. Press to capture the image and convert the document into speech. Press to exit.
--	---------------------	---

ZOOMAX TECHNOLOGY CO., LIMITED
 Add.: 9F, Building D, Paradise Software Park, No.3 Xidoumen Road, Xihu District, Hangzhou, China
 Tel: +86-571-87006308
 Fax: +86-571-87397220
 Email: support@zoomax.com
 Visit us at: <https://www.zoomax.com>