

## User Guide



[www.bjliveat.com](http://www.bjliveat.com)

### **BJ Adaptaciones**


C/ Mare de Déu del Coll, 70, Bajos  
08023 Barcelona, Spain  
Phone: +34 93 285 04 37  
Fax: + 34 93 553 56 34  
e-Mail: [info@bjliveat.com](mailto:info@bjliveat.com)



**SPECIAL NEEDS**  
**COMPUTERS**   
**1-877-724-4922**

## BJ-106 Wobble Switch

This switch consists of an 11 cm padded wand that can be moved in any direction.

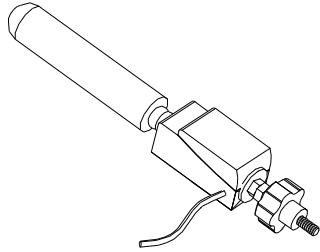

A swiping motion in any direction of the wand activates the switch. This is one of the more universal switches as it can be activated in multiple ways.

### Instructions

- By swiping the wand in any direction or pushing it down will activate the switch. We'll notice that we have activated the switch thanks to the "click" as auditory feedback produce by the switch. When you stop the pressure, the wand returns to its original position

### Features

- 1,7 m. cable male-plug , 3.5 mm mono output.
- Rod length: 11cm.
- Dimensions (excluding rod): 72x30x30 mm.
- Fixing screw: 1/4 inch, Whitworth thread.

### Maintenance and precautions

- Do not pull strongly on the cable.
- Do not clean the product with substances that discolor fabrics.

Do not clean the product with substances that discolor fabrics.

### Suggestions

- This switch can be attached directly to any arm, hoses and brackets in our catalog.