

## Cleaning & Decontamination



Tough-PAC® is a hard shell **anti-microbial case for mobile computers**. A silver biocide 'Biomaster' is permanently built in to the casing during manufacture. An **antibacterial screen protector** is also used.

Day-to-day cleaning should still be carried out as part of any infection or decontamination control policy. These recommendations apply to all Tough-PAC® products.

In all cases consult your local infection control team, policies and procedures.

### General day-to day cleaning

- Wear appropriate PPE.
- Turn off iPad using the sleep button in the handle aperture.
- Clean all external surfaces of Tough-PAC using a moist general cleaning wipe\*.
- Dry the surfaces and screen using a clean microfibre cloth.

\*Follow the user instructions for the cleaning product used.

### Suggested general cleaning products

- Clinell® Detergent wipes
- Clinell® Universal wipes
- Clinell® Alcohol wipes
- Synergy Health AzoActive wipes
- Synergy Health AzoMax wipes



Tough-PAC® Designed in Hampshire, Made in Britain  
European Registered Designs 1962978, 20154070 Copy & Design Rights Reserved

## Cleaning for outbreaks or Isolation

- Wear appropriate PPE.
- Turn off iPad using the sleep button in the handle aperture.
- Clean all external surfaces of Tough-PAC using a moist decontamination wipe, air dry\*.
- Clean again using a detergent wipe particularly the front screen and case contacts (to remove residues)
- Dry the surfaces and screen with a clean microfibre cloth.

\*0.1-0.5% bleach or a peracetic acid wipe. Follow the user instructions for the cleaning product.

## Suggested products

- Clinell® 'Clorox' wipes
- Clinell® Sporocidal wipes
- Clinell® Detergent wipes
- Guest Medical Chlor-Clean wipes



## Cleaning for body fluid contamination

- Wear appropriate PPE.
- Turn off iPad using the sleep button in the handle aperture.
- Remove all gross contamination\*.
- Clean all external surfaces of Tough-PAC using a moist chlorinated\*\* cleaning wipe, air dry.
- Clean again using a detergent wipe particularly the front screen and case contacts (to remove residues)
- Dry the surfaces and screen with a clean micro-fibre cloth.

\*If contamination has been rinsed/washed off with a cleaning solution we advise opening the case after cleaning, when completely dry externally to check for any fluid ingress; if present it must be removed. Replace the screen and port plugs on reassembly. Tough-PAC is not 100% waterproof – do not immerse in fluid.

\*\*0.5% bleach (5200ppm available chlorine) Follow the user instructions for the cleaning product.

## Suggested products

- Clinell® 'Clorox' wipes
- Clinell® Detergent wipes
- Guest Medical Chlor-Clean wipes

The Clinell® range of products have been tested on Tough-PAC and not found to cause any significant harm to the product. Others include 100% isopropyl alcohol, household Dettol multi-action and Power & Pure antibacterial sprays, Flash with bleach, Flash cleaning wipes. Synergy Health AzoActive, AzoMax, AzoWipette. Guest Medical Chlor-clean wipes

