QUICK START GUIDE

Blue2™ 4th Generation

wireless accessibility switch

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:55 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. On an iPad or iPhone go to Settings App > Bluetooth and make sure Bluetooth is turned on.
2. Turn Blue2 on.
3. On Blue2, press the mode button to select a use mode.
4. Under devices, select BLUE2-XXXXXXXX (X's denote Blue2's serial number). If you see more than one Blue2 listed, look on the back of Blue2 and match the serial number on Blue2 to the serial number shown on the iPad or iPhone.
5. Configure Switch Control in the Settings App on the iPad or iPhone that is paired with your Blue2.
6. Your Blue2 is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

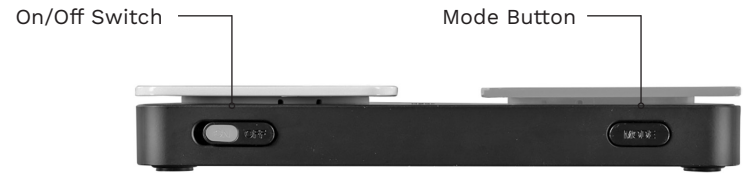
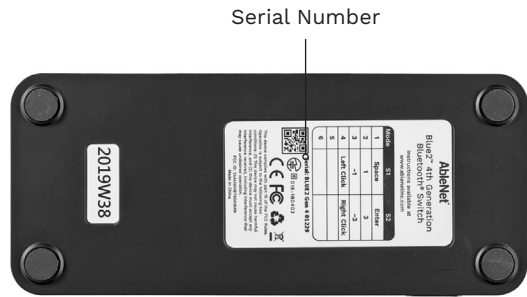
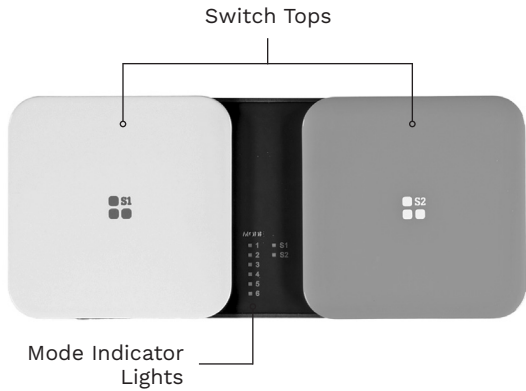
ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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