QUICK START GUIDE

Wireless Receiver

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 2:00 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Add two alkaline AA batteries to wireless receiver (batteries not included).
2. Turn wireless receiver on.
3. With wireless switch and receiver turned on, press and hold the receiver's "test" button until the light turns green (about 5 seconds).
4. Activate wireless switch.
5. Press and release the receiver's "test" button to exit pairing mode.
6. Plug the receiver into any switch-accessible device or computer switch interface.
7. Your wireless receiver is now ready to use with your wireless switch.

Note: AbleNet wireless receiver is only compatible with AbleNet Jelly Beamer and Big Beamer wireless switches.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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