

# VOICE BUDDY TIPS

## Wearing the Amplifier

1. Adjust the belt accordingly and attach the amplifier around the waist. You may also use the neck strap, but most would waist mount it. The amplifier may be worn in front, or on the hip.
2. Adjust length of strap of lanyard appropriately if wearing around the neck.
3. Wear the amplifier so that the speaker is resting against your body pointing outward.



## Wearing the Headset Microphone:

1. Fit the headset around the back of the head and rest it on top of the ears with the microphone on the left or right, depending on which mic was ordered.

NOTE: if you wear glasses, remove glasses before putting the headset on.

2. Position the microphone just off the corner of the mouth so that you don't receive any puffs of air through the amplifier speaker.
3. Try to get the microphone as close to the mouth as possible for maximum clarity and volume. Be certain that the flat spot of the mic is facing the mouth directly. Use thumb and forefinger to compress windscreen when confirming.



## Turning on Device and Adjusting the Volume Setting

1. Make sure that the microphone is plugged into the amplifier's headset microphone port at the top.
2. Turn the amplifier on by turning the volume control knob clockwise.
3. Gradually increase the amplifier volume to an appropriate level.
4. Check the volume by talking into the microphone. You want to get the volume set to as loud as required. You may wish to get someone in the room to confirm proper volume level.
5. Run the mic wire under your shirt or tie up the extra in a bunch to avoid hooking on and breaking it. A broken wire can cause static and be intermittent and will necessitate replacement of the headset.

## Charging

1. Insert the battery charger mini-plug into the hole on top of the device.
2. Plug adapter into a power outlet.
3. Charge for eight hours for the first pre-charge. The amplifier should operate for 4-8 hours depending on its use. Subsequent charging should be around 2 hours

NOTE: Flashing Red/Green= charge is in progress

Solid Green = charge is complete