
Talkable 6 Spinning Communicator w/Levels #2409

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Most accessible!

Our most popular communicators now come in a round, spinning version. A built-in Lazy Susan makes it easy to reach all six messages. Additional features include:

- 6 levels
- 225 seconds total recording time
- Records 6 messages per level
- Easy single message recording
- 6 Levels six seconds per message
- Volume control
- 6 clear covers for icons

Size: 8½"Diameter x 3"H. Requires 4 AAA Batteries. Weight: 1¾ lbs.

Operation:

1. Carefully turn over unit to reveal battery compartment. First remove the Velcro attached Lazy Susan to access the battery compartment. Remove the battery cover with a small Phillips Head screwdriver. Install four AAA size batteries into the holders observing polarity. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn the unit on by rotating the ON/Off/Volume knob clockwise and adjust to an audible level. Next turn the **LEVEL SELECTOR** knob to Level 1. To turn the unit off rotate the **VOLUME CONTROL** knob counterclockwise until you hear an audible "click".
3. To record, depress and hold the red **RECORD** button located on the back of the unit. While still holding the **RECORD** button, depress one of the six message switches on the front of the unit and speak into the microphone located on the back of the unit.
4. Release both the message switch and the recording button when the recording is finished. Play back your message by pressing and releasing the same message switch used to record. Each of the remaining switches can be recorded in the same manner.

5. The unit will retain recorded information indefinitely, even if the batteries are removed. Previously stored information will only be erased when a new recording is performed.
6. Move the **LEVEL SELECTOR** knob to Level 2 and repeat Step 3. Continue programming additional levels in the same manor, if needed.
7. It is recommended after use that the unit be turned off, to extend overall battery life. This can be done by turning the **VOLUME CONTROL** knob counterclockwise, until you hear an audible "click".
8. We have included 6 clear covers for placing your icons or photos. If you would like to add textures to the switch plates you can use or Gumball overlay kit #786A sold separately.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Disconnect the power by turning OFF the unit for 20 seconds to reset the unit. If the unit still does not work correctly after the power is reconnected, check and replace the batteries observing proper polarity.

Action #2: If the unit still does not work, make sure there is no dirt or obstacles between the message plates and the housing of the unit.

Action #3: Replace the batteries with fresh alkaline type batteries.

Care of Unit:

The Communicator can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.