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INSTRUCTIONS FOR USE GEL FOR ECG, EEG, EMG AND DEFIBRILLATION

GEL ECG is designed to reduce the resistance between skin and electrodes. The gel is also suitable for the defibrillation procedure effected with reusable pads.

The products must be used by qualified healthcare personnel.

GEL ECG does not oxidize, it does not contain fatty substances, it does not leave permanent residue on the electrodes and is easy to remove. It contains bactericidal Euxyl® K 100 whose active components are Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone. Conforming to ISO 10993-1 standard on biocompatibility

WARNINGS

- External use only.
- The product supplied is not sterile.
- This is a single-use product, do not re-use it. Reusing the product may lead to: alteration of materials; loss of initial functional features of product.
- Do not apply gel over irritated eyes or skin or with abrasions or wounds.
- The product may cause irritation or reddening of skin, especially on allergic people.
- Don't use on wounds.
- Close the bottle after use.
- For application of defibrillation cover carefully the complete pads' surface with gel; uncovered metal parts can cause burns. Do not use for pregelled pads.

FIRST AID MEASURES

- **Eyes:** In case of accidental contact with eyes, rinse profusely with water for 15 minutes. If irritation persists, consult a doctor.
- **Skin:** If skin becomes irritated, wash thoroughly with water. If irritation persists seek medical attention.
- **Ingestion:** Accidental ingestion usually is safe, but consult a doctor, if complications arise

INSTRUCTIONS FOR USE

- People sensitive to MCI/MI should wear protective gloves during the use.
- Clean and degrease the patient skin.
- Apply the desired amount of GEL.
- Apply and connect the electrode and equipment by following the instructions supplied by the manufacturers.
- For application of defibrillation please cover carefully the complete pads' surface with gel and carry out the procedure by following the devices' manual.
- After the procedure remove all residues of the gel with paper, TNT (Non-woven) or gauze moistened with water.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

GENERAL NOTES

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force. This gel is water-soluble and is no cause of environmental pollution.

The container can be given to the diversified disposal

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