G004**Gel spray per ECG in flacone da 230ml**

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EN INSTRUCTIONS FOR USE**G004****230 ml bottle of spray gel for ECG**

The ECG SPRAY GEL is a high quality product designed to facilitate the connection between skin and electrodes for ECG.

The products must be used by qualified healthcare personnel.

GEL ECG does not oxidize, it does not leave permanent residue on the electrodes and is easy to remove.

Thanks to its unique composition and the degreaser and disinfectant in it, it allows the reuse of reusable electrodes without frequent washing, increasing both the duration of the same report that the toilet from patient to patient.

It contains bactericidal Euxyl® K 100 whose active components are Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone.

Conforming to ISO 10993-1 standard on biocompatibility

WARNINGS

- External use only.
- The product supplied is not sterile.
- This is a single-use product, do not re-use it. Reusing the product may lead to: alteration of materials; loss of initial functional features of product.
- Do not apply gel over irritated eyes or skin or with abrasions or wounds.
- The product may cause irritation or reddening of skin, especially on allergic people.
- Do not swallow.
- Do not use the gel after the expiration date on the bottle.

FIRST AID MEASURES

- **Eyes:** In case of accidental contact with eyes, rinse profusely with water for 15 minutes. If irritation persists, consult a doctor.
- **Skin:** If skin becomes irritated, wash thoroughly with water. If irritation persists seek medical attention.
- **Ingestion:** Accidental ingestion usually is safe, but consult a doctor, if complications arise.

INSTRUCTIONS FOR USE

- People sensitive to MCI/MI should wear protective gloves during the use.
- Clean and degrease the patient skin.
- Apply the SPRAY GEL G004.
- Apply and connect the electrode and equipment by following the instructions supplied by the manufacturers.
- After the procedure remove all residues of the gel with paper, TNT (Non-woven) or gauze moistened with water.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

WARRANTY AND LIMITATIONS

Compliant with current European legislation on Medical Devices. Manufactured according to the procedures of the ISO 13485 certified Quality System. No responsibility can be attributed to the Manufacturer, who will not be obliged to compensate medical expenses or direct or indirect damages, deriving from the non-functioning or anomalies of the above devices, if they are used differently than as foreseen in these instructions for use. It is recommended to promptly inform the Manufacturer's Quality Assurance Service for any malfunction or defect that was discovered regarding this device. Any serious incidents relating to the device must be reported to the Manufacturer and to the competent authority of the Member State where the user and / or patient is located..

WAST DISPOSAL

The waste coming from the sanitarian structures must be discharged according to the regulation in force.

This gel is water-soluble and is no cause of environmental pollution.

The container can be given to the diversified disposal.

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