

## **Digital Dentures Workflows**



80% of patients coming for full dentures already have a denture, and with our digital process, we can help eliminate 75% of your appointment slots for copy dentures. That's a significant time saving for the dentist, and a much simpler process for the patient.

## Copy Dentures

#### First Appointment

 Reline impression of existing denture if required.
 IOS scan, CBCT or Lab scan

of existing denture



Upload your records via our website & complete online prescription form. Send your IOS via your IOS provider or upload via our website.

Quoris3D designs, manufactures and ships your new Copy Denture within 3 working days. Post to patient!
No denture fit appointment is required for copy dentures, although some dentists may choose to have a 2nd appointment.

#### Sending Digital Files

1 LAB SCAN





(2) IOS





3 CBCT







## Copy Denture Plus

#### First Appointment

- 1. Reline impression of existing denture if required.
- 2.IOS scan, CBCT or Lab scan of existing denture
- 3. Photographs (if altering aesthetics only)
- 4. Opposing and bite (if altering occlusion only)



Upload your records via our website and complete online prescription form. Send your IOS via your IOS provider or upload via our website.

Receive your denture design for review. Advise of any edits or approve.

Q3D manufactures your denture and ships within 5

working days

Post to patient! No denture fit appointment is required for copy dentures, although some dentists may choose to have a 2nd appointment.

# Digital Dentures Workflows

Replacement Dentures



Go from 5-7 appointments to just 3 with our replacement dentures workflow!

#### First Appointment

**Preliminary impression** 



Package and ship to Quoris3D, complete online prescription form via our website. Quoris3D creates wax bite block with hard base and ships to dentist.

## Second Appointment

Final impression & records

Reline & bite block records marking: centre line, cuspid positions, relaxed & high smile line. Photographs: Full face, opposing and bite





Receive your denture design for review. Advise of any edits or approve.

Q3D manufactures your denture and ships within 5 working days

## Third Appointment

Fit final denture



## Immediate Dentures

### First Appointment

- 1.IOS or conventional impression
- 2. Opposing and bite
- 3. Photos of full face and retracted bite
- 4. Details of teeth to be removed
- 5.Shade





Upload your records via our website and complete online prescription form. Send your IOS via your IOS provider or upload via our website.

For conventional impressions, package and send to Quoris3D.

Receive your denture design for review. Advise of any edits or approve.

Q3D manufactures your denture and ships within 5 working days

### Second Appointment

Fit final denture

