



# SCAN ASSISTANCE FOR DUAL SCANS

Watch this video before performing a dual scan

<https://www.youtube.com/watch?v=jGNIYLtMLGc&t=2s>

## HOW TO PREPARE FOR THE DUAL SCAN

Request SureMarks from Quoris3D or use your own gutta percha markers.

- Use 6 SureMarks or gutta percha markers
- Place half lingual and half buccal
- Place on varying horizontal planes
- Place on flange, below gingival plane of the teeth
- Use a #6 or #8 round bur to a depth of 1mm. Keep markers spherical in shape, no larger than illustrated
- Fill flush with gutta percha

## DUAL SCAN PROTOCOL

### Scan 1

- Seat appliance(s) and ensure fit with no rocking - if there is a rock, hard relines or adjust until good seating (NO soft relines ever)
- Fully Edentulous scan appliance - important to bite IN OCCLUSION with NO space between the opposing! No bite material, no cotton rolls.
- Partially Edentulous scan appliance - important to place bilateral cotton rolls and capture the CT
  - Bite with medium pressure and do not move

### Scan 2

- Scan appliance by itself sitting on any type of foam or scan table that is NOT plastic. Must sit above plastic scan table!