

# *Dovgan MTA Carrier*

## **Cleaning Instructions**



 **Quality  
Aspirators**

**CE**  
0483

# ***Dovgan MTA Carrier: Cleaning Instructions***

**Important:** The MTA Carrier must be cleaned immediately after use to avoid damage. MTA sets up hard, clogging the carrier if not removed properly after use.

During the procedure:

Dip the tip of the carrier into sterile water or saline between placement and repacking with more MTA.

Move the plunger wire back and forth using short strokes to clear the tube of any MTA before it sets up. Repack and continue procedure.

Following the procedure:

Remove Plunger and using a syringe, clear debris from the tube by rinsing with at least 6 cc of water within 15 minutes of use. Replace plunger and move wire through the carrier using short strokes mimicking material placement. Remove Plunger.

Place separated plunger and carrier into ultrasonic and ultrasonically clean for 15 minutes, rinse, carefully reassemble. **DO NOT BEND WIRE** by using excessive force during plunger replacement. Sterilize by any means.



P.O. Box 382120 • Duncanville, Texas 75138  
Phone: (972) 298-2669 • Fax (972) 298-6592  
1-800-858-2121

Email: [support@qualityaspirators.com](mailto:support@qualityaspirators.com)

Web: [www.QualityAspirators.com](http://www.QualityAspirators.com)