


Suction Irrigator


K-COMP

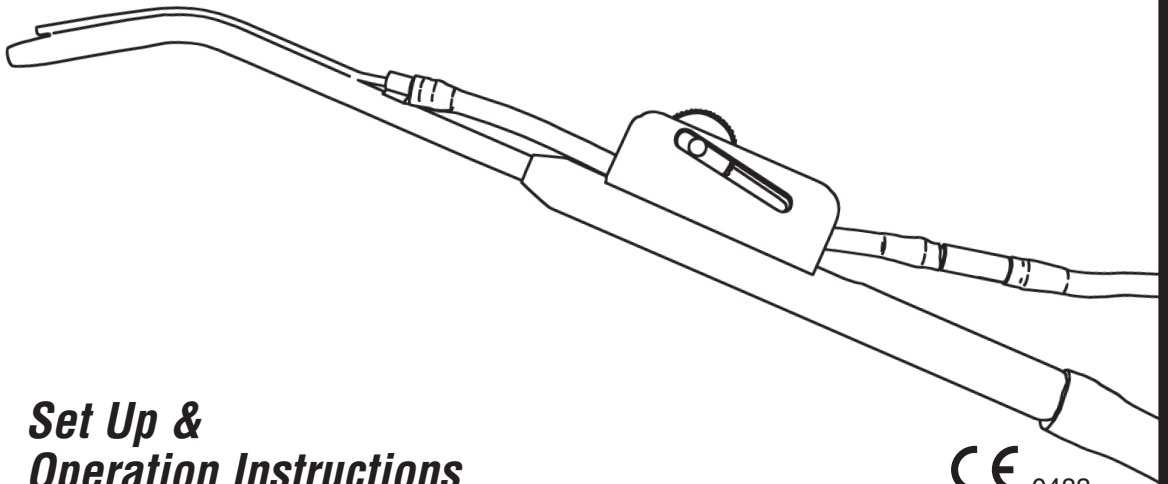

Set Up & Operation Instructions

CE 0483


K-COMP: Parts List

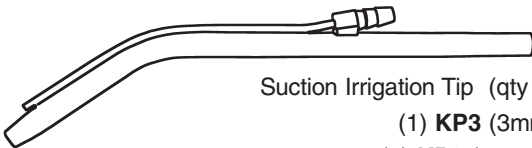

Suction Irrigation Tip (qty 3)
(1) **KP3** (3mm)
(1) **KP2** (2.5mm)
(1) **K04** (1.7mm)
WI lists KP1 & K05 as well.




KCOC – Irrigation line cut off clamp (qty 1)

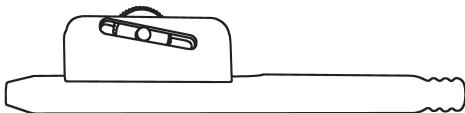


KH - Flow Control Handle (qty 1)



KSSS – Stainless steel striker (qty 1)

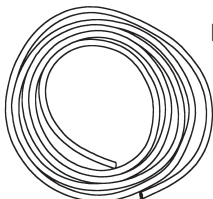

IRT - 7 ft. Silicone
irrigation line (qty 1)

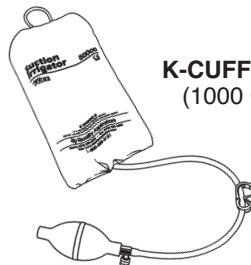


K-CUFF – 500 CC Pressure cuff (qty 1)
(1000 CC & 3000 CC also available)

Set-up / Assembly Instructions

1

Sterilize all parts before using.


2

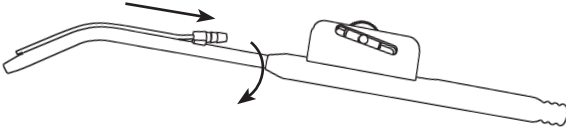

Insert Suction Irrigation Tip into Flow Control Handle. A very slight twisting motion will help "lock" the tip into position.

NOTE: over twisting may cause damage.

3

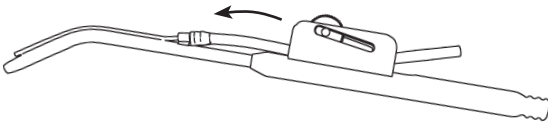

Feed the 7" silicone tubing (K-Tube) under roller ball on handle and onto barbs of the suction irrigation tip.

4

Place 500cc bag of saline or sterile water into pressure cuff.





Hang bag on supplied hook



6

Close off irrigation line using cut off clamp.

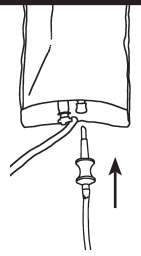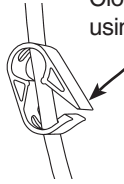

Insert irrigation line into K-CUFF bag.

7


Pressurize cuff by:

1. Closing air release valve.

2. Pump bulb until desired pressure is achieved.



8

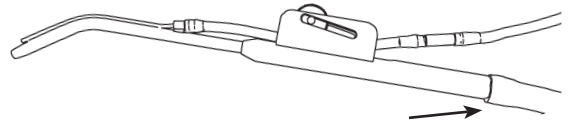

Attach surgical suction line to handle.

Open irrigation line which was closed in step 7.

Use the roller ball on the Flow Control Handle to start and stop the flow of the irrigation fluid.

Note: If only high volume suction is available, a surgical tubing adapter kit may be purchased separately.