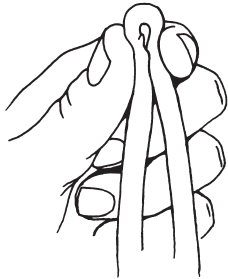

READ THIS FIRST!

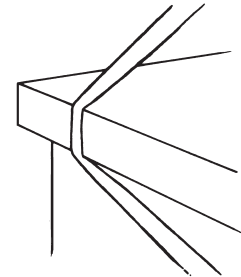
C A U T I O N

Your light guide contains small **GLASS FIBERS** which **WILL BREAK** if the light guide is crimped, stretched, rolled over with a chair or stepped on. This breakage is not covered by your warranty. With use, some individual fibers will break and appear as small dots of light along your light guide. This is normal and should be of no concern. Severe breakage is indicated by large bands of light where the light guide has been crimped, crushed or otherwise mishandled. Your light guide will provide you with quality service as long as you supply the care and maintenance.



DO NOT
CRIMP, STEP ON
OR ROLL OVER
THE LIGHT GUIDE
WITH A CHAIR.

DO NOT
STRETCH, PINCH
OR CRUSH THE
LIGHT GUIDE



STERILIZATION: Your light guide may be autoclaved or chemically. Do not exceed 275°F (135°C). The use of autoclave bags can reduce cosmetic damage and fiberoptic bundle deterioration as well as providing a suitable post-sterilization package. Please note that exposure to heat will reduce the life of your light guide. The average life expectancy of fiberoptics is 18 to 24 months with autoclaving or chemically. This time could be longer or shorter depending on frequency and temperature of autoclave or chemically.



Quality Aspirators

P.O. Box 382120 • Duncanville, Texas 75138 USA
1-800-858-2121 • (972) 298-2669 • Fax (972) 298-6592

