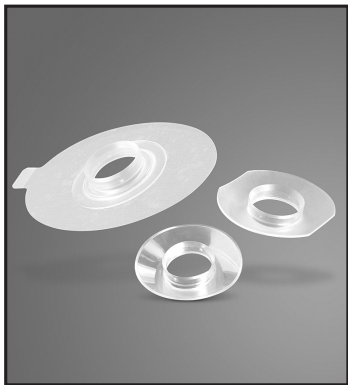
Blom-Singer®

voice restoration systems

Blom-Singer® Tracheostoma Housings

USER Instructions For Use

INHEALTH®
TECHNOLOGIES

R1

37587-01F

37587-01F Effective March 2014 / Gültig ab März 2014 / Efectivo a partir de marzo de 2014 / Date d'entrée en vigueur : mars 2014 / Valido da marzo 2014 / Met ingang van maart 2014 / Vigente em de março de 2014 / V platnosti od března 2014 / Wchodzi w życie w marcu 2014 r.

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BLOM-SINGER® TRACHEOSTOMA HOUSINGS

Introduction

Please refer to the diagrams located at the back of this instruction manual. A glossary of terms appears at the end of this chapter.

The Blom-Singer Tracheostoma Housing attaches to the skin surrounding the tracheostoma and acts as a holder for tracheostomal prosthetic devices, such as: the Blom-Singer HME/HumidiFilter System for heat and moisture exchange, the Blom-Singer Adjustable Tracheostoma Valve for hands-free speech, or the Blom-Singer Shower Guard for protection while bathing.

INDICATIONS

The Blom-Singer Tracheostoma Housing is designed to be applied to the skin area surrounding the tracheostoma.

HOW SUPPLIED

The Blom-Singer Tracheostoma Housings are available in two models: a preassembled with adhesive backing, disposable, one-time use housing; or, a reusable valve housing that requires user assembly and an application of medical grade liquid silicone adhesive for attachment of adhesive discs (supplied separately).

PRODUCT DESCRIPTION

The Blom-Singer Tracheostoma Housing is available in two models as shown in the chart below.

Blom-Singer Tracheostoma Housings	Self-adhesive, One-time use	Reusable	Use with Adhesive Disc
TruSeal® Adhesive Housing	•		
Tracheostoma Valve Housing		•	•

TruSeal® Adhesive Housing

The Blom-Singer TruSeal Adhesive Housing (diagram 1) is a disposable, all-in-one design that eliminates the need for a Tracheostoma Valve Housing, Adhesive Disc or medical grade liquid silicone adhesive. The TruSeal housing is a lightweight, pliable material with adhesive backing that comfortably fits around the tracheostoma. The TruSeal housing is available in one size and a variety of shapes in packages of 30 each.

Tracheostoma Valve Housing

The Blom-Singer Tracheostoma Valve Housing (diagram 2) is a reusable, flexible, circular housing that is used with an Adhesive Disc and medical grade liquid silicone adhesive for attachment (user assembly required). The Tracheostoma Valve Housing is available in two sizes, standard and large, and in two different materials, in packages of 1 each. Adhesive Discs and silicone adhesive not included and are available separately.

WARNINGS AND PRECAUTIONS

InHealth relies on the qualified, trained medical professional to instruct patients on the use and care of this device and to provide patients with the Blom-Singer instructions for use supplied with the product.

If there are tears, cracks, or structural damage to the device, discontinue use.

Extreme care should be used to avoid silicone adhesive, adhesive remover, protective skin barrier or fumes from entering the tracheostoma.

Although the occurrence of irritation is rare, discontinue use if skin irritation appears. Consult with your physician.

INSTRUCTIONS FOR USE

Please refer to the diagrams located at the back of this instruction manual.

Your clinician will instruct you on the routine use and care of the Blom-Singer Tracheostoma Housing.

The Tracheostoma Housing and user's hands **should always be thoroughly** clean to prevent contaminants from entering into the tracheostoma, puncture, or esophagus.

Removal and attachment of any devices worn over the tracheostoma should only be done with the user positioned in front of a mirror with a bright light focused directly on the stoma.

Do not use solvents or petroleum-based products for cleaning the device. These materials may damage the device.

A properly attached Tracheostoma Housing provides an airtight seal around the stoma.

Note: For patient-changeable voice prosthesis users, the voice prosthesis neck strap may interfere with an adequate seal around the stoma. To reduce the possibility of air leaks, trim the excess neck strap that extends outside of the housing. First, hold the housing over the tracheostoma with the voice prosthesis in place. Mark the amount of neck strap that extends outside the outer edge of the housing. Note the point where the neck strap meets the housing. Trim the excess neck strap just below this point so the remaining neck strap will be completely covered by the housing when it is properly in place. **Do not trim below the portion of the neck strap that has the small safety hole in it that attaches to the safety peg on the voice prosthesis inserter** (diagram 3a).

The following procedural instructions are provided by Eric D. Blom, Ph.D. Please refer to the diagrams located at the back of this instruction manual.

TruSeal Adhesive Housing

1. Gently clean the skin area around the tracheostoma with mild soap and warm water. Rinse and dry the skin completely. Next, gently wipe the skin area with an alcohol pad. **Avoid inhalation of alcohol fumes** which may cause coughing. Allow the skin area to dry completely. Apply one to two layers of a protective skin barrier, such as Shield Skin or Skin-Prep. **Apply carefully. Avoid entry of skin barrier or inhalation of fumes into the tracheostoma** which may cause coughing and irritation. Allow the skin area to dry completely.
2. If a more durable seal is necessary, a very thin coating of medical grade liquid silicone adhesive may be applied to the skin prior to attaching the TruSeal housing (step 4). Allow the silicone adhesive to dry for at least 3 to 4 minutes before attaching the housing. **Note:** a thin coating of silicone adhesive works better

than a thick application. **Apply carefully. Avoid entry of silicone adhesive or inhalation of fumes into the tracheostoma** which may cause coughing and irritation.

3. Remove the TruSeal housing from the plastic bag and place on a clean, dry surface.
4. Using the pull-tab, slowly peel the paper backing from the TruSeal housing to expose the adhesive. Avoid allowing the adhesive liner to fold onto itself.
5. Stretch the skin gently away from the stoma and affix the TruSeal housing to the recessed area surrounding the tracheostoma. Orient the housing with the pull-tab down (diagram 4).
6. Press the TruSeal housing against the skin surface to avoid air pockets between the skin and the housing. Gently rub the TruSeal housing against the skin to achieve maximum adhesive contact.

Removal and Cleaning

To avoid skin irritation, carefully and slowly remove the TruSeal adhesive housing from the skin. Any left-over adhesive on the skin surrounding the tracheostoma can be removed with medical grade adhesive remover, such as Remove, followed by gentle cleansing with soap and water. **Extreme care should be used to avoid entry of adhesive remover or fumes into the tracheostoma when cleaning the surrounding skin area.**

Troubleshooting

The following information is provided by Eric D. Blom, Ph.D.

Problem

TruSeal adhesive housing will not adhere securely to skin.

Causes and Remedies

1. Excessive resistance to airflow through the voice prosthesis and/or throat. See your surgeon or speech pathologist for assessment.
2. Insufficient TruSeal housing application and/or use. Review Instructions for Use.
3. Failure to allow the liquid silicone adhesive to dry thoroughly, at least 3-4 minutes, prior to attaching the housing.

-
4. Failure to carefully remove the ATSV or HME/HumidiFilter System prior to coughing.
 5. Excess internal pressure against the housing seal caused by trying to talk too loudly.

Problem

Skin irritation from TruSeal adhesive housing.

Causes and Remedies

1. Avoid repeated application of the adhesive housing in the same day.
 2. Remove adhesive housing slowly and carefully to avoid irritation.
 3. Apply Shield Skin or Skin-Prep to skin surrounding tracheostoma prior to the next application of TruSeal housing.
- If difficulty with your TruSeal adhesive housing cannot be easily remedied, consult your physician or speech pathologist for assistance.

Tracheostoma Valve Housing

1. Gently clean the skin area around the tracheostoma with mild soap and warm water. Rinse and dry the skin completely. Next, gently wipe the skin area with an alcohol pad. **Avoid inhalation of alcohol fumes** which may cause coughing. Allow the skin area to dry completely. Apply one to two layers of a protective skin barrier, such as Shield Skin or Skin-Prep. **Apply carefully to avoid inhalation of fumes** which may cause coughing. Allow the skin area to dry completely.
2. Apply a very thin coating of medical grade liquid silicone adhesive to the skin area surrounding the tracheostoma where the valve housing will be positioned. **Note:** a thin coating of silicone adhesive works better than a thick application. **Do not inhale silicone adhesive or permit it to enter the stoma.** Allow the silicone adhesive to dry for at least 3 to 4 minutes prior to positioning the assembled adhesive disc/tracheostoma valve housing.

Blom-Singer Adhesive Tape Disc Assembly

1. Using an adhesive tape disc, attach the disc to the housing by first slowly peeling the backing paper from one side of the disc.

Avoid pulling the backing paper from the reverse side of the disc (diagram 5).

2. Place the circular housing on top of the disc.
3. Flatten the circular housing against the disc surface. Rub firmly against the entire circular area of the housing to ensure complete contact with the disc (diagram 6).
4. Remove the paper backing on the reverse side of the disc to expose the second adhesive surface (diagram 7).

Foam Disc Application (refer to diagrams 5-7)

1. Remove the backing paper from one side of the adhesive foam disc. Avoid pulling the backing paper from the reverse side of the adhesive foam disc.
2. Place the circular housing on top of the adhesive foam disc.
3. Flatten the circular housing against the adhesive foam disc surface. Rub firmly against the entire circular area of the housing to ensure complete contact with the foam disc.
4. Remove the paper backing on the reverse side of the adhesive foam disc to expose the second adhesive surface.

Attaching Housing to Skin

1. Position the circular housing over the recessed area surrounding the tracheostoma. Press the housing flush against the skin surface to avoid air pockets between the skin and the housing.
2. Rub the circular housing firmly against the skin to achieve maximum adhesive contact (diagram 8).

Removal and Cleaning

Carefully remove the reusable circular housing from the skin. Peel back the tape and adhesive foam disc from the housing. Any left-over adhesive on the skin surrounding the tracheostoma can be removed with medical grade adhesive remover, such as Remove, followed by gentle cleansing with soap and water. **Extreme care should be used to avoid getting adhesive remover or fumes in the tracheostoma when cleaning the surrounding skin area.**

Clean any left-over adhesive from the surface of the housing with

medical grade adhesive remover, such as Remove, or by soaking the housing overnight in water and then rubbing off the adhesive under warm tap water.

Replace housing when flexibility decreases.

Troubleshooting

The following information is provided by Eric D. Blom, Ph.D.

Problem

Tracheostoma Valve Housing will not adhere securely to skin for a 12–14 hour period.

Causes and Remedies

1. Excessive resistance to airflow through the voice prosthesis and/or throat. See your surgeon or speech pathologist for assessment.
2. Insufficient Tracheostoma Valve Housing application and/or use. Review application steps in Tracheostoma Valve Housing instructions for use.
3. Failure to allow the liquid silicone adhesive to dry thoroughly, at least 3-4 minutes, prior to attaching the housing.
4. Failure to carefully remove the ATSV or HME/HumidiFilter System prior to coughing.
5. Excess internal pressure against the housing seal caused by trying to talk too loudly.

Problem

Skin irritation from adhesive tape.

Causes and Remedies

1. Avoid repeated application of the housing in the same day.
2. Remove housing slowly and carefully to avoid irritation.
3. Apply Skin-Prep to skin surrounding tracheostoma prior to applying silicone adhesive and housing.

If difficulty with your Tracheostoma Valve Housing cannot be easily remedied, consult your physician or speech pathologist for assistance.

SPECIAL ORDER PRODUCTS

If this instruction manual accompanies a Special Order Product, there may be differences in the physical characteristics between the product and the product descriptions described in this instruction manual. These differences will not affect the safety or desired beneficial effect of the special order product. Special Order products are nonreturnable.

ORDERING INFORMATION

USA

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337, Monday–Friday, 9:30am–7:00pm, Eastern Standard Time. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: order@inhealth.com ORDER ON-LINE: www.inhealth.com POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service

International

Please contact our customer service department for a distributor referral.

Consumer Affairs

If you have any questions or dissatisfaction with a product please contact our customer service department via telephone, fax, post, or Email: productcomplaints@inhealth.com

RETURNED GOODS POLICY

All returned merchandise must have a Return Merchandise Authorization number (RMA), and must be unopened and undamaged. RMA numbers are obtained from InHealth Customer Service. Returns without an RMA will not be accepted. Products must be returned in unopened packages, with manufacturer's tamper evident seals intact to be accepted for replacement or credit. Products will not be accepted for replacement or credit if they have been in possession of the customer for more than 14 days. Special order products are nonreturnable. There is a 20% restocking fee on all returned merchandise.

LIMITED WARRANTY, LIMITED REMEDY AND LIMITATION OF LIABILITY

InHealth Technologies, a Division of Helix Medical, LLC, warrants that the Product will be free from defects in workmanship and materials on the date of purchase. THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES OR REPRESENTATIONS, EXPRESS OR IMPLIED, ARISING BY LAW OR CUSTOM, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SAID IMPLIED WARRANTIES BEING HEREBY EXPRESSLY DISCLAIMED. Purchaser's sole and exclusive remedy for a valid warranty claim is replacement of the Product. This remedy does not include the cost of installation, removal, dismantling, delivery or reinstallation. InHealth shall have no liability to the extent Products are or have been modified or subject to misuse, abuse or improper storage, installation or maintenance or damaged in transit. No returns will be accepted without prior approval and authorization. INHEALTH SHALL NOT BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT, CONSEQUENTIAL, PUNITIVE, OR EXEMPLARY DAMAGES OF ANY KIND.

GLOSSARY

Airway

The air passage by which air reaches the lungs.

Candida albicans

A yeast fungus that is a commonly occurring condition that can infect the mouth or the skin and may multiply and overgrow on to the voice prosthesis.

Clinician

A health professional, such as a physician, licensed speech-language pathologist, or nurse who is involved in clinical practice.

Contraindications

A reason that makes it inadvisable to prescribe a particular procedure or treatment.

Esophagus

The muscular membranous tube for the passage of food from the pharynx to the stomach; the "food pipe."

Fistula; TE-fistula

An abnormal duct or passage that connects an abscess or cavity to the body surface; a small opening between the trachea and the esophagus.

Indications

A reason to prescribe a drug or perform a procedure.

Respiration

The act or process of inhaling and exhaling; breathing; moving air into and out of the lungs. Also called ventilation.

Respiratory Tract (laryngectomy)

The air passage from the tracheostoma to the air sacs of the lungs, including the trachea, and bronchi through which air passes during breathing.

Tracheoesophageal puncture (TEP)

An opening made by a surgeon between the trachea and the esophagus. The opening is made at the time of initial surgery or later so that a voice prosthesis can be placed into this passage.

Trachea

The airway through which respiratory air passes to and from the lungs (windpipe).

Tracheoesophageal (TE)

Pertaining to the trachea and esophagus.

Tracheostoma

An opening through the neck and into the trachea. It is through this opening that the laryngectomye breathes.

Voice prosthesis

A one-way speech valve that is inserted into the TEP to allow air to pass from the trachea through the prosthesis and into the esophagus to produce sound.

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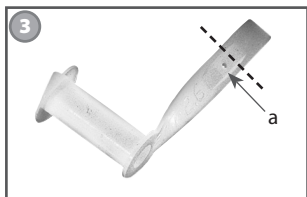
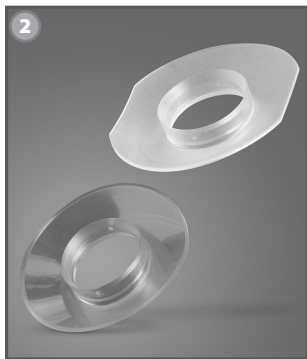
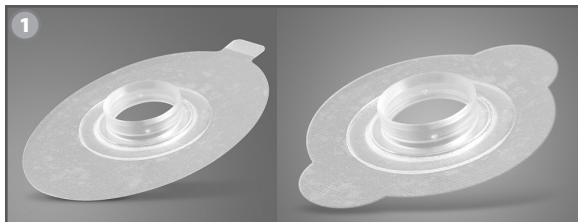
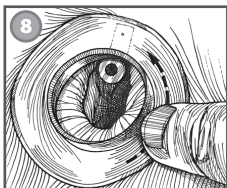
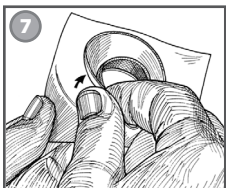
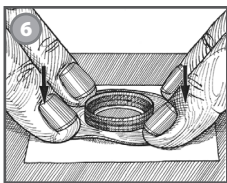
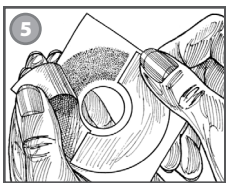
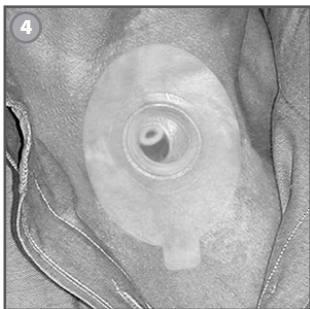
Additional references available upon request.

Weitere Referenzen aus der Literatur sind auf Anfrage erhältlich. / Se pueden solicitar más referencias. / Autres références disponibles sur demande. / Riferimenti aggiuntivi sono disponibili a richiesta. / Meer informatie is op aanvraag beschikbaar. / Referências adicionais mediante solicitação. / Další odkazy na literaturu jsou k dispozici na požádání. / Dodatkowe informacje dostępne na żądanie.

1. Blom, E.D., "Tracheoesophageal Speech," **Seminars in Speech and Language**. New York: Thieme Medical Publishers. 16:3, 191-204, 1995.
2. Blom, E.D., Hamaker, R.C.: "Tracheoesophageal Voice Restoration Following Total Laryngectomy," **Cancer of the Head and Neck**, E.N. and Suen, J. (Eds). W.B. Saunders Publishers. 839-852, 1996.
3. Blom, E.D., Hamaker, R.C., Freeman, S.B.: "Postlaryngectomy Voice Restoration." Lucente, F.E. (Ed.): **Highlights of the Instructional Courses**, Vol. 7. St. Louis: Mosby-Year Book. pp 3-10, 1994.

INHEALTH[®]
T E C H N O L O G I E S

Helix Medical, LLC

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
www.inhealth.com

EMERGO EUROPE

Molenstraat 15
2513 BH, The Hague
The Netherlands

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© 2014 Helix Medical, LLC (InHealth Technologies)
Blom-Singer is a registered trademark of Hansa Medical Products, Inc.
InHealth Technologies is a registered trademark of Helix Medical, LLC.