



**CLEANING INSTRUCTIONS  
FOR  
BARTON-MAYO TRACHEOSTOMA BUTTON**

The Barton-Mayo Tracheostoma Buttons are not sterile when packaged. Autoclaving the Barton Buttons is recommended prior to insertion. Trial buttons used during the sizing session should be cleaned with detergent and water and steam autoclaved.

Patients should clean their buttons with mild detergent and water and allow to dry after each use.

**STERILIZATION PROCEDURE  
FOR  
BARTON-MAYO TRACHEOSTOMA BUTTON**

Recommendation for sterilization of the Barton-Mayo Tracheostoma Button is a Steam Autoclave cycle at 270 degrees F for 5 minutes.

This is normally followed by a drying cycle.



6256 34<sup>th</sup> Avenue NW  
Rochester MN 55901  
1.507.289.0761 Phone  
1.507.281.9096 Fax  
Medicalinnovations.com