

Now, choose the heat and moisture exchange system right for your breathing needs and activity levels



Introducing the new  
Blom-Singer®  
EasyFlow™ HME

Blom-Singer®  
ClassicFlow™ HME

EasyFlow™ HME



ClassicFlow™ HME



**INHEALTH®**  
TECHNOLOGIES

*We speak ENT*

# Choosing the right HME is easy: Blom-Singer® HMEs

Take your pick: the Blom-Singer ClassicFlow or the new Blom-Singer Easy-Flow. Both help ensure that you breathe humidified, filtered, and warmed air. Research shows that long-term HME use results in:

- *Less mucus*<sup>1</sup>
- *Less coughing*<sup>1</sup>
- *Less risk of shortness of breath, fatigue, sleep problems, anxiety, and depression*<sup>1,2</sup>




## The new Blom-Singer® EasyFlow™ HME

*Breathe easier at work or play*

With EasyFlow, you can breathe a little easier. Its foam filter is more porous than the ClassicFlow. This allows you to breathe more freely to match your activity level and pulmonary needs.

Average Differential Pressure<sup>3</sup>

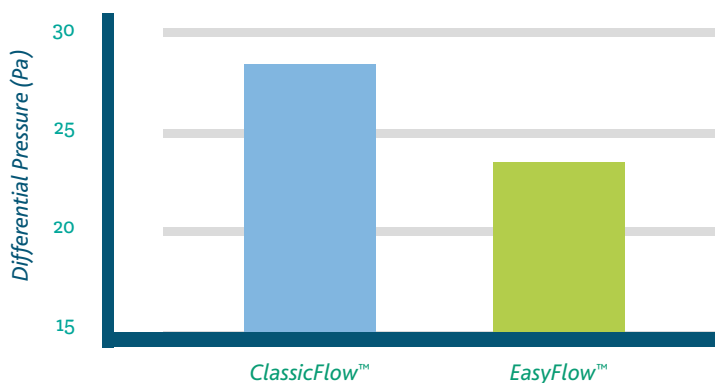

HME Cassette	Airflow Resistance <sup>3</sup>	Moisture Exchange <sup>3</sup>	Hygroscopic Antimicrobial Filter	24-Hour Use	EasyTouch Push to Speak	MucusShield Protection
EasyFlow™	Lower	Good	✓	✓	✓	✓
ClassicFlow™	Low	Better	✓	✓	✓	✓

Plus, EasyFlow's beige tint and non-reflective surface texture is less noticeable.



## The Blom-Singer® ClassicFlow™ HME

*Breathe healthy day and night*

ClassicFlow helps assure you breathe warmed, filtered, and humidified air—all day, every day—to reduce respiratory symptoms after a total laryngectomy.<sup>1</sup> ClassicFlow is designed for increased moisture retention and resistance.



### Both share unique advantages

#### *MucusShield™ helps protect the foam filter*

With other HMEs, if mucus gets on the filter, the efficiency of the filter is reduced. Not so with Blom-Singer HMEs. The MucusShield technology guards the foam filter from direct contact with mucus to help enhance efficiency.

#### *EasyTouch™ speech button makes speaking easier*

When you want to talk, press the EasyTouch speech button to seal the valve and speak easily. Upon release, the EasyTouch returns to its open, breathing position.

#### *24/7 heat and moisture exchange*

Our HMEs are "daily-disposable," 24-hour-use devices. You simply put on a new one each day, and the benefits continue through the night.



*Also available*

## Blom-Singer® HumidiFilter™ System

HumidiFilter is the simple, cost-effective alternative for heat and moisture exchange. Simply place a new filter once a day into the holder to get the benefits of wearing an HME. The holder is reusable and designed for daytime use.

# Blom-Singer®

The voice of experience since 1978

## For everyday or more active use, the choice is easy: Blom-Singer® HMEs

For more information or to order the new Blom-Singer® EasyFlow™ HME, Blom-Singer® ClassicFlow™ HME, or Blom-Singer® HumidiFilter™ System, contact InHealth Technologies.

Call **800.477.5969**

Email [order@inhealth.com](mailto:order@inhealth.com)

Visit or order online [www.inhealth.com](http://www.inhealth.com)

REF	DESCRIPTION
BE 1055EF	Blom-Singer EasyFlow HME — for 24-hour use, 30 per pack
BE 1055EZ	Blom-Singer ClassicFlow HME — for 24-hour use, 30 per pack
BE 1060	Blom-Singer HumidiFilter Holder, 1 each
BE 1080	Blom-Singer HumidiFilter Holder Foam Filters, 30 per pack

# INHEALTH®

TECHNOLOGIES

*We speak ENT*

1. Ackerstaff AH, Hilgers FJ, et al. Improvements in respiratory and psychosocial functioning following total laryngectomy by the use of a heat and moisture exchanger. *Annals of Otolaryngology, Rhinology & Laryngology*. 1993;102(11):878-883.

2. Ackerstaff AH, Hilgers FJ, et al. Heat and moisture exchangers as a treatment option in the post-operative rehabilitation of laryngectomized patients. *Clinical Otolaryngology*. 1995;20:504-509.

3. Research and Product Development Study, HME Test. RD-237.

**UNITED STATES**  
1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
Toll Free: 800.477.5969  
Toll Free Fax: 888.371.1530

**ORDER ONLINE**  
[www.inhealth.com](http://www.inhealth.com)