

Description

Tracheostomy P.A.M.™ Pediatric Airway Model is designed for use by healthcare practitioners to educate students, families, patients, and clinicians about tracheostomy in the pediatric airway and the proper application of the Passy Muir® Valve.

Design

The anatomy and airway shown by Tracheostomy P.A.M. illustrate the approximate anatomy of a toddler (ages 2-4). Consistent with the anatomy of a toddler, P.A.M.'s epiglottis and vocal folds are represented in a high position, the soft palate and tongue are larger in relation to the total size of the oral cavity, and P.A.M.'s vocal folds are displayed in an anterior slanted position. Tracheostomy P.A.M. is designed to accommodate a pediatric sized tracheostomy tube.

Accessories

P.A.M. is packaged with helpful educational accessories to enable clinicians to provide a wide variety of education related to tracheostomy. The kit includes three demonstration Passy Muir® Valves, a cuffed tracheostomy tube, and syringe for education and practice related to use of speaking valves with and without a ventilator. A nasogastric tube is provided for the display of nasogastric intubation. A customized Tracheostomy P.A.M. product package is provided for easy storage and transport of the entire P.A.M. kit.



Included with Model

- Tracheostomy P.A.M.™
- Demonstration tracheostomy tube with inflatable cuff
- 10 mL syringe
- No-leak Speaking Valves:
PMV® 007 (Aqua Color™)
PMV® 2001 (Purple Color™)
PMV® 2000 (clear)
Demonstration only non-functional devices
- PMV® Secure-It®
- Tracheostomy tie
- Nasogastric tube
- Warning label
- Zippered accessories pouch
- Removable stand with hex wrench and 2 screws
- Custom storage packaging

The products in this kit are for demonstration purposes only and not intended for patient use

Assembly

Invert P.A.M. and place the stand with the curved edge facing the front of P.A.M. Use the wrench to install the two screws to ¼ turn past hand tight.

Cleaning and Maintenance

The P.A.M. Pediatric Airway Model and accessories should be wiped clean using warm water and mild soap. Do not use solvent or alcohol.

Technical Assistance

The clinical staff at Passy Muir is available to answer questions you have regarding assembly or to provide suggestions on how to use the P.A.M. Pediatric Airway Model for teaching purposes.


Instructions for Use

Passy Muir has prepared instructional videos that provide instructions for use of the Tracheostomy P.A.M. Pediatric Airway Model and helpful educational tips.


Visit www.passymuir.com for additional information or scan this code:




Also Available
Trach T.O.M.™
Our adult-size teaching model
&
Pocket T.O.M.™
Our pocket-size teaching model

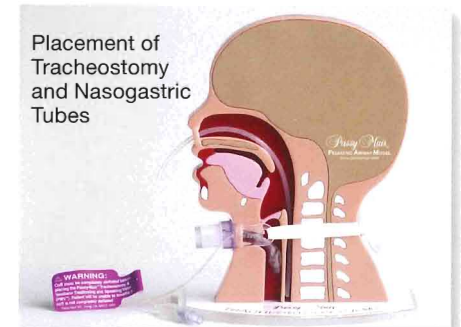

Placement of Tracheostomy and Nasogastric Tubes

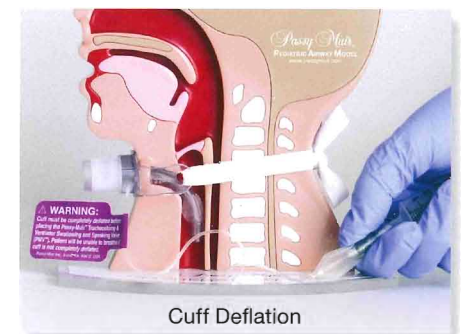

Cuff Deflation

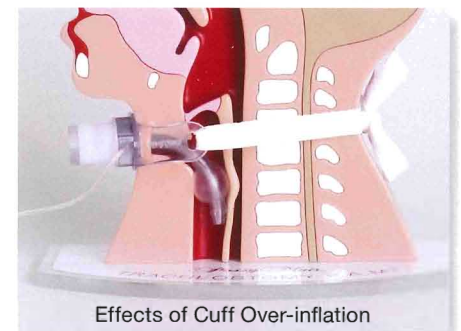

Effects of Cuff Over-inflation



Application of Passy Muir® Valve