Bionova® MiniBio Auto-reader

Operating relative humidity
Humedad relativa de funcionamiento
Umidade relativa de funcionamento
Umidade relativa de funcionamento
Umidade relativa de funcionamento

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EN Bionova® MiniBio Auto-reader

Product description

Bionova® MiniBio Auto-reader has been designed for the incubation and automatic readout of Terragene® Rapid, Super Rapid and Ultra Rapid self-contained biological indicator (SCBIs) appropriate for sterilization and environmental disinfection process control applications. Bionova® MiniBio allows two different incubation temperatures, 37 °C or 60 °C. Microorganisms contained within the SCBIs vary depending on the sterilization process for which they have been designed, therefore, incubation temperatures and incubation times differ depending on the SCBI being used. Bionova® MiniBio allows independent time settings selection for each one of the three incubation positions. SCBIs for Ethylene Oxide (EO), should be incubated at 37 °C, while those used for the monitoring of the rest of the sterilization processes should be incubated at 60 °C.

Important: Simultaneous incubation of SCBIs for monitoring EO processes along with SCBIs for monitoring other sterilization processes is not possible. Please check the Product Compatibility and Incubation programs section for more information about Incubation Programs and new available Indicators for your Bionova® MiniBio. Bionova® MiniBio Auto-reader allows easy and rapid detection of positive and negative SCBIs using advanced fluorescence techniques. A positive result can also be evidenced by culture medium color change when performing extended incubations. Read the instructions for use of each SCBI for more information. The option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital. Bionova® MiniBio Auto-reader also provides a printed ticket each time an incubation is completed for recording the results. This allows for easy result management, for documentation compliance and safe-keeping.

Indications for Use
Terragene® Bionova® Reader Incubators MiniBio incubate at 60 °C and 37 °C and read the Terragene® Bionova® SCBI for fluorescent results at the times prescribed in the User Manuals.

Safety Information

To avoid risks and/or damaging the device:

- Do not place the Auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
- Do not place the Auto-reader near devices that emit strong electromagnetic fields.
- Do not use the Auto-reader on leaned surfaces or on surfaces that are subjected to shocks, vibrations, temperature or high relative humidity.
- Disconnect the power cord before cleaning.
- Do not use abrasive or corrosive cleaners or disinfectants.
- Do not immerse into any liquid. Do not pour any liquid inside.
- Make sure the Auto-reader is connected to an appropriate electrical mains outlet socket.
- Use only the included power supply (AC power adapter), power supply's AC plug, power supply cords, and USB cable. Verify that all the included elements are in good condition on daily basis. If any of them is damaged, discontinue their use. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injuries.
- Do not attempt to repair the Auto-reader by yourself. That could lead to major and irreversible damages to the device. In case of device malfunction, contact your local distributor for further assistance.
- Do not plug any other device into the incubator's USB port other than a personal computer (PC). The PC has to be compliant with IEC60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its USB ports. Ask a qualified technician to verify device compatibility. Attaching any other device to the USB port may damage the incubator and may not be safe for the user.

To reduce the risk of using incompletely sterilized loads:

- Please read, understand, and follow the Instructions for Use of each SCBI before its incubation.
- Do not remove the SCBI before the incubator reports the final readout result.
- Check that culture medium completely wets the spore carrier.

To avoid the risk of injury, related to glass fragments produced when crushing the glass ampoule inside the SCBI tube:

- Cool the SCBI during the incubated time before crushing the ampoule.
- Do not handle the SCBI excessively since this might cause the glass ampoule to burst.
- Wear safety gloves and glasses when removing the SCBI from the sterilizer, pressing the SCBI's cap, and crushing the SCBI's ampoule.
- Do not use your finger for crushing the SCBI. Use the Ampoule Crusher instead.

To avoid a potentially hazardous situation:

- Avoid contact with the hot metal block inside each incubation position.
- Do not insert your fingers, or any other element, inside the incubation positions.
- Place only compatible indicators inside the incubation positions.

To avoid SCBIs from absorbing fluorescent particles:

- Avoid direct contact between the SCBIs and Chemical Indicators or Tapes before the SCBI incubation.
- Avoid excessive SCBI handling that may lead to fingerprints or glove tact imprinted over the indicator.

Important:

- Do not use this product in a manner not specified by Terragene® S.A., otherwise the protection provided by the product might be affected.
- Only Terragene® S.A. authorized personnel can access or service the internal components of the Auto-reader. Parts or components inside the Auto-reader should not be manipulated by the user.

Operating conditions

Power supply specifications

Input parameters
Voltage range (100-240)
Frequency
*Current
**Corriente

Output parameters
Voltage range
**Corriente

* 1 Amper in USA and Canada
** 3 Amperes in USA and Canada

Terragene® S.A. recommends the use of UPS instead of voltage stabilizers, since they fulfill two functions: to stabilize and maintain the energy during a power outage.

Environmental operating conditions

Environmental operating conditions

Regulatory compliance

Bionova® MiniBio Auto-reader incubator complies with the following standards and directives:

Electrical Safety
IEC 61010-1
IEC 61010-2-010
Low Voltage Directive 2014/35/EU

Electromagnetic Compatibility (EMC)
EN 61326-1
EMC Directive 2014/30/EU

European Commission
RoHS Directive 2011/65/EU
WEEE Directive 2012/19/EU

Bionova® MiniBio Auto-reader incubator do not represent photobiological risk and do not generate dangerous optical radiation in any of its normal operation conditions as per the requirements of IEC 62471

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

Cleaning and maintenance

Cleaning and decontamination of external surfaces
Disconnect the power supply cable and the USB cable from the Auto-reader. If the device is hot, wait until it has cooled down before handling it.
Clean the external surfaces of the device using a microfibr cloth moistened with a solution of mild dish washing detergent and water. Wring the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the Auto-reader. Afterward, moisten a clean microfibr cloth with water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the Auto-reader to air dry for at least 1 hour before connecting the power supply cable or the USB cable again.
This cleaning procedure can be followed whenever considered appropriate.
This cleaning procedure must be followed every time a spill occurs on any of the external surfaces of the Auto-reader.
If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.

Do not clean the internal parts of the device.
Do not pour or immerse the device into any liquid. Do not allow any liquid to run inside the device during its cleaning.

Maintenance

MiniBio Auto-reader does not require routine maintenance.

Warranty

Terragene® S.A. guarantees both the quality of the product material components and the quality of its manufacturing process. Should any material or manufacturing faults be detected within the warranty period, the only obligation of Terragene® S.A. will be product repairing or substitution.

Term

The warranty period for electronic products marketed by Terragene® will be 1 (one) year from the date of first use of the product, and may never be extended beyond the product's useful life -5 (five) years from its manufacture.

Limitation of liability
Terragene® S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance
Terragene® S.A.
Ruta Nacional N° 9, Km 280 - CP 2130. Parque Industrial Micropi-Alvear-Santa Fe-Argentina.

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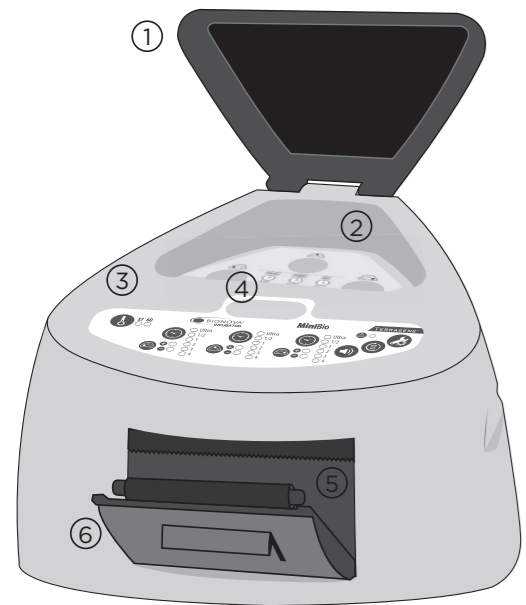
EN Instructions for use

Setting Up your Auto-reader
Place the Bionova® MiniBio Auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or during its use. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

Using your Auto-reader
1 | Power on your MiniBio Auto-Reader by connecting the AC plug of the power supply to the mains and the jack at the other end of the power supply to the rear of the Auto-Reader. (See image 1)
2 | Press the temperature selection button for 3 seconds to enter the temperature selection mode. Press again to select the desired temperature. After 4 seconds the temperature modification will be accepted. (See image 2)
3 | Wait until the temperature stability indicator stops blinking. Select the appropriate incubation time for the SCBI you are planning to incubate by pressing the incubation time selection button. (See image 3)
4 | Place the SCBI in the chosen position. The incubation will start automatically. (See image 4)
5 | Wait until the incubation is completed. The position status indicator light will turn green to indicate a negative result or red to indicate a positive result. If a positive result is detected, press the alarm button to cancel the alarm. Do not remove the SCBI until the ticket with the result of the incubation is printed.
Disposal: Dispose of the SCBIs in accordance with your country's waste-disposal guidelines.

Connectivity
To connect your MiniBio Auto-reader to a PC via a USB port, connect an end of the USB cable (included) to the rear part of the incubator and the other end to the USB port of your PC. Connecting the Auto-reader to a PC allows the use of the Bionova® Traceability Software. Please see the User's Manual for further information.

Product Compatibility and Incubation programs

Bionova® MiniBio Auto-reader Incubation Programs
The following incubation programs are available for your Bionova® MiniBio Auto-reader.

Temperature	Time	Incubation Program
60 °C	20 min. (Ultra)	20 min. at 60 °C
	30 min.	30 min. at 60 °C
	1 hs	1 hour at 60 °C
	2 hs	2 hours at 60 °C
	3 hs	3 hours at 60 °C
37 °C	4 hs	4 hours at 37 °C

! Follow each Indicator's Instructions for use before its incubation.

Bionova® MiniBio Auto-reader compatible indicators

We develop new indicators for extending the features of our Auto-readers regularly. Please visit www.terragene.com/MINIBIO to access the Compatibility Matrix for Terragene® Auto-readers. The Compatibility Matrix specifies all compatible Indicators developed for your Auto-reader up to date. Stay tuned for new and exciting updates!

Troubleshooting
Refer to the user manual for a detailed troubleshooting routine. Please visit www.terragene.com/MINIBIO or scan the QR code to access the Auto-reader user manual.

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