



1 Power on

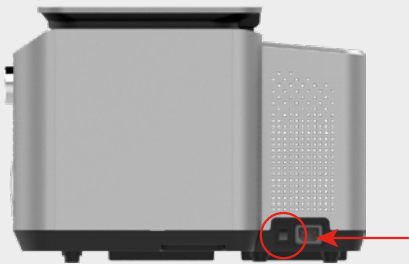

- Turn on the power switch on the side of the sterilizer.

- ✓ Make sure the power line is plugged in.
- ✓ Make sure the power button is on.

2 Preheating

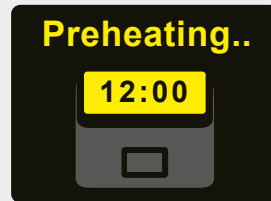

- When the power is turned on, the preheating proceeds automatically.

- ✓ Takes about 15 minutes maximum.
- ✓ Automatic entry to the Ready screen when 100% is reached.

3 Load Cartridge (Step 1)



- Lower the sterilant compartment door handle on the front of the product.



- Hold the door handle and lift it up to open the compartment.

4 Load Cartridge (Step 2)




- Load the sterilant consumables by sliding them into the open compartment.
- ✓ Push it in until it clicks into position.




- After checking that the sterilant consumables are loaded in the correct position, close the ring by pulling it down.

- ✓ Error will occur if it is not closed all the way

5 Sterilization

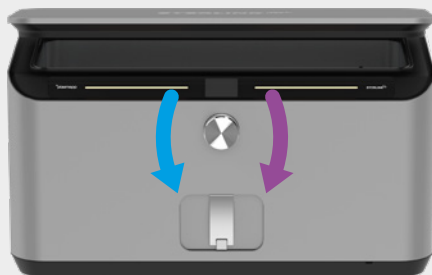

- Hold the door, open it, and load the object to be sterilized.

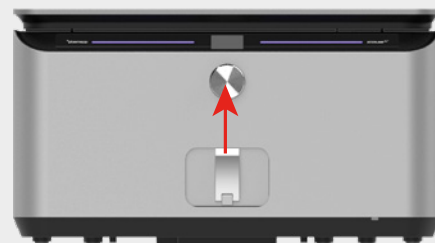

- After loading, close the door and turn the dial to select the desired sterilization process.
- Press the dial to perform the sterilization process.

6 Sterilization Complete

Standard Mode
Sterilization

17 min

Standard Mode
Sterilization

Complete

Advanced Mode
Sterilization

24 min

Advanced Mode
Sterilization

Complete

- When the sterilization process is completed and the 'Complete' screen is displayed, open the door and retrieve the object.

- ✓ The sterilization execution screen for each selected mode and the screen displayed after the sterilization is completed. (Remaining time is shown)

How to Take Action in the Event of Problems

Phenomenon	Description	How to Take Action
 Caution Description	Empty Cartridge	▪ Replace cartridge
	Expired Cartridge	▪ Replace cartridge
	shutdown occurred / during process	▪ Restart after checking the power cable ▪ Call service center in case of recurrence
	Emergency stop (When Purification process is completed)	▪ Restart the process
	sliding door / was opened (When Purification process is completed)	▪ Restart after confirming that the cartridge door is closed ▪ Call service center in case of recurrence
	chamber door / was opened (When Purification process is completed)	▪ Restart after confirming that the chamber door is closed ▪ Call service center in case of recurrence
 Code XX Description	Code. 10 Rebooting or / contact service center	▪ Restart the process ▪ Call service center in case of recurrence
	Code. 20 Base pressure / issue (When Purification process is completed)	▪ Restart the process after removing moisture and oil from the chamber and sterilization object ▪ Call service center in case of recurrence
	Code. 21 Diffusion pressure / issue (When Purification process is completed)	▪ Restart after checking the cartridge status
	Code. 30 Invalid cartridge	▪ Replace cartridge

※ Please refer to your user manual and contact the manufacturer if the issue is not resolved or recurs.

How to Clean

Do not use organic solvents other than aqueous alcohol for exterior and interior cleaning.

- If there is dust, lightly wipe the contaminated area with a wet tissue.
- If the stain is severe or cannot be removed with a wet wipe, use a cleaning sponge with water and wipe off any remaining stains with a cleaning wipe or soft cloth.
- If foreign substances including oil are present, spray disinfectant ethanol on the contaminated area and wipe off the remaining stain with a cleaning wipe or soft cloth.
- In the case of crayons and colored pencils, wipe by gently rubbing the contaminated area with a clean eraser.

※ **Cautions**

- Clean only when the machine is turned off.
- Do not use cleaners containing sodium hypochlorite-based or chlorine-based solutions.
- Do not use ketone substances and substances such as acetone, ethyl alcohol, toluene, methyl acid or methyl chloride.

STERLINK U510 Self Check List (for reference)

NO.	Contents	Check list	Results	
			Yes	No
1	Installation	Have you read the user manual thoroughly before the installation?		
2		Check if the product has been installed on a level surface (within 3 degrees)		
3		Check if the product has been installed with a minimum clearance of 5cm from the back/front/left/right sides		
4		Check if the power cable provided by Plasmapp is being used Check if the power strip or the wall outlet is properly grounded		
5	Using	Are you using a dedicated CARTRIDGE for your product?		
6		Check if the CARTRIDGE has been correctly placed under the loading block		
7		Check if the products are thoroughly dried before initiating the cycle		
8	Clean	Check if the power is turned off		
9		Check if any prohibited chemicals or solutions have been used for cleaning the product		



User Manual



Plasmapp Channel

Manufacturer

Company	Plasmapp Co., Ltd.
Address	(41061) 102, Cheombok-ro, Dong-gu, Daegu, Republic of Korea
Website	www.plasmapp.com