

vive®




# TRANSFER BENCH

Owner's Manual  
LVA1041

[vivehealth.com](http://vivehealth.com)

## WHAT'S INCLUDED

- A) 1 x Small Seat Section
- B) 1 x Large Seat Section
- C) 2 x Bench Legs w/ preinstalled Suction Cup Feet
- D) 2 x Bench Legs w/ preinstalled Capped Feet
- E) 4 x Collared Locking Pins
- F) 1 x Backrest Assembly
- G) 1 x Arm Rest
- H) 2 x Backrest Frame caps

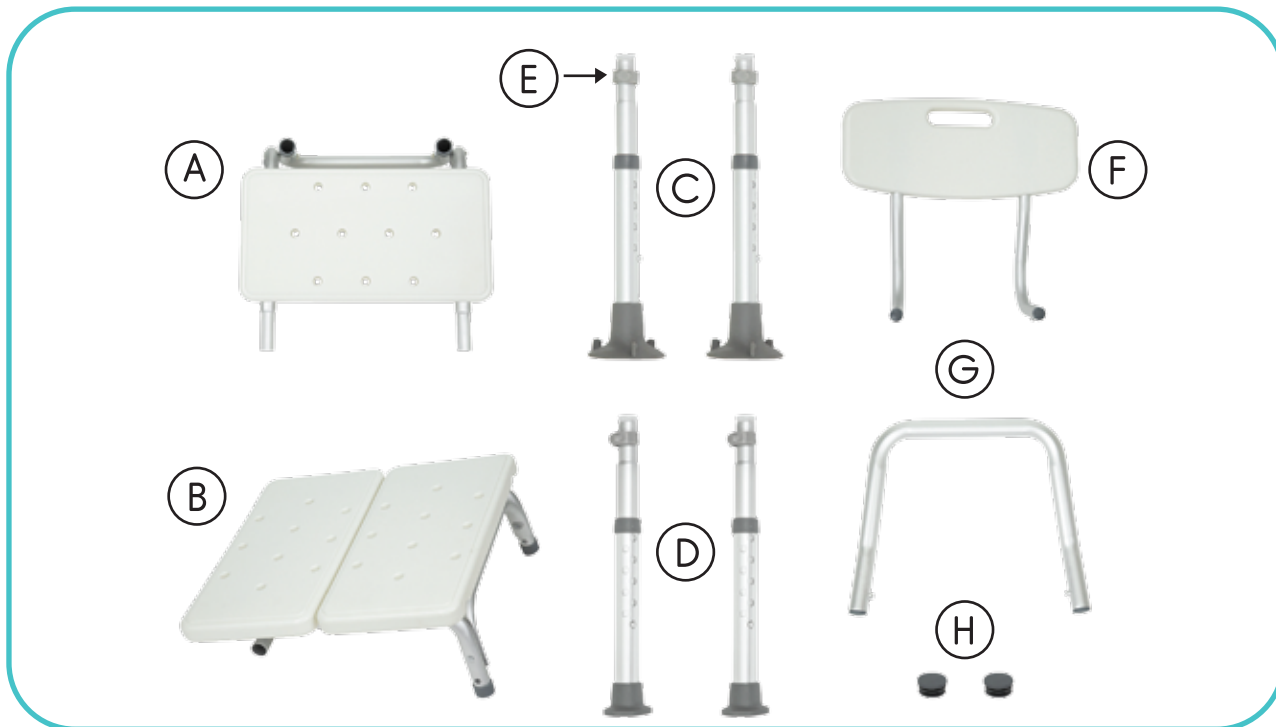

## OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Transfer Bench as easy as possible. Throughout the manual we will include QR codes and links to important videos that you can visit for help.



To see all of the FAQs in one place,  
visit [vhealth.link/3Od1d](https://vhealth.link/3Od1d)

# ASSEMBLY INSTRUCTIONS



To see a video demonstration,  
check out [vhealth.link/O41](https://vhealth.link/O41)

1. Turn the Seat Sections (A) and (B) over so the leg sockets are facing up.

2. Connect the two male frame ends on the Small Seat Section (A) into the female frame ends under the Large Seat Section (B) by depressing the locking pins and pushing the seat sections together until the pins pop into place with an audible click.



3. Fit each Bench Leg w/ preinstalled Suction Cup Foot (C) into the leg sockets on the Small Seat Section (A). Make sure each leg is fitted in place, then align the holes in the Leg with the holes in the socket and secure the pieces in place with a Collared Locking Pin (E).

a. NOTE - The plastic stoppers fitted at the end of each Leg do not need to be removed before installation.



4. Repeat Step 2 with the other Bench Legs (D) in the sockets on the Large Seat Section (B).
  - a. If needed, adjust the height of each leg by depressing the buttons at the bottom of each leg. With the button depressed, slide the leg up or down until the pin aligns with an adjustment hole at the desired height. The pin will click into place. Make sure the legs are adjusted and the seat is level when installed in the bathtub.



5. Turn the bench onto its long side and attach the Backrest Assembly by depressing the locking buttons and fitting the backrest bars into the tubes on the underside of the bench. The backrest can be attached on either side, to make the bench reversible depending on the configuration of your shower.



6. Stand the bench up on its 4 feet and attach the arm rest by depressing the locking buttons and fitting the bar into the sockets on one side of the bench. Make sure the armrest bar curves away from the seat.



7. OPTIONAL - Fit the Backrest Seat Caps (H) into the open tube ends on the underside of the bench opposite from the attached backrest.



## WARNINGS ⚠

- Seat back is not height adjustable
- 350 lbs weight limit
- Wipe down with dry cloth after use
- Make sure all components are securely connected before sitting or leaning on the bench.
- Do not sit on the bench if the seating surface is not level.
- Check periodically to ensure that water is not collecting inside the legs. If water collects inside, remove the legs, drain the water inside, and reattach.

## GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions  
at [vivehealth.com/30dtd](https://vivehealth.com/30dtd) for helpful answers.



And if that doesn't answer your question, our customer  
service team would love to help! Feel free to connect with  
them by phone, e-mail, or chat on our website.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808



[vivehealth.com](https://vivehealth.com)