


WIRELESS PAGER ALARM


User Guide
LVA1074PGR

QUICK SET UP GUIDE

Step 1: Open the battery door of the pager and insert two AA batteries as shown. Close the battery door.

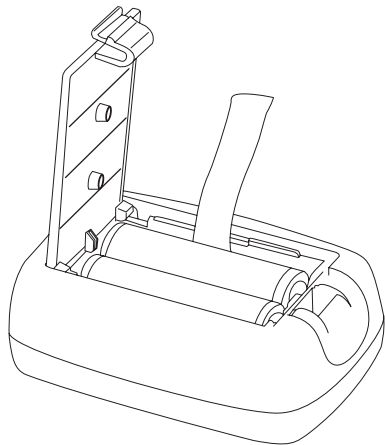


Step 2: On the pager, press and hold both UP and DOWN arrows for 5 seconds. You will see the number "1" is blinking in red, indicating that the pager is now in programming mode.

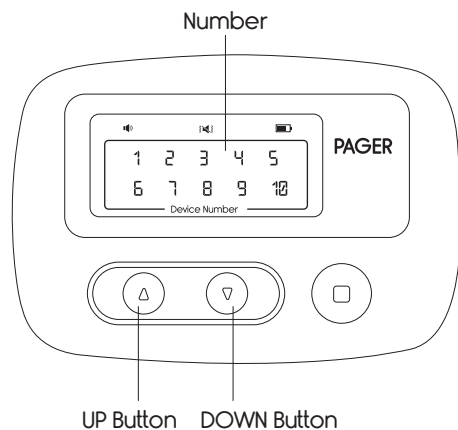

Step 3: There are 10 channels on the pager for programming up to 10 individual wireless devices. Select a number for programming by pressing the UP or DOWN arrow on the front of the pager.

Step 4: While the pager is in pairing mode, a red light should flash over the device number (#1-10) that you are attempting to pair the alarm with. While this light is flashing, press the button on the transmitter once to wake it up, then hold for 5 seconds until the alarm light on the transmitter flashes orange. Then release the button on the transmitter.

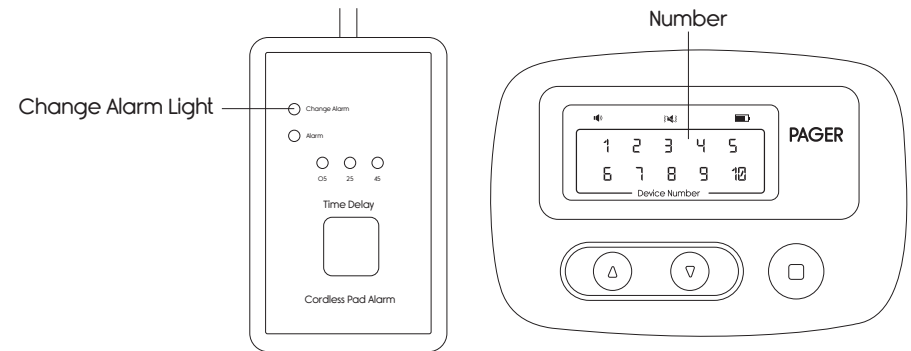




Step 5: The pager will beep to signal a successful connection. Then it will re-enter programming mode in case you need to connect other wireless devices. If you do not need to connect more devices press and hold both the UP and DOWN arrows on the front of the pager to exit programming mode.



Step 6: Test the connection by sitting down on the sensor pad and standing up. The alarm should sound on the pager and the connected channel number should flash in red while the alarm sounds. To silence the alarm press the STOP button on the pager. If this does not occur follow these steps again from the beginning or contact us for help.


Note: This set up is only for the sensor pads with transmitters. If you are trying to pair a call button or door alarm, please refer to those specific instructions. All devices are sold separately.



NOTIFICATION MODES FOR THE WIRELESS PAGER

STEP 1: Set notification mode for the WPG.
The WPG has three notification modes.

Notification Mode	Notification	Sign
Audible notification	Beeps	
Vibration notification	Vibration	
Audible + Vibration notification	Audible and Vibration	

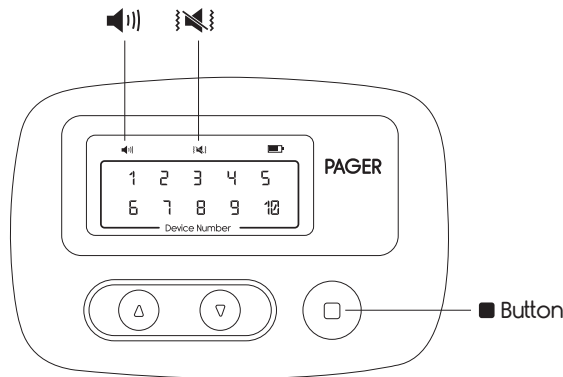To set the notification modes, press and hold down both  Button on the front of the WPG for more than 5 seconds until the  image is blinking.



To select another notification mode, press and release the UP Button or DOWN Button on the front of the WPG. The next notification light  will start blinking. Continue to press and release the Up or Down buttons until the desired notification light is selected. Once you've landed on your desired notification light, press and release the  Button, you will hear a 1 second beep and/or feel a 1 second vibration. This means the current notification mode is recorded to the WPG.

Step 2: Exit programming mode: Press and hold both the UP and Down buttons on the WPG for more than three seconds until you see the number light turn off. The WPG has now exited programming mode.

WARNINGS

This device will not prevent a fall or injury. It is a supplemental part of an overall fall management program.

Failure to follow all product instructions may result in the failure or improper function of the product.

The WPG is able to communicate up to 150 feet away from the patient in open field. The actual communication distance or wireless range will vary from building to building. Test the wireless range in your building prior to putting the system into service. Always keep the pager located within the wireless range.

The pad, transmitter and any other devices should be checked prior to each use for proper functioning.

GUARANTEE

You are protected by Vive Health's industry leading guarantees and customer service.



If you did not purchase through vivehealth.com, please register at vivehealth.com/register to validate your guarantee.

Product Code: LVA1074PGR

 service@vivehealth.com

 1-800-487-3808
Monday - Friday 9am - 5pm EST

vive[®]
vivehealth.com

© Copyright and Distributed by Vive Health
8955 Fontana Del Sol Way, Naples, FL 34109
All Rights Reserved. Made in China.
Email: service@vivehealth.com