

BATHTUB RAIL

QUICK START GUIDE


We are constantly answering questions and recording helpful videos to make using your Bathtub Rail as easy as possible. Check out the included link and QR code to help you through the process.



To see all of the FAQs in one place visit vhealth.link/pO

ASSEMBLY INSTRUCTIONS

No Tools Required! ✂

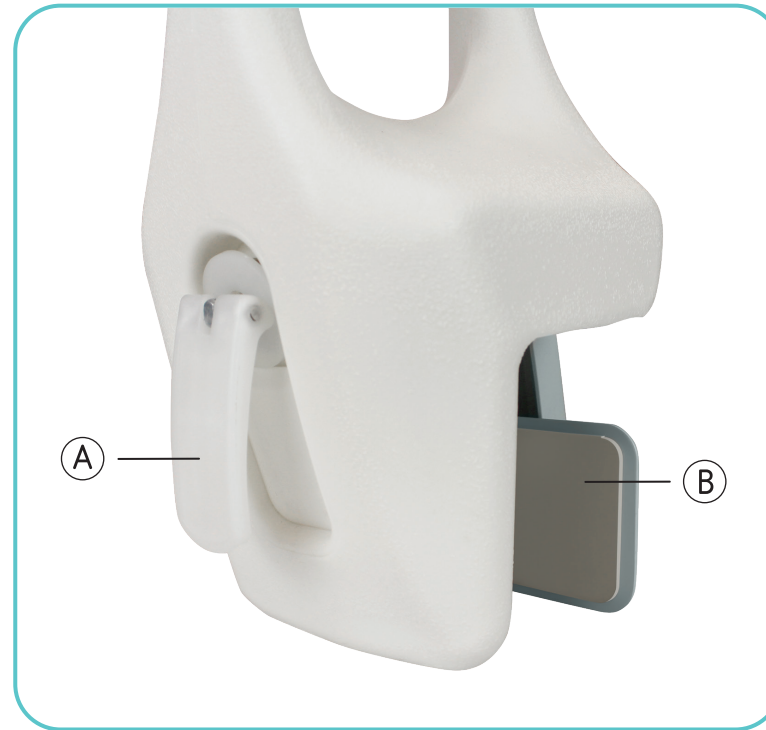

1. Clean bathtub thoroughly and wipe dry before applying rail. Surface must be dry and free of soapy residue for proper installation.
2. Lift up knob (A) and rotate counterclockwise to open. Pull out clamp (B) until rail is wide enough to slip on side of bathtub.
3. Place Safety Rail on bathtub with knob (A) facing outside of tub and tighten clockwise until rail feels tight and secure.
4. Push knob (A) down to original position.
5. Test installation of rail before using. Pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and re-tighten.

WARNING ⚠

- Check periodically before use and re-tighten as needed. Rail may require tightening from time to time.

SPECIFICATIONS

- Height: 21"
- Length: 7.5"
- Depth: 6.5"
- Adjustable Clamp: 3.75" - 6.5"
- Product Weight: 5.2 LBS
- Max User Weight: 300 LBS

A large teal square occupies the right half of the page. In the center of this square, the word "vive" is written in a white, lowercase, sans-serif font. A registered trademark symbol (®) is positioned to the upper right of the letter "e".

vive®