

## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/bf84c](https://vivehealth.link/bf84c) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808



[vivehealth.com](https://vivehealth.com)



X002QZ5R9R

Offset Bariatric Cane  
MOB1042BLU



(01)00810041984555(241)MOB1042BLU

Distributed by

**vive**  
health

8955 Fontana Del Sol Way  
Naples, FL 34109  
1-800-487-3808  
[www.vivehealth.com](https://www.vivehealth.com)  
Made in China



vivehealthusa

# vive®

## OFFSET BARIATRIC CANE

Owner's Manual  
MOB1042BLU


[vivehealth.com](https://vivehealth.com)



To see all of the FAQs  
visit [vhealth.link/bf84c](https://vhealth.link/bf84c)

## FIND YOUR CORRECT CANE HEIGHT


1. Let your arm hang naturally by your waist. Adjust the cane height so the handle touches your wrist.

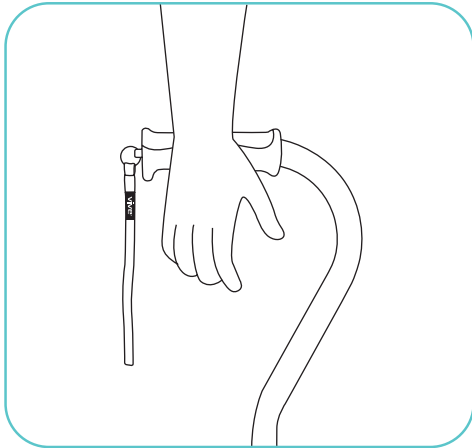

## CANE HEIGHT ADJUSTMENT

1. Loosen the locking latch.

2. Adjust cane so that the top of the handle is as close as possible to the crease between your hand and wrist.
3. Slide extension tube up or down to desired height. Button snap should secure cane into place.
4. Tighten the locking latch.

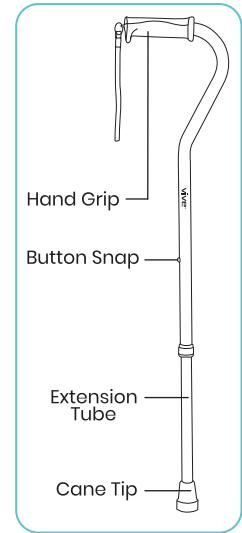

## WARNING

- Make sure the brass pin is locked into place before use.
- Weight Capacity: 600 LBS