




vive®

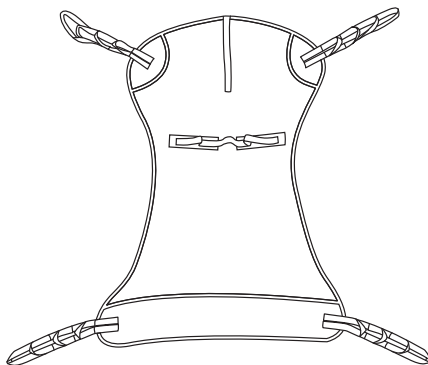

TRANSFER BLANKET WITH HANDLES

Owner's Manual
LVA2000

vivehealth.com

WHAT'S INCLUDED

- 1 pc Transfer Blanket with Handles

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Transfer Blanket with Handles as easy as possible.




To see all of the FAQs in one place visit vhealth.link/6a8ab

HOW TO USE THE TRANSFER BLANKET


Step 1:

Before starting, make sure the bed is flat and the brakes are locked. Roll the patient on their side to one side of the bed.

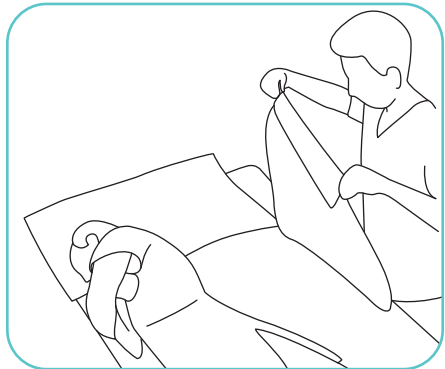


Step 2:

With the patient facing away from you, place the transfer blanket on the bed and tuck it under the patient.

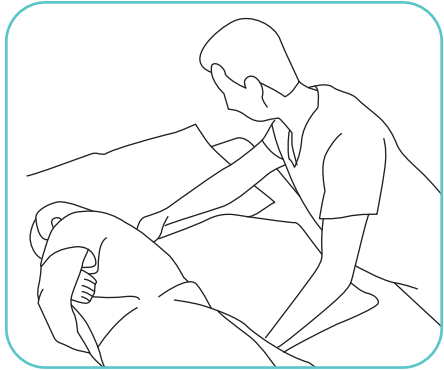


Step 3:

Roll the patient onto their back with the blanket underneath them.

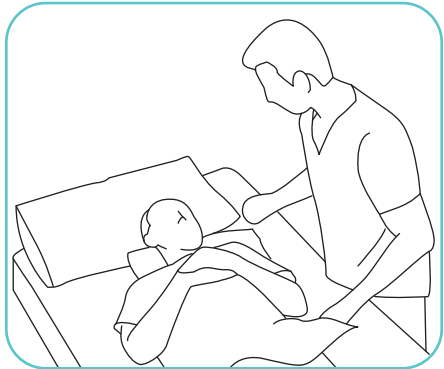


Step 4:

Roll the patient towards you on their opposite side.

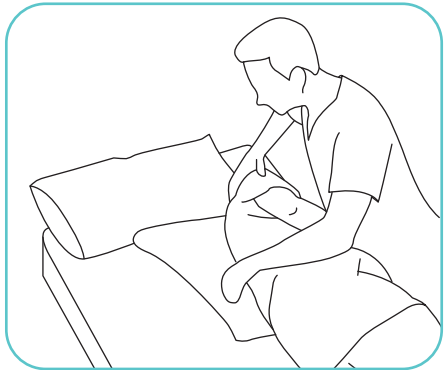


Step 5:

Pull the blanket out from underneath them and straighten it. Roll the patient onto their back. Make sure that the blanket is beneath the patient evenly, with the patient positioned in the middle.



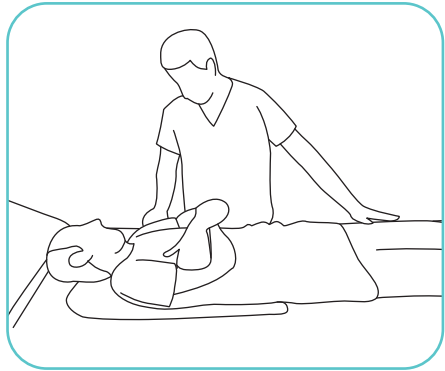
**Step 6:**

This transfer blanket has 4 handle positions for moving the patient. Depending on the weight and size of the patient, you will require the assistance of at least one other person for proper re-positioning. With one person on either side of the bed, grip the handles and lift the patient slowly to transfer to the desired location.

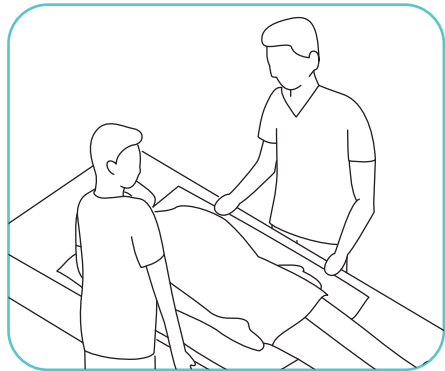

Step 7:

To remove the transfer blanket from under the patient, roll the patient to one side and fold the blanket under them. Roll the patient to the opposite side and slide the blanket out from under them.

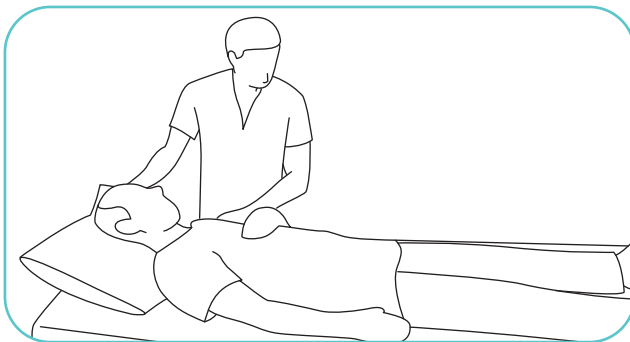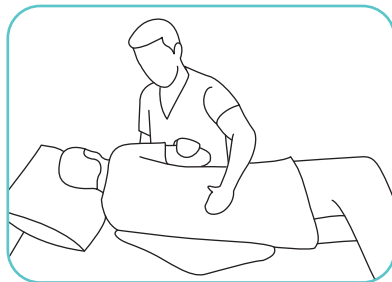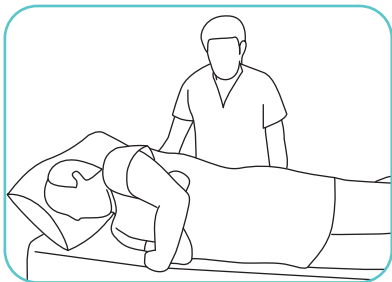

CARE INSTRUCTIONS

- Hand or machine wash on the gentle cycle in cold water with a mild detergent.
- Hang dry or machine dry on low.
- For spot cleaning, wipe the surface with a damp cloth. Allow to air dry.

WARNINGS

- This blanket can support up to 600 pounds.
- Do not try to move a patient without proper training.
- Make sure to review a patient's care plan to ensure the patient should be moved.
- Do not try to move a patient if the patient does not have the strength to roll over or stay on their side.
- Make sure you have the proper equipment and assistance before starting to move a patient.

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vhealth.link/6a8ab for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.

 service@vivehealth.com

 1-800-487-3808

 vivehealth.com