


vive®

FOLDING WALKER

User Manual
MOB1005

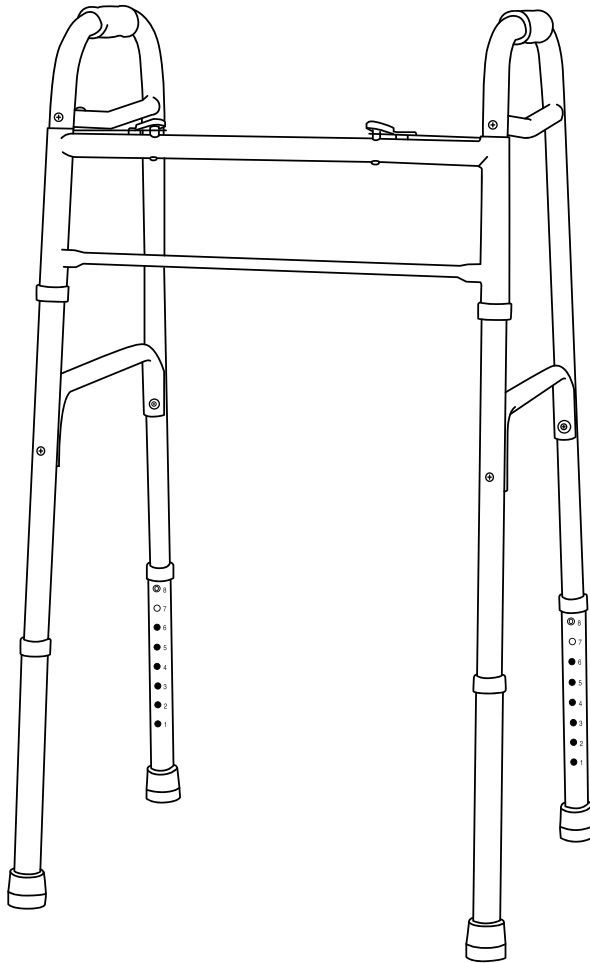

vivehealth.com

PRODUCT DESCRIPTION

The Vive folding walker is a heavy-duty mobility aid designed for ease-of-use and portability. It is extremely lightweight and folds up easily with the aid of a two-button push release system, making it ideal for travel and transporting in and out of vehicles. It is adjustable in 1" increments to accommodate most heights, and with its sturdy frame is capable of supporting up to 250 lbs. Protected by a lifetime warranty.

WHAT'S INCLUDED

- 1x Folding Walker
- Lifetime Warranty

SPECIFICATIONS

- Width - 20"
- Depth - 16"
- Adjustable height - 32" - 39"
- Product Weight - 6.5 LBS
- Max User Weight - 250 LBS

WARNING

Do not push one side of the walker ahead and then other side without lifting the walker first.

Use Vive accessories only. Use of other manufacturers' accessories can result in improper fit and risk fall and/or injury.

Do not use on stairs or escalators.

OPERATING INSTRUCTIONS

Product is fully assembled and ready to use – (This item will require no tools to assemble.)

To fold the walker, simply press down on either of the red tabs on the center support bar and fold the side leg in accordingly to the tab you pressed down.

WALKING WITH THE WALKER

Align the middle of your foot with the back legs of the walker.

Lift the walker up and place it comfortably in front of you, making sure all four legs end up on the floor.

Carefully, walk forward one step at a time.

GUARANTEE REGISTRATION

You are protected by Vive Health's industry leading guarantees and customer service:



If you did not purchase through vivehealth.com, please register at vivehealth.com/register to validate your guarantee.

Product Code: MOB1005



We sincerely appreciate your business.
We strive to provide you with the best
quality products at great value.

If you have any questions please contact us.



service@vivehealth.com



1-800-487-3808

Monday - Friday 9am - 5pm EST