


vive®

# SHOWER STOOL

User Manual

LVA1007

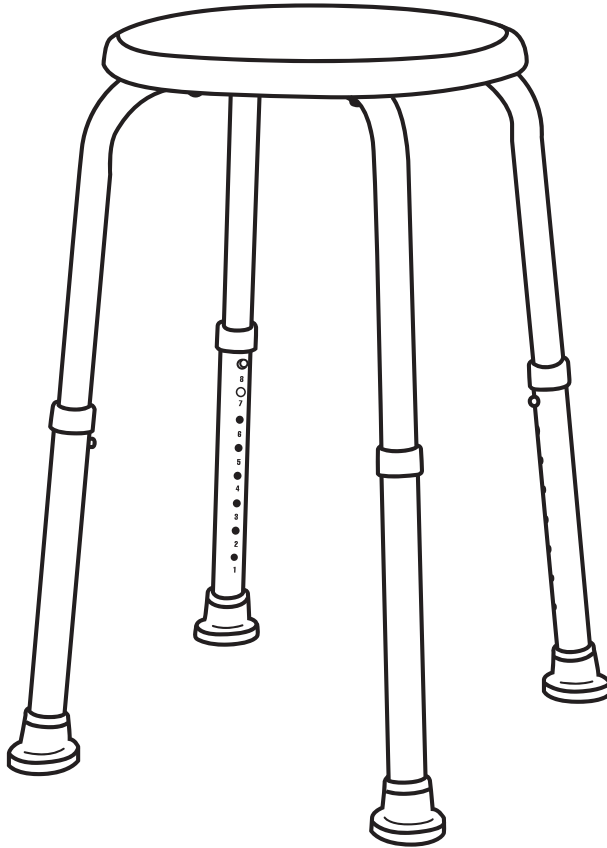

[vivehealth.com](http://vivehealth.com)

## PRODUCT DESCRIPTION


The Vive portable shower stool is a simple solution for a nagging issue. Many tire easily while standing or struggle with their balance and in turn require some form of assistance. A shower stool is a perfect product for these individuals and gives the freedom for people to now shower alone or allow for those who do need assistance to do so while in comfort. This stool is equipped with four anti-slip rubber caps as well as a textured surface to ensure safe functionality. Additionally the height is customizable while the seat measure 12.5" in diameter. Protected by a lifetime warranty.

## WHAT'S INCLUDED

- 1x Shower Stool
- Lifetime Warranty

## PARTS LIST

- A) 1x Seat
- B) 2x Structural Legs with Included Leg Extensions
- C) 5x Hand Knob Screws
- D) 5x Screw Washers

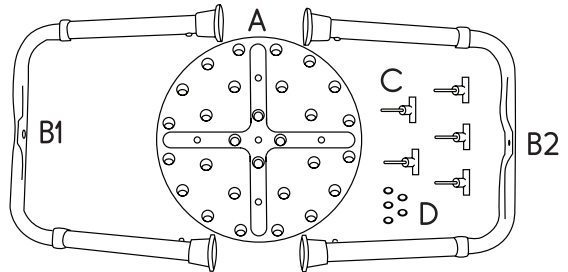

## SPECIFICATIONS

- Seat Diameter: 12.5"
- Width: 14" at full height or 13" at lowest height
- Depth: 14" at full height or 13" at lowest height
- Adjustable Height: 13.5" - 19.25"
- Product Weight: 3 LBS
- Max User Weight: 250 LBS


## WARNING ⚠

- Check periodically to make sure all fixings are tight and secure.

## ASSEMBLY INSTRUCTIONS

No Tools Required! 

1. Remove product from carton and read instructions carefully before attempting to assemble the product. No tools required.
2. Place the round seat upside down on a flat surface so the holes are showing.
3. Align the leg section (B1) that has the concave area on the underside with the holes on the seat. Place and tighten the two hand knob screws (C) in the outer holes, making sure the washers (D) are resting properly in the screw hole pockets. Do not affix the middle hand knob screw yet.
4. Align leg section (B2) that has the concave area on the top side with the holes in the seat. Place and tighten the two outer holes using the hand knob screws, making sure the washers are resting properly in the screw hole pockets.
5. Place the 5th and final hand knob screw through the middle screw hole to secure both legs to the seat.

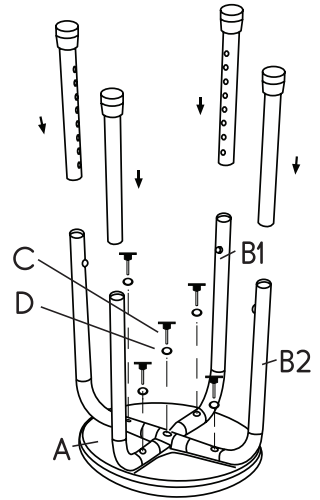

## WARRANTY REGISTRATION

This product is guaranteed to be free from manufacturing defects. We will replace or exchange any product that is defective to your complete satisfaction. This warranty extends for the lifetime of the purchaser, as long as the product has not been misused or abused.

To ensure your warranty,  
please register your product at  
[www.vivehealth.com/register](http://www.vivehealth.com/register).

Product Code: LVA1007



We sincerely appreciate your business.  
We strive to provide you with the best  
quality products at great value.

If you have any questions please contact us.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808

Monday - Friday 9am - 5pm EST