


vive®

# TOILET SAFETY RAIL

User Manual

LVA1010

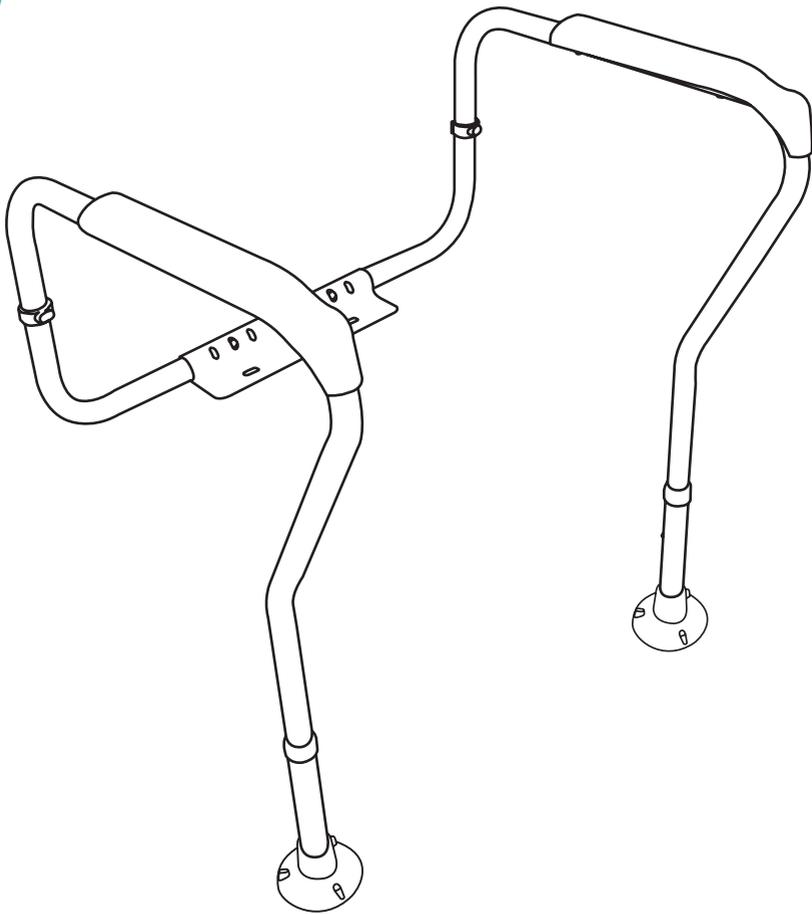

[vivehealth.com](http://vivehealth.com)

## PARTS LIST

A) 1x Rear Bracket

B) 2x Armrests

C) 2x Slide Assist Rails


D) 2x Backrest Screw Knobs

E) 2x Screw-on Suction Leg Tips

F) 2x Armrest Knobs

G) 2x Armrest Screws

H) 2x Armrest Washers

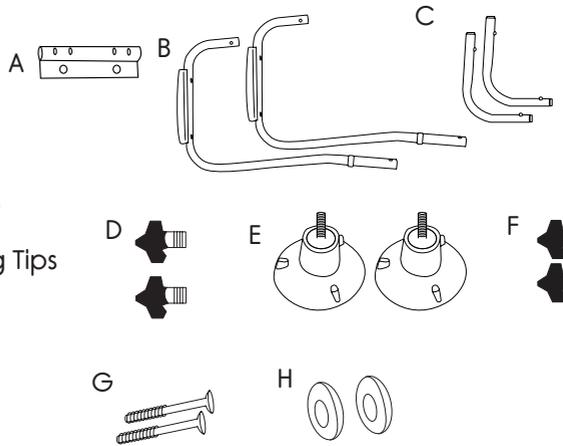

## ASSEMBLY INSTRUCTIONS

▶ Watch at: [bit.ly/toilet-rail](https://bit.ly/toilet-rail)

1. Detach the toilet seat lid from the toilet where you will be using this product.
2. Locate the rear white bracket (A) with the two screw holes on the toilet safety rail.
3. Position the screw holes on the bracket (A) to the screw holes on the toilet, making sure the bracket is flat against the toilet.
4. Add the toilet seat lid on top of the rear bracket (A), align all the holes and fasten with the same screws and bolts. **See Fig. 1.**
5. Screw suction leg tips (E) onto the ends of armrests (B).
6. Attach armrests (B) using the knobs (F), screws (G) and washers (H) to the assist rails (C). Make sure all knobs are secured before use of product. **See Fig 2.**

Fig. 1



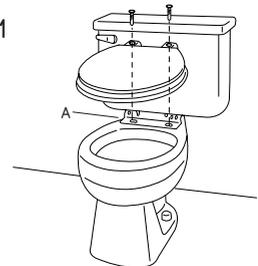


Fig. 2

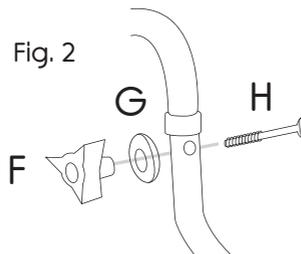

- Slide rear assist rails (C) into rear bracket (A).  
Set arm rests to desired width.  
Use Backrest Screws Knobs (D) to secure  
in place. **See Fig. 3.**

Fig. 3

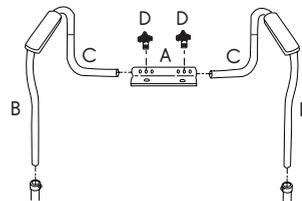

- Adjust assist rail leg extensions according to the height of the toilet.

## WARNING

- Check periodically to make sure all fixings are tight and secure.
- Max User Weight: 300 LBS

## GUARANTEE REGISTRATION

You are protected by Vive Health's industry leading guarantees and customer service:



If you did not purchase through [vivehealth.com](https://vivehealth.com), please register at [vivehealth.com/register](https://vivehealth.com/register) to validate your guarantee.

**Product Code: LVA1010**



We sincerely appreciate your business.  
We strive to provide you with the best  
quality products at great value.

If you have any questions please contact us.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808

Monday - Friday 9am - 5pm EST