

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vhealth.link/1f735 for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

vive®



TRANSFER BELT WITH HANDLES AND LEG STRAPS

Owner's Manual
RHB101L

vivehealth.com

OVERVIEW


We are constantly answering questions and recording helpful videos to make using your Vive Transfer Belt as easy as possible. Check out the included link and QR code to help you through the process.




To see all of the FAQ's in one place visit vhealth.link/1f735

QUICK START GUIDE

1. Insert strap into the buckle through the end with the teeth.

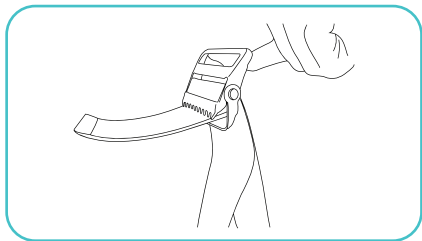

2. Slide strap through the hole opposite the teeth.

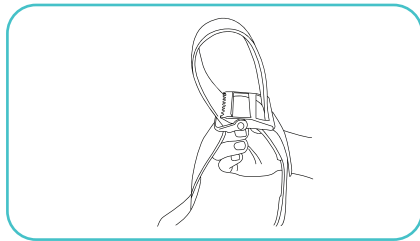

3. Slide leg straps under the patient's legs and buckle the clip. Adjust the strap for a snug fit.

QUICK RELEASE

To quickly release the strap, pull the release tab and slide the strap free.

