

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vivehealth.link/n6j for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com



X002NGV0AJ

Carbon Fiber Cane
MOB1039BLK



(01)00810041983404(241)MOB1039

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China



[vivehealthusa](http://vivehealthusa.com)

vive®

CARBON FIBER CANE

Owner's Manual
MOB1039BLK

vivehealth.com

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Carbon Fiber Cane as easy as possible. Check out the included QR code and video link to help you through the process.



To see all of the FAQs in one place visit vhealth.link/n6j

FIND YOUR CORRECT CANE HEIGHT


1. Let your arm hang naturally by your waist. Adjust the cane height so the handle touches your wrist.



CANE HEIGHT ADJUSTMENT

1. Loosen the locking latch.

2. Slide extension tube up or down to desired height. Button snap should secure cane into place.
3. Tighten the locking latch.

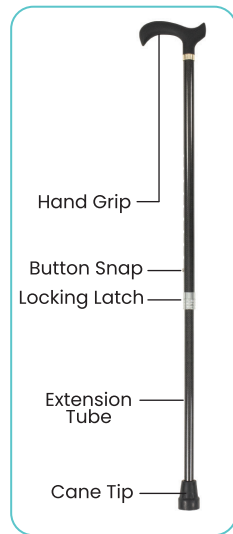

WARNING

- Make sure the brass pin is locked into place before use.
- Weight Capacity: 250 LBS