

OVERVIEW

The Vive CPAP Hose Holder keeps your CPAP tubing tangle-free and out of the way while you sleep. Easy to set up and use, the CPAP Hose Holder mounts to a night stand or headboard, and is bendable to whatever angle works best for your sleeping arrangements.

We are constantly answering questions and recording helpful videos to make using your Vive CPAP Hose Holder as easy as possible. Check out the included link and QR code to help you through the process.



To see all of the FAQs in one place, check out **vhealth.link/72b**

WHAT'S INCLUDED

1x Mount and Bendable Rod 1x Hose Holder Clamp

INSTRUCTIONS

Assemble the CPAP Hose Holder

1. Unscrew the Locking Nut from the Hose Holder Clamp.





Place the Locking Nut upside down on the Bendable Rod.



 Snap the Clamp onto the Bendable Rod and screw the Locking Nut into place.



 Attach the Mount to a sturdy surface such as a headboard, table, or nightstand, and tighten. Do NOT over-tighten, but ensure that the Mount is securely attached before use.



5. The Bendable Rod can be configured however you like. It is recommended to straighten the Rod out as much as possible before further configurations are made.



To hold your CPAP hose, open the clamp and insert the hose.



Note: The CPAP hose should not be pinched or otherwise have the tubing constricted in any way by the clamp.

7. Adjust the positioning of the hose holder as needed.



CARE INSTRUCTIONS

Clean with a damp cloth.

