

vive[®]
MOBILITY



**FOLDABLE WHEELCHAIR
TRANSPORT**

**Owner's Manual
MOB1021**

vivehealth.com

WHAT'S INCLUDED

- Transport Wheelchair Frame
- Foldable Hand Grips (installed)
- Independent Hand Brakes (installed)
- Front and Rear Wheels
- Adjustable Foot Rests
- Folding Foot Plates
- Seat (installed)
- Padded Arm Rests (installed)
- Wrench

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Mobility Foldable Wheelchair Transport as easy as possible. Throughout the manual we will include QR codes and links to important videos that you can visit to help you.



To see all of the FAQs in one place
visit vhealth.link/kq

FOLDING / UNFOLDING THE CHAIR



For a video demonstration,
check out our link here: vhealth.link/eq

From a stable position, grip the armrests or the chair frame under the sides of the seat and pull the frame away from the center of the chair until the fabric seat is fully unfolded.






Grab the folded down hand grips and pull them up until the backrest upholstery is full extended and both hand grip bars snap into place locking them at a 90 degree angle.



To fold the chair, Push the two black tabs beneath the hand grips up and in towards the frame to disengage the locking pins. You can then lower the handles down. Next, lift the chair by the seat strap until the seat folds together, or lift by the arm rests and push the two sides together until the seat fold together.

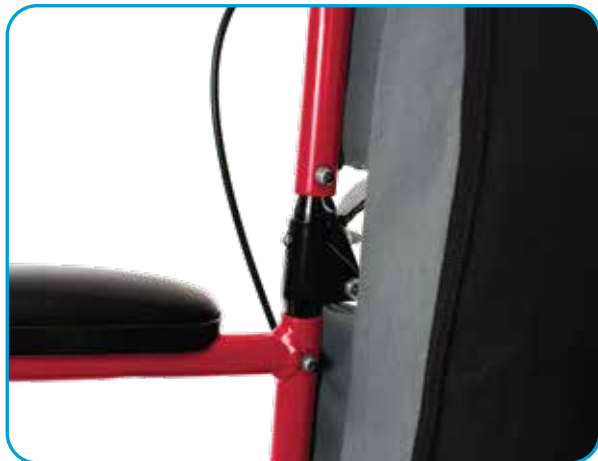

ATTACHING / REMOVING THE FOOT RESTS



For a video demonstration,
check out the link here: vhealth.link/f1

Fit the foot rest over and down onto the two metal placement pins as shown. Then rotate each foot rest forward until the silver clasp locks into place and the foot rest is securely fastened to the chair frame.



To remove the foot rests, pull the wide end of the silver clasp in towards the middle of the chair. You will feel the clasp release its grip on the frame, and the foot rest will swing out where it can be lifted off the two metal placement pins.

Adjusting the Height of the Foot Rests

With the included wrench, grip the bolt on the bottom of the Foot Rest and turn to loosen. Once the bolt is loose enough, the metal support for the foot rest can be slid in or out to adjust the length as desired. Then just tighten the bolt to secure the foot rest and reattach to the chair frame.



MAINTENANCE

Tire Maintenance

Tires should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears. Front casters should be checked periodically for cracking and lubricated occasionally, rear wheels should be inspected at least once a month to ensure proper tightness.

General Maintenance

Wipe metal parts with a soft cloth at least once a week. All aluminum parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

CAUTION

Please note that this chair is designed to use for transporting another individual and is not meant for self transportation. This Transport Chair is designed to safely be a stable center of gravity but it is important to try and keep a stable center of gravity of all times while performing activities that may change your center of gravity or a passengers center of gravity. Never reach out so far that it would require you to shift in the seat or bend over out of the chair. When you have to reach for objects that require you to lean, be certain your front wheels are pointing forward as this will stabilize the chair.

Ramps

Be sure of your own capabilities, limitations and strength before attempting to push another individual up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. If you must stop on an incline, avoid any and all sudden or abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Do not attempt to change direction when moving in incline or decline, this could result in tipping.

WARNING

- Wheelchair will support up to a 300 lb individual. Do not exceed chair weight limit.
- Do not attempt to use wheelchair without reading carefully all instructions in this manual.
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind.
- Always transfer wheelchair user to a vehicle manufacturers approved seat and use safety restraints available.
- Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.
- Do not attempt to operate chair on inclines without proper assistance.
- Wheelchair should be on a smooth, stable and level area with the hand brakes engaged by the transporter before transferring the passenger in or out of the wheelchair.
- When transferring in or out of wheelchair, do not stand on or apply weight to foot rests.
- Do not lean over or apply weight to the top of wheelchair back. This can cause wheelchair to tip over.
- Do not stand on wheelchair.
- Abruptly changing directions while going down an incline could cause instability and/or increase the possibility of tipping over.

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vivehealth.link/kq for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109