

vive®



# FOOT STOOL

Owner's Manual  
LVA1030N

[vivehealth.com](http://vivehealth.com)

## OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Foot Stool as easy as possible. Throughout the manual we will include QR codes and links to important videos that you can visit to help you through the process.



To see all FAQ's in one place  
visit [vhealth.link/9t7](https://vhealth.link/9t7)

## WHAT'S INCLUDED

- 1 Stool Base with Pad (A)
- 2 Leg Sections (B1 - concave, B2 - convex)
- 5 1 inch Hand Screws (C)
- 9 Lock Washers (D)
- 9 Curved Washers (E)



## ASSEMBLY INSTRUCTIONS

1. Turn Stool Base (A) over with leg attachment holes facing up.



2. Fit Concave Leg Section (B1) into groove on underside of Seat (A) with screw holes aligned.



3. Fit Convex Leg Section (B2) across Concave Leg Section (B1) so that center screw holes overlay.  
NOTE: B1 And B2 should interlink at the overlapping portion. If they do not, then the legs have been fastened in the wrong order.



4. Thread the 1 inch Hand Screw (C) through Lock Washer (D) first then follow it by Curved Washer (E).



5. Place down through the center screw hole first. Tighten down securely.



6. Repeat Step 4 with remaining Hand Screws, Lock Washers and Curved Washers and secure them in the remaining screw holes on all four of the legs.



## WARNING

- Stool has a maximum user weight of 300lbs
- Always stand in the middle of the stool and avoid reaching for items that can cause your balance and weight to shift abruptly.

## GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vivehealth.com/9t7](https://vivehealth.com/9t7) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808



[vivehealth.com](https://vivehealth.com)