

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions
at [vhealthJink/bf84c](#) for helpful answers.



And if that doesn't answer your question, our customer
service team would love to help! Feel free to connect with
them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

vive®

OFFSET CANE

Owner's Manual
MOB1002



X000Q15AXB

Offset Cane - Black
MOB1002BLK



(01)00618323026122(241)MOB1002BLK

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com



[vivehealthusa](http://vivehealthusa.com)

vivehealth.com

OVERVIEW


We are constantly answering questions and recording helpful videos to make using your Vive Offset Cane as easy as possible. Check out the included link and QR code to help you through the process.



To see all of the FAQs in one place visit vhealth.link/bf84c

FIND YOUR CORRECT CANE HEIGHT

1. Let your arm hang naturally by your waist. Adjust the cane height so the handle touches your wrist.

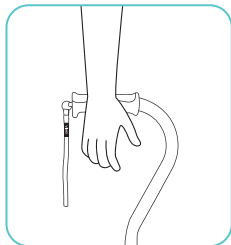


CANE HEIGHT ADJUSTMENT

1. Loosen the locking latch.

2. Adjust cane so that the top of the handle is as close as possible to the crease between your hand and wrist.

3. Slide extension tube up or down to desired height. Button snap should secure cane into place.

4. Tighten the locking latch.

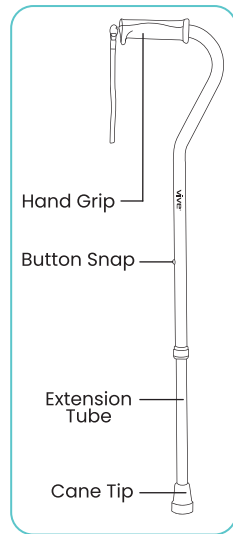

WARNING

- Make sure the brass pin is locked into place before use.
- Weight Capacity: 250 LBS