GOT **MORE QUESTIONS?**

Check out our list of Frequently Asked Questions at vhealth.link/864f4 for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com



X001AK0LJX

Quad Cane - Black MOB1003BLK



Distributed by





VIVE QUAD CARE

Owner's Manual MOB1003

vivehealth.com

BEFORE YOU USE THE CANE

It's important that you secure the base and the feet by placing it on a flat surface and pressing down firmly on all four legs.


FIND YOUR CORRECT CANE HEIGHT

 Let your arm hang naturally by your side.
Adjust the cane height so that the handle touches your wrist.



ADJUST YOUR CANE HEIGHT

- 1. Loosen the locking ring.
- 2. Depress the locking pin.
- 3. Slide the extension tube up or down to your desired height.
- 4. Make sure the locking pin locks in place
- 5. Tighten the locking ring.

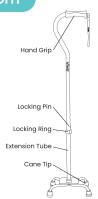

WARNING

- Make sure the locking pin is locked in place before use.
- Make sure the locking ring is securely tightened before use.
- Secure the base and the feet by placing it on a flat surface and pressing down firmly on all four legs before use.
- · Weight capacity is up to 250 pounds.