

vive®



ROLLATOR SEAT BAG

Owner's Manual
LVA2O37BLK

vivehealth.com

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Rollator Seat Bag as easy as possible. Check out the included link and QR code to help you through the process.



To see all FAQ's in one place,
visit vhealth.link/3hz

INSTALLATION

Secure your bag to a rollator by following these steps.

1. Loosen all straps and unclip the buckles.



2. Lift your rollator seat and position the bag centered.



3. Secure the buckle straps to the rollator frame at the front and back. If your seat does not come off without tools, you may leave the middle strap at the back unattached.



4. Tighten the straps to keep your bag secure during use. Put the seat down and make sure it closes properly, if not readjust the straps as necessary.



CARE

Spot clean with a damp cloth and air dry. Do NOT machine wash or dry. Do NOT bleach.

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vivehealth.link/3hz for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.

 service@vivehealth.com

 1-800-487-3808

 vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109