

vive®



TOILET SEAT RISER

Owner's Manual
LVA1070

vivehealth.com

WHAT'S INCLUDED

- 1x Hinged Toilet Seat Riser
- 2x Long Screws (flat head - 6.125")
- 2x Round Plastic Washers (1.18" outer diameter)
- 2x Round Metal Washers (1.25" outer diameter)
- 2x Wing Nuts

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Toilet Seat Riser as easy as possible. Throughout the manual we will include QR codes and links to important videos that you can visit to help you.



To see all of the FAQs in one place visit vhealth.link/898a3

INSTALLATION INSTRUCTIONS




Want to watch us do this? Check out vhealth.link/c31fc for an easy instructional video

1. Unfasten the existing toilet seat and lid from the toilet and set aside.
2. Place the toilet seat riser on the toilet rim with the black grip pads down and align screwholes in back with seat mounting holes on toilet.
3. Attach the original toilet seat and lid on top of the toilet seat riser using the included hardware. Thread the screws through the holes in the back of the seat.

*Note: The plastic washers are normally placed on the underside of the toilet between the toilet and the metal washer, but placement can be adjusted depending on fit.

4. Tighten each screw with the included wing nuts.

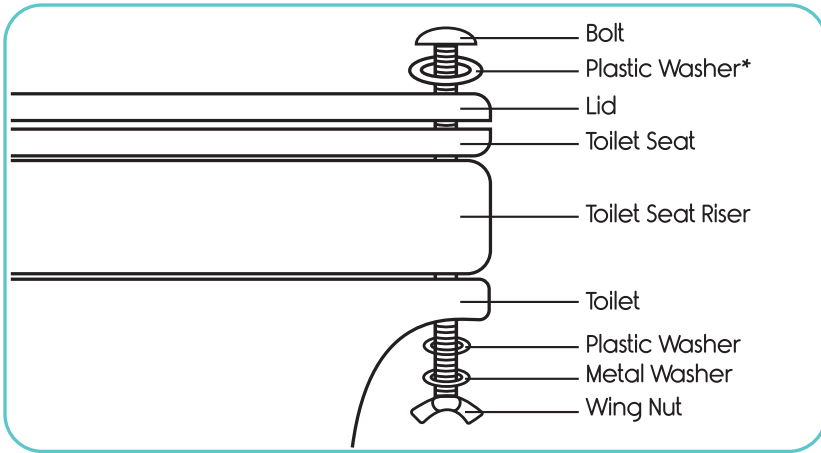

FAQ

Q: Does this include a seat / lid?

A: No, but the riser is designed to fit under your existing toilet seat / lid.

Q: What are the slots on the side of the riser for?

A: These are handles included to help lift the riser for cleaning?

Q: How do I clean the riser?

A: The riser doesn't need any special cleaners or techniques, and can be wiped down with a traditional disinfectant.

SPECIFICATIONS

Dimensions Of Standard Seat:

- Length of seat from front to back: 17.25"
- Length of seat inside from front to back: 11.25"
- Width of seat from left to right: 13.75"
- Width of seat inside from left to right: 9.75"
- Height of seat: 3.5"
- Thickness of seat: 2"

Dimensions Of Elongated Seat:

- Length of seat from front to back: 19.25"
- Length of seat inside from front to back: 13.25"
- Width of seat from left to right: 14"
- Width of seat inside from left to right: 10.5"
- Height of seat: 3.5"
- Thickness of seat: 2"

Weight Capacity: 350 Pounds

WARNING

- Always make sure the toilet seat is securely fastened and in the correct place before using.
- Do not use this raised toilet seat if it does not fit the style of toilet you own.

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vivehealth.link/898a3 for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.

 service@vivehealth.com

 1-800-487-3808

 vivehealth.com