



vive®

TRANSFER DISC

Owner's Manual
RHB2022BLK

vivehealth.com

INTRODUCTION

The Vive Transfer Disc helps transfer patients with impaired lower body mobility or limited upper body function due to paralysis. Assistance is required to use this tool.

We are constantly answering questions and recording helpful videos to make using your Vive Transfer Disc as easy as possible. Check out the included QR code and link to help you through the process.



To see all FAQ's in one place
visit vhealth.link/cs1

WHAT'S INCLUDED

- 1pc Transfer Disc

USING THE VIVE TRANSFER DISC

Single Leg Transfers (Assistance Required)

For use with patients who can fully bear weight onto one leg.

1. Ensure the patient is wearing footwear with good grip.

2. Place the disc on the floor in front of the patient's stable foot, then place the patient's unstable foot onto the middle of the disc.



3. Place one of your feet on the disc to act as an anchor while the patient stands. This will help stabilize and prevent any loss of control.



4. Assist the patient into a standing position.



5. Help the patient turn towards the intended location.



Full Transfers (Assistance Required)

1. Ensure the patient is wearing footwear with good grip.

2. Place the disc on the floor in front of the patient's feet, then place both of the patient's feet onto the middle of the disc.



3. Place one of your feet onto the disc to act as an anchor while the patient stands. This will help stabilize and prevent any loss of control.



4. Assist the patient into a standing position.



5. Help the patient turn towards the intended location.

CARE

- Clean with disinfecting spray and wipe down.

WARNINGS ⚠️

- When helping patients stand up, always keep a foot on the disc to act as an anchor to help stabilize the patient and prevent loss of control.
- Patients should always be using footwear with good grip.
- DO NOT allow patients to self transfer.

GOT MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vhealth.link/cs1 for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China