SL2110

Culla pediatrica altezza variabile con trendelenburg
Pediatric cradle height adjustable with trendelenburg

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Thanks for choosing "GIVAS SRL"!

GIVAS SRL is committed to continuously improve its products and its service for a full customer satisfaction. Our service centres are at your disposal whatever installation, use or maintenance needs you may have.

Copyright © 2016 GIVAS S.r.l.

The reproduction of images, texts and other parts of this manual is prohibited without prior written permission from "GIVAS SRL".

This material is protected by international Copyright laws. The information contained in this manual can be modified or updated without notice.

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QUALITY CERTIFICATION BUREAU ITALIA

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EN

Contents

Ecc declaration of conformity	28
General provisions.....	29
Purpose of this manual	29
Customer service.....	29
Definitions.....	29
Safety	30
General rules.....	30
Introduction.....	31
Description of the crib	31
Reference standards.....	31
Environmental restrictions for use.....	31
Intended Use	31
Manufacturer	31
Life foreseen.....	32
Markings	32
General description	33
Technical features	33
Names of the main parts.....	34
Transportation and packaging.....	34
Packing list verification	35
Installation	36
Preparation of the area for installation	36
Check equipment supplied	36
Assembly.....	36
Testing.....	36
Operation and use	37
How to lock the crib.....	37
How to lift and lower the crib	38
Trendelenburg and reverse-trendelenburg	39
Accessories	40
SZ0095 Lower bathtub for pediatric crib.....	40

Disinfection	42
Sanitization products	42
Sanitization intervals.....	42
Maintenance	43
Periodic review.....	43
Cleaning and disinfection.....	43
Storage.....	43
Setting the item aside	43
Disposal	43
Certificate of warranty	44

Ecc declaration of conformity

Company: Givas s.r.l.

Address: Viale Veneto, 2 Z.A.
35020 Villatora di Saonara (PD)

Products: SL2110
Pediatric cradle height adjustable with trendelenburg

**Progressive system
attributed to MD:** 383993/R

CND code: Z12080499

The products described above are in conformity with:

Document No	Title	Edition / Date of issue
93/42/CEE	Directive 93/42/CEE of Council, of 14 th june 1993 and subsequent changes additions, concerning the Medical Devices.	14 th june 1993

Additional information

The products are included amongs the aids regulated by EEC Directive n. 42/1993 - class 1, and are subject to the procedure provided for by Art. 11 point 5 of 93/42/CEE and subsequent changes additions (Directive 2007/47/CE).

Saonara, on 22nd November 2016

Sole Director Berto Silvio

General provisions

Purpose of this manual

This manual is an integral part of the article.

Read all warnings and instructions very carefully as they provide important safety precautions and information about the USE and MAINTENANCE OF THE PRODUCT.

Descriptions and illustrations contained in this manual are not binding; "GIVAS SRL" reserves the right to make any modification in order to improve this document without a commitment to updating it.

The illustrations and images used in this manual are example only and may differ in practical application.

In no event shall the manufacturer be liable for any consequential or incidental damages, including any loss of business profits or any other commercial damage that comes from improper use and non-compliance to the instructions described in this manual.

Customer service

The customer and product assistance services are important parts of "GIVAS SRL" company structure.

Customer service is readily available for further information on use, maintenance and service of this product.

Definitions

The following graphic and linguistic definitions have been used in this manual:

CAUTION! *This message may appear before some procedures. Non-compliance may cause damage to the article.*

WARNING! *This message may appear before some procedures. Non-compliance may cause injury to the operator and to the patient, and may cause damages to the article.*

Safety

General rules

WARNING! Improper use of the article and improper maintenance can cause damage to people or objects.

WARNING! IN CASE OF BLOODTRANSMITTED DISEASES: to reduce the risk of exposure while using the armchair, carefully follow the maintenance instructions written in this manual, and the personnel safety regulations as established by the person in charge of the Medical Emergency Service.

Operators must read and follow this manual carefully, and familiarise themselves with the correct operating and maintenance procedures of the bed.

Always use and carry out the maintenance procedures of the article as prescribed in this manual, and use only "GIVAS SRL" spare parts and "GIVAS SRL" customer service centre.

Do not use the bed for any purpose other than that for which it was intended and designed.

Always advise the patient before adjusting the bed.

While stopped always secure the brakes to lock the bed.

Never leave the patient on the bed unattended.

Keep this manual for future reference and to support personnel training. Keep this manual together with the product in the event that you sell or give the product to a new user.

Introduction

Description of the crib

The variable height crib for babies has an epoxy powder coated steel section structure, with a "C" shape that allows to bring the crib next to the bed, with 4 double swivel wheels with a 75mm diameter all with independent brake and anti-trace rubber wheel tread.

The big painted aluminum extruded column assures good stability. Lifting is achieved by means of gas springs millimetrically adjustable through a bilateral lever.

The oscillating tub support with safety lock and leveling indicator, adjustable and lockable with handwheels located on both ends; the crib can be positioned horizontally, in trendelenburg (max -13°) and in counter-trendelenburg (max 13°) position.

The removable tub is made of non-toxic and shatterproof thermoformed polycarbonate, it is see-through and allows to monitor the baby; with front name label and air exhaust and aeration holes included on the bottom, and assures safety for the baby and for the operator since its round shape makes it easy to clean.

Reference standards

This item has been designed and manufactured by GIVAS S.R.L. in conformity with the safety provisions of directive 93/42/EEC dated 14.06.93 (class I) and s.m.i. (Directive 2007/47/EEC, concerning medical devices.

Environmental restrictions for use

The environmental conditions for crib use are listed below:

temperature: 10-40°;

humidity: 10%, 90% (not condensed).

The baby crib is not suitable for use in areas subject to explosion hazards.

Intended use

- Device type: baby crib.
- The device is intended to be used exclusively as a baby crib during diagnosis, treatments and to monitor the newborn baby during the first months of life.
- Usage environment: hospital in application environment 2.
- Product to be used by: specialised operators and doctors.
- Supervision and responsibility: the baby crib must be used under a doctor's supervision.
- Warning: the baby crib cannot be used in a potentially explosive atmosphere.
- Usage limitations: the baby crib can be used only according to the present manual.

Manufacturer

The article described herein is manufactured by:

V.le Veneto, 2 - 35020 SAONARA Z.A.I. - PADOVA ITALIA

Tel. +39 049 8790199 Tel. +39 049 8790710

Fax +39 049 08790711 e-mail: info@givas.it

www.givas.it

EN

Life foreseen

Givas baby crib is designed to operate for 10 years without danger or risk to persons and things within the limits and conditions set forth herein, after which it is recommended to replace the crib. Proper operation is guaranteed only by contacting our Customer Service Srl GIVAS whenever breakdown of the crib occurs.


Markings

CAUTION! The tag on the device must not be removed.

The item is identifiable by the tag, placed on the item, where you can find the following data:

1. Name and address of the manufacturer;
2. EEC Marks;
3. CAUTION: read the instructions for installation and use;
4. Model number;
5. Serial number;
6. Lot type;
7. Date of manufacturing;
8. Safe operating load;
9. Maximum patient weight.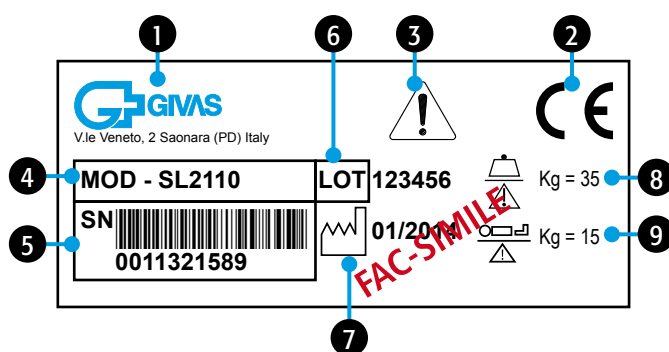

General description

Technical features

Dimensional Technical Features

CODE		SL2110
Dimensions(length x width)	mm	790 x 581
Max height from the ground	mm	920 ÷ 1190
Travel range	mm	270
Trendelenburg angle	deg	13
Reverse-trendelenburg angle	deg	13
Safe work load*	Kg	35
Maximum patient weight	Kg	15
Weight	Kg	18
Wheel diameter	mm	75

* SAFE WORK LOAD results from: patient (15 Kg) and accessories (20 Kg).

Technical data of the mattress*

Dimensions	mm	670 x 400
Thickness	mm	60
Fire reaction class	-	1IM
Density	kg/m ³	31,5
Composition	-	polyurethane
Manufacturer	-	Hurry Up Fedars s.r.l.

*the mattress is sold separately.


WARNING! *Mattresses with technical features and dimensions that are different from the ones specified on the use instructions may be risky for the patient.*

Names of the main parts

1. Base;
2. Crib tub;
3. Brakes;
4. Wheels;
5. Knobs for trendelenburg reclining regulation;
6. Height regulating levers;
7. Lower bathtub for pediatric crib (accessory SZ0095).

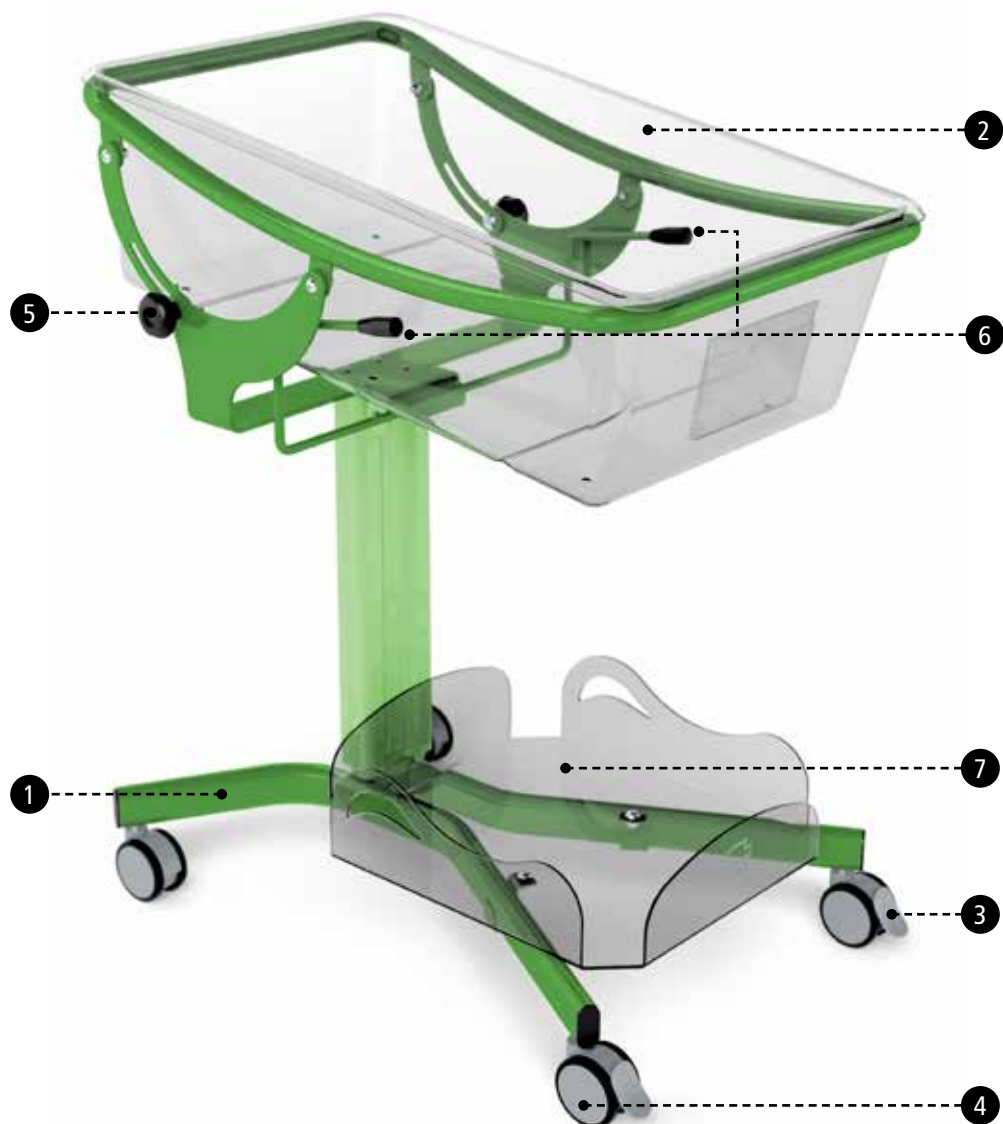Transportation and packaging

WARNING! *Lifting and handling operations must be conducted by skilled personnel trained for this purpose.*

Delivery of this item includes an option of: road, rail, sea and air.

The item is packed in recyclable cardboard boxes, designed for handling.

Handling must be carried out by the suitable equipment.

The equipment suitable for handling are as following: Self-propelling lift truck, hand lift truck.

Packing list verification

Remove the content of the package and check it against the packing list.

If all components are present and undamaged, the packaging can be disposed of or recycled in proper collection areas inaccessible to children and animals.

If the article has been damaged during shipment, keep the package and the rest of packaging, and attach to the provisions contained in the terms of sales.

Installation

Preparation of the area for installation

The place where it is to be installed must:
have a rigid, flat and level floor.

Check equipment supplied

The package includes:

- a crib;
- a Plexiglas tub;
- other accessories ordered;
- instruction manual for use.

Assembly

! CAUTION! *The assembly area must be clean and free of obstacles; there must be the necessary space to allow assembly operations.*

The assembly area must have the following features:

- flat non-yielding floor;
- 400 LUX light;

How to insert/remove the tub from the crib

Follow the instructions below to **insert** the tub in the crib:

- grab the tub on the sides and make it lay into the support of the crib.

Follow the instructions below to **remove** the tub from the crib:

- Remove the tub from the support holding it on the sides.

Testing

! CAUTION! *The following check must be repeated regularly in order to verify the perfect working condition of the article.*

Before using the article:

- perform the "Periodic review" provided for in "Maintenance" section;
- if the check is positive, the article is ready for use, otherwise call the "GIVAS SRL" customer service centre immediately.

Operation and use

Warnings for use

The crib cannot be used in areas where there are explosion and fire hazards.

*Sanitize the crib following the instructions in the relevant chapter **SANITIZATION**.*

The brakes must be activated while the crib is not being used. The crib with a patient must never be left unattended.

Just use the crib for intended use and avoid using it for purposes for which it has not been designed.

How to lock the crib

The locking device of the wheels allows to:


- leave the wheels free in position **(A)**;
- lock the wheels in position **(B)**.

Follow the instructions below to **lock** the crib:

- with your foot push the brake pedal **(3)** until position **B** is reached.

Follow the instructions below to **release** the crib:

- with your foot push the brake pedal **(3)** until position **A** is reached.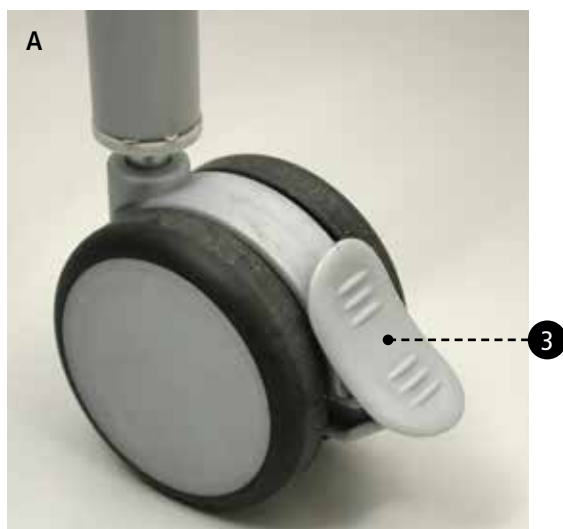


How to lift and lower the crib

The height of the crib can be regulated by using the two levers (6) on the side of the tub support.

How to lift the crib

Follow the instructions below to lift the crib:

- grab the two levers (6) on the side of the tub and pull the crib upwards until the desired position is reached. Release the levers. The device will remain locked in the position chosen.

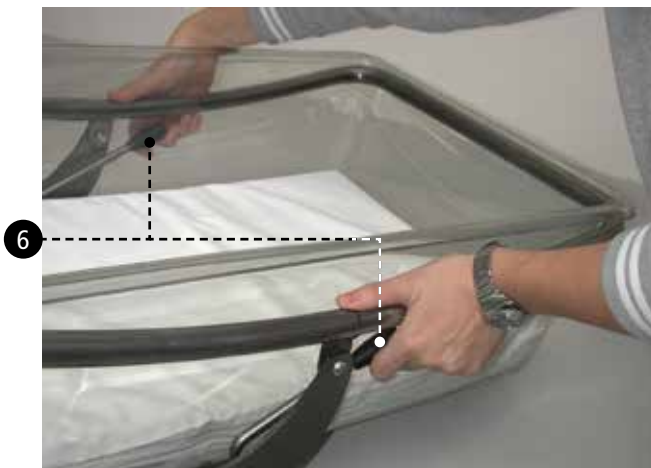How to lower the crib

Follow the instructions below to lower the crib:

- grab the two levers (6) on the side of the tub and push the crib downwards until the desired position is reached. Release the levers. The device will remain locked in the position chosen.

Trendelenburg and reverse-trendelenburg

To regulate the trendelenburg position use the two knobs (5) on the sides of the tub support.

Tendelenburg / Reverse trendelenburg position

Follow the instructions below to regulate the trendelenburg or reverse trendelenburg position:

- turn the knobs (5) counter clockwise contemporarily;
- recline the crib tub until the position desired is reached;
- fix in position by turning both knobs (5) clockwise.

Accessories

SZ0095 Lower bathtub for pediatric crib

Technical introduction

The lower bathtub for paediatric crib is constituted by one tub and two painted steel supports.

Directives reference

This item by GIVAS SRL in conformity with the safety requirements within the meaning of directive 93/42/EEC dated 14.06.1993 (class I) and following mod. supplements, concerning medical devices 2007/47/EEC.

Intended use


The lower bathtub for pediatric crib has been designed and built to be installed by nursing personnel on GIVAS SL2110 paediatric cribs. The lower bathtub for paediatric crib must be used as indicated and in compliance with the restrictions indicated in this manual.

Materials used

The lower bathtub for paediatric crib is made of high-impact Plexiglas.

Names of the parts

1. Bathtub;
2. Bathtub supports;
3. Fixing screws.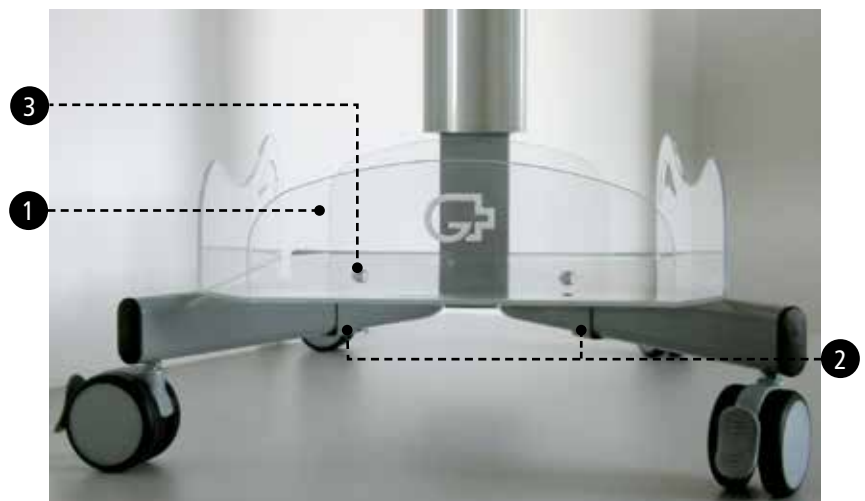

Technical characteristics


Technical characteristics	mm	450x400
Safe operating load	kg	20

Identification

CAUTION! The tag on the device must not be removed.

The following label is attached to the item: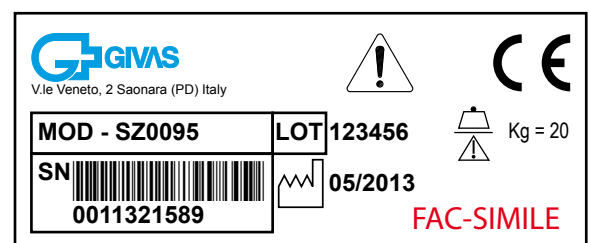

Inspection of parts included in the supply

The package contains:

- Plexiglas bathtub;
- supports with screws;
- use manual.

Preparation of the installation area

The lower bathtub for paediatric crib is to be installed on the "GIVAS" SL2110 crib with the relevant supports.


Installation of the lower bathtub for paediatric crib

WARNING! *This item must be installed only by qualified personnel.*

Follow the bathtub installation instructions below:

- position the supports under the base of the crib next to the holes of the tub and fix the screws as shown in the figure below.

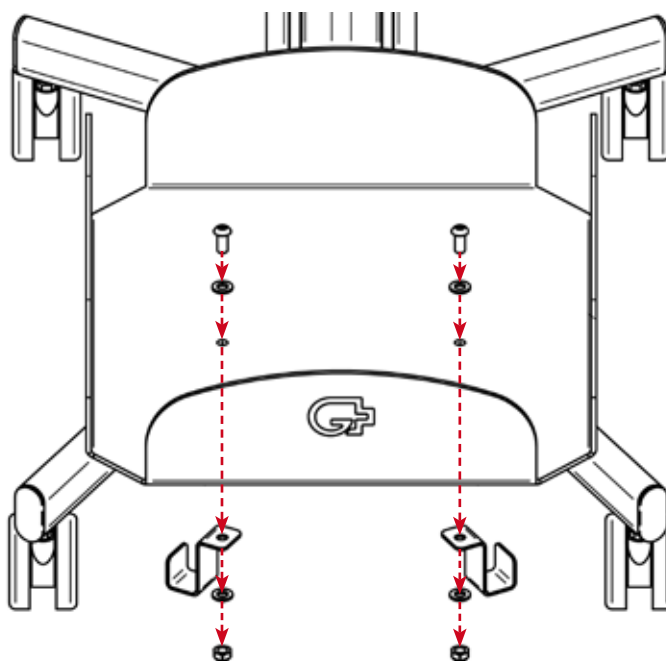Functional test

CAUTION! *This test is to be repeated periodically to check the efficiency of the product.*

CAUTION! *The lower bathtub for pediatric crib must be used only for the purposes prescribed in this manual.*

Before using the crib:

- check that bathtub is installed properly referring to the "Installation of the lower bathtub for paediatric crib" paragraph;
- if the check is passed the item can be used, otherwise contact immediately the Customer Service of GIVAS SRL.

Disinfection

Sanitization products

CAUTION! *Sanitization agents are corrosive.*

Carefully observe the instructions provided by the manufacturer of the sanitization agent during use. Whenever possible, ask the manufacturer for guarantees in regards to the degree of corrosion of the cleaning agents. It is extremely important to follow the specifications regarding the concentration, temperature and reaction times.

Any modification of these features may damage the item.

In order to sanitize the product use a solution of water and sodium hypochlorite at 1,5%.

Sanitization intervals

The user defines the sanitization intervals to be adopted, based on their requirements, the information reported in this manual and the instructions listed on the selected sanitization products.

Maintenance

Periodic review

The users must control the product at least once a year; the check must see that there is not damage that can compromise integrity and proper operation of the product. I.E.:

- screws properly fixed;
- correct movement functionality, with reference to the section in this manual called "Operating and Use";
- correct insertion and fixing of accessories;
- cleaning of wheels and of the product (for instructions refer to the paragraph "Cleaning and sanitization").

 **CAUTION!** *If there is any sign of damage, disable the item, until the necessary repairs or replacements have been completed.*

 **CAUTION!** *It is recommended that any technical adjustments, as well as any repairs of the item not specifically listed in this section is carried out by qualified technicians.*

Cleaning and disinfection

For a better and long life of the product it is essential to carry out periodically a careful general cleaning. We recommend you to proceed as follows.

 **CAUTION!** *Do not sprinkle directly detergent on the mechanical parts. UNDER NO CIRCUMSTANCES USE any solvents, detergent or strong abrasive agents to eliminate "hard-to-clean" spots and stains.*

If need be or every 15 days:

- clean the metal parts with water and soap, rinse with a damp cloth and dry completely;
- carefully clean the wheels with water and soap, rinse and dry completely, and check their efficiency.

Storage

Long term storage must be performed according to the following conditions:

- the article must be packed;
- the storage area must be dry and not exposed to direct sunlight;
- the article must not be stacked on top of each other.

Setting the item aside

In case the product needs to be stored for a long period of time, do not forget to:

- Place the product in a dry place away from direct sunlight;
- Protect it from dust by covering it with a nylon cloth;
- Grease the parts which could get oxidised or damaged in case of drying.

Disposal

If you decide to no longer use the article, it is necessary to declare it inoperable. We also recommend that any dangerous part which could cause damage are made safe.

Evaluate the class of the object according to the degree of disposal.

Dispose together with iron material and send it to the appropriate waste disposal centres.

If you consider the item special waste, disassemble it and divide into similar size pieces, then dispose of it according to the legislation in force.

Certificate of warranty

This certificate must be kept until its expiry date.

It must be shown every time technical assistance is deemed necessary, along with the invoice, receipt, delivery note and document containing the name of the dealer who sold the product.

User unable to show the proper documentation will no longer have the rights to the warranty.

The warranty commences from the date of purchase as shown on the invoice, receipt or shipping documents, and is valid for 24 months concerning the electric and moving parts, for 36 months concerning the structure of the article.

The warranty covers free replacement or repair, within the established terms, of those components that, at the manufacturer's unquestionable judgement, are found to have been defective at origination due to faulty manufacturing.

This warranty does not cover:

- damage due to transportation (scratches, dents and similar);
- damages due to falls;
- damages due to carelessness, negligence, mishandling, incapability of using the equipment, or repairs made by unauthorised personnel;
- damages due to incorrect installation of the equipment, in the event that it is directly installed by the user or by unauthorised personnel;
- damages due to a poor or inadequate electrical system or to changes due to environmental or climatic conditions, as well as damages caused by circumstances of misuse of the equipment;
- technical assistance related to the installation of equipment and its connection to the power supply systems;
- Assistance for routine service check-ups or to check for possible defects;
- ordinary maintenance;
- ordinary wear and tear;
- the replacement of the equipment is in any case excluded.

After-sales service in the warranty period can also be refused in the event that the equipment has been modified or transformed in any way.

The equipment is to be repaired by authorised technicians after the user has made a request to the retailer or directly to the manufacturer, and within time limits compatible with organisational requirements.

In the event of agreed upon assistance, the user will pay the fixed fee for transportation expenses. If the equipment is repaired at one of authorised Manufacturer's After-Sale Technical Assistance Centres, the user will pay charges and transportation risks.

Repairs made in the warranty period do not imply extensions or renewals of warranty coverage.

The parts replaced during the warranty period remain the Manufacturer's property.

No one can modify warranty terms and conditions or issue other verbal or written warranties.

The Manufacturer declines all responsibility for any damages, whatever their nature, that may directly or indirectly occur to people, animals, or objects in case of improper use of the equipment or if the instructions contained in this manual - mainly those related to equipment installation, use and maintenance - have not been followed.

Moreover, the manufacturer cannot be held responsible for any damages caused to people or objects due to failure or forced halt to the equipment in operation..

Any possible controversy is to be subject to the territorial competence of the court of Padua.

Note:

V.le Veneto, 2 - 35020 SAONARA Z.A.I. - PADOVA ITALIA
Tel. +39 049 8790199 Tel. +39 049 8790710
Fax. +39 049 08790711 e-mail: info@givas.it
www.givas.it